

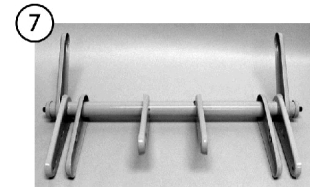
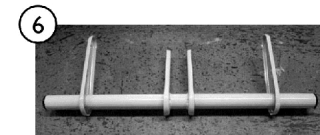
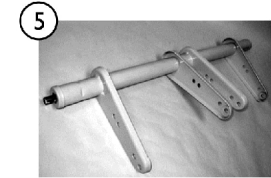
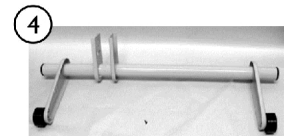
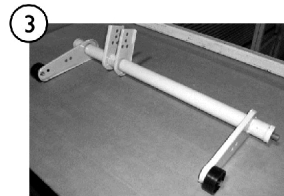
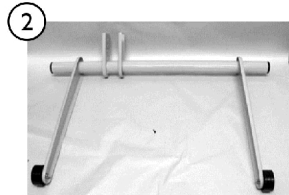
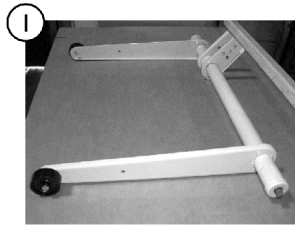
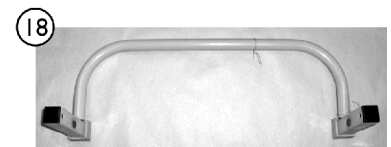
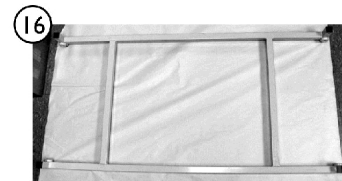
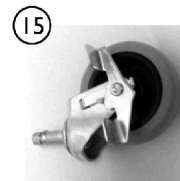
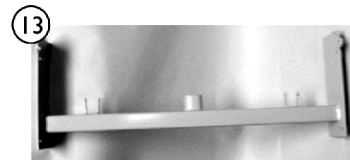
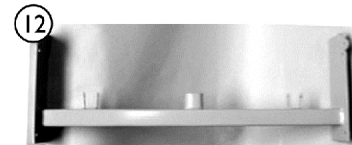
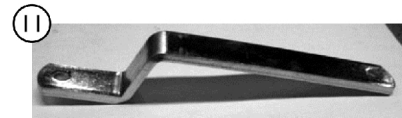


Steel and Frame Parts - BD0880/76

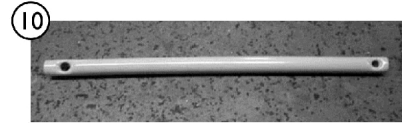

SICC_120A

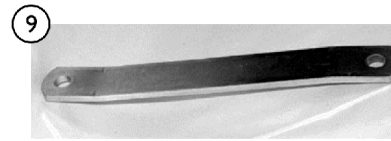

Steel and Frame Parts - BD0880/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	GATHEADLHA SY-NLA	NLA - Head Gatch (Pushpin on end of round tube)	1	1
2	B	1185149-NLA	NLA - Head Gatch Assembly (Black Insert w/ Steel Pin)	1	1
3	A	GATFOOTLHA SY-NLA	NLA - Foot Gatch (Pushpin on end of round tube)	1	1
4	B	GATFOOTASSY 1-NLA	NLA - Foot Gatch (Black Insert with Steel Pin)	1	1
5	A	HILOUPPERSE RV-NLA	NLA - Upper Gatch "4 Arm" (Push Pin on End of Round Tube)	1	1
6	C	HILOUPPERAS SY-NLA	NLA - Upper Gatch "4Arm" (Black Insert w/ Steel Pin)	1	1
7	B	HILOLOWERPN TS-NLA	NLA - Gatch, 8 Arm Lower (Pushpin on End of Round Tube)	1	1
8		TBR78164214P N-NLA	NLA - Tube, Hilo Drive (Circular)	1	1
9		LNK001OFF-NL A	NLA - Offset Bar	1	1
10		TBR78161812P N-NLA	NLA - Arm, Solo Control	1	1
11	C	LNK0013PNT-N LA	NLA - Linkage, Hilo Head/Foot	1	1
12		MNT0018CPNT- NLA	NLA - Plate, 76" Crank Mount	1	1
13	C	MNT0018BPNT	Plate, 80" Crank Mount	1	1
14		WHLEW1520R	Caster, Non-Locking (3")	1	1
15		WHLEW1520RR B	Caster, Locking	1	1
16		LWRFRASSY-N LA	NLA - Frame, Lower	1	1
17		LEG0004BASSY -NLA	NLA - Leg, Head End Hoop	1	1
18		LEG0003CASSY -NLA	NLA - Leg, Foot End Hoop	1	1

NOTE: A - Used prior to 8/2001
 B - Used After 8/2001
 C - Order through SFI Medical Solutions