

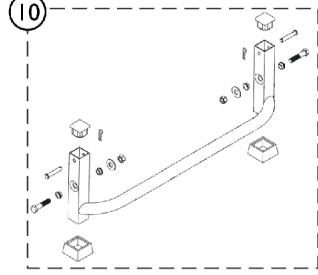
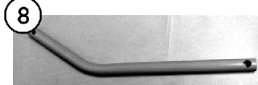
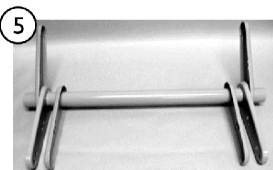
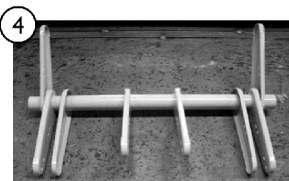
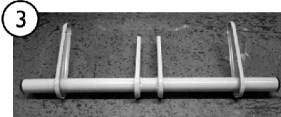
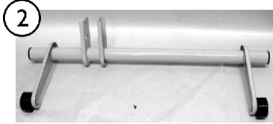
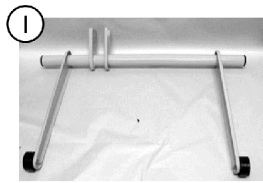
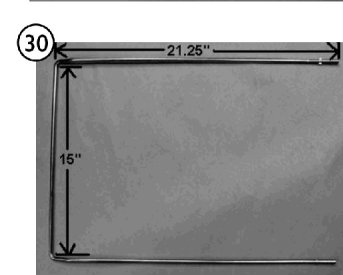
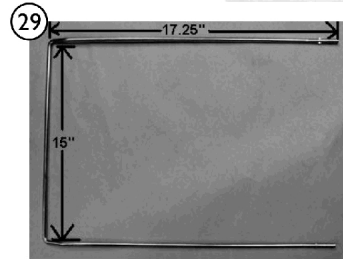
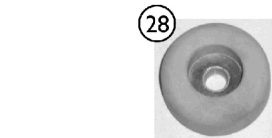
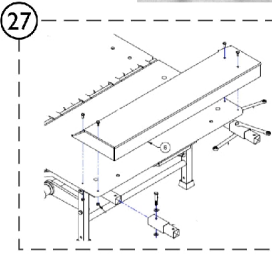
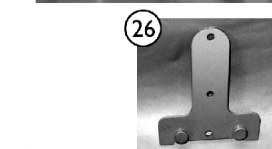
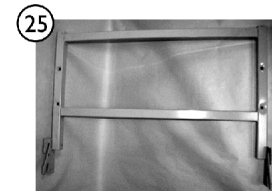
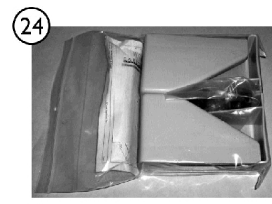
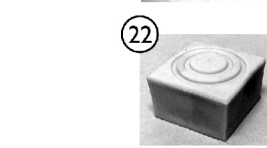
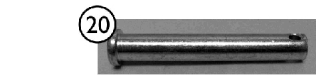
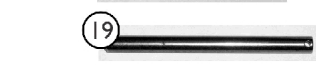
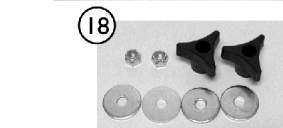
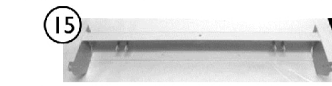
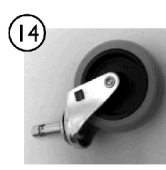
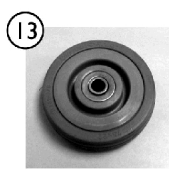


Steel and Frame Parts - BP1180/76

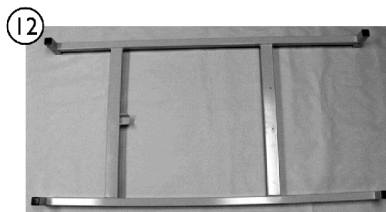

SICC_I24A

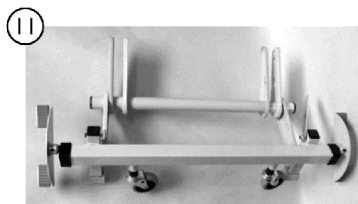

Steel and Frame Parts - BP1180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1185149-NLA	NLA - Head Gatch Assembly (Black Insert w/ Steel Pin)	1	1
2		GATFOOTASSY 1-NLA	NLA - Foot Gatch (Black Insert with Steel Pin)	1	1
3	A	HILOUPPERAS SY-NLA	NLA - Upper Gatch "4Arm" (Black Insert w/ Steel Pin)	1	1
4	A	HILOLOWERAS Y1-NLA	NLA - Gatch, Lower 8 Arm Head	1	1
5	A	HILOLOWERAS Y2-NLA	NLA - Gatch, Lower 6 Arm Foot	1	1
6		DRVTUBESOLO P1-NLA	NLA - HiLo Drive Tube	1	1
7		LNK0017PL-NLA	NLA - Linkage, Drive Tube	1	1
8		TBR781617131B -NLA	NLA - Solo Pro Control Arms	1	1
9		LNK0018PL-NLA	NLA - HiLo HD/FT Leg Linkage	1	1
10		LEG0004BASSY -NLA	NLA - Leg, Head End Hoop	1	1
11		LEG0003EASSY -NLA	NLA - Foot End Leg Assembly	1	1
12		LWRFRASSY2- NLA	NLA - Lower Frame	1	1
13		LEG0004BASY- NLA	NLA - Head End 75mm Fixed Caster (Non-Lock)	1	1
14		WHLEW1520R	Caster, Non-Locking (3")	1	1
15		MNT0018GPNT- NLA	NLA - Crank Mount Plate	1	1
16		1186525	Post, Head	1	1
17		1186524	Post, Foot	1	1
18		KIT78	Kit, Head & Foot Board Mounting Hardware	1	1
			Knob, Three Arm	1	2
19		PIN38412-NLA	NLA - Pin, 4-1/2" Straight	1	1
20		PIN380212	Clevis Pin (3/8 x 2-1/2" L)	1	1
21		PINBOW516-NL A	NLA - Pin, Cotter, Bowtie	1	1
22		GLDBEDFOOT	Foot, Bed - Rubber Santoprene	1	1
23		1186862	Plug, Under Bed Foot	1	1
24		MATTKPRKIT	Kit, Mattress Keeper, Left and Right Corners	1	1
			Mattress Keeper, Left Corner	1	2
			Mattress Keeper, Right Corner	1	2
25	B	IHECTA-NLA	NLA - Trapeze Adapter	1	1
26		FRENCHERMAL E-NLA	NLA - Plate, Frencher - Male	1	1
27	C	KIT19EARRO-N LA	NLA - Extension Kit - 4"	1	1
28	D	GLDBUMPER	Rubber Bed Foot & Wall Bumper (Optional, 2 per Wireform)	1	1
29		WBBSOLOPRO 76P-NLA	NLA - WireForm, 76"	1	1
30		WBBSOLOPRO 80P	NLA - Wireform. 80"	1	1

Steel and Frame Parts - BP1180/76

NOTE: A - Order through SFI Medical Solutions
B - Trapeze Adapter comes with Frame, Mounting Hardware, and 2 ea. Male Frencher Plates.
C - Does not come with Bolster Pad, must use 84" Mattress. Order through SFI Medical Solutions.
D - Optional (2 per Wireform).