
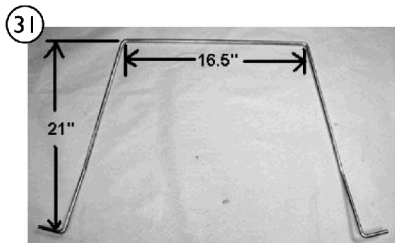
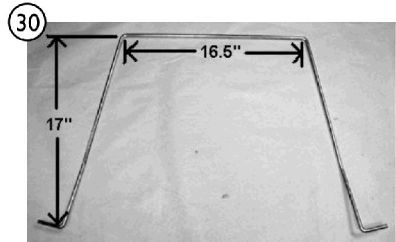
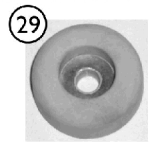
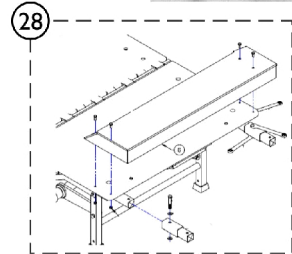
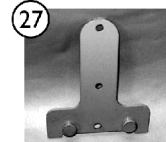
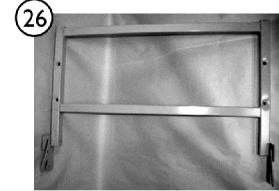
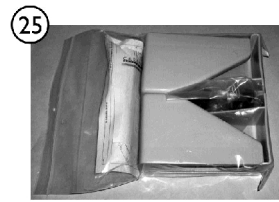
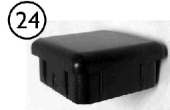
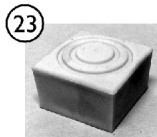
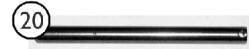
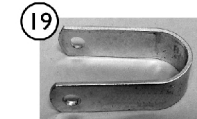
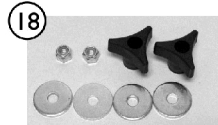
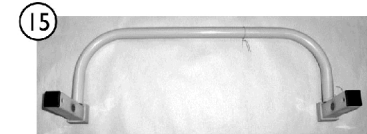
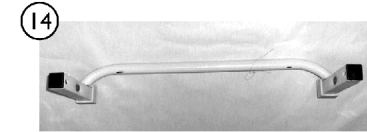
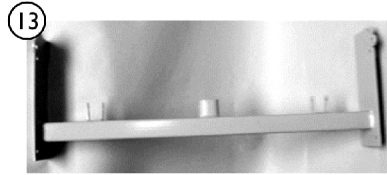
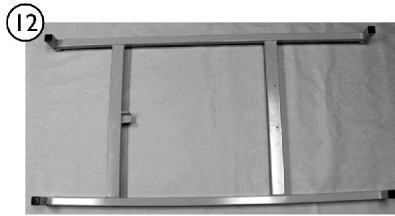
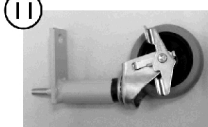
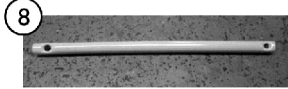
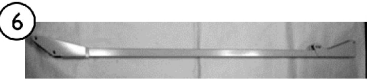
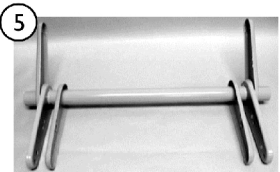
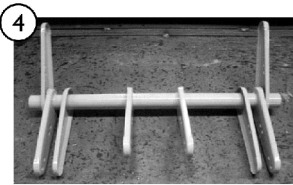
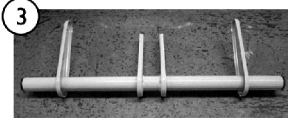
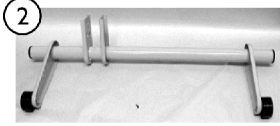


# Steel and Frame Parts - BS1180/76

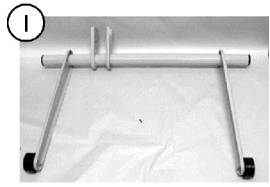

SICC\_I27A

## Steel and Frame Parts - BS1180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1185149-NLA	NLA - Head Gatch Assembly (Black Insert w/ Steel Pin)	1	1
2		GATFOOTASSY 1-NLA	NLA - Foot Gatch (Black Insert with Steel Pin)	1	1
3	A	HILOUPPERAS SY-NLA	NLA - Upper Gatch "4Arm" (Black Insert w/ Steel Pin)	1	1
4	A	HILOLOWERAS Y1-NLA	NLA - Gatch, Lower 8 Arm Head	1	1
5	A	HILOLOWERAS Y2-NLA	NLA - Gatch, Lower 6 Arm Foot	1	1
6		DRVTUBESOLO P-NLA	NLA - Tube, Hilo Drive	1	1
7		LNK0017PL-NLA	NLA - Linkage, Drive Tube	1	1
8		TBR78161812P N-NLA	NLA - Arm, Solo Control	1	1
9	A	LNK0013PNT-N LA	NLA - Linkage, Hilo Head/Foot	1	1
10	A	WHLBRACKASS Y3-NLA	NLA - Caster Assembly (Non-Lock)	1	1
11	A	WHLBRACKASS Y4-NLA	NLA - Caster Assembly, Lock	1	1
12		LWRFRASSY1- NLA	NLA - Frame, Lower	1	1
13	A	MNT0018BPNT	Plate, 80" Crank Mount	1	1
14		LEG0004BASSY -NLA	NLA - Leg, Head End Hoop	1	1
15		LEG0003CASSY -NLA	NLA - Leg, Foot End Hoop	1	1
16		1186525	Post, Head	1	1
17		1186524	Post, Foot	1	1
18		KIT78	Kit, Head & Foot Board Mounting Hardware Knob, Three Arm	1 1	1 2
19	A,B	MTRBRACKP1	Bracket, Motor Mount, Floating	1	1
20		PIN38412-NLA	NLA - Pin, 4-1/2" Straight	1	1
21		PIN380212	Clevis Pin (3/8 x 2-1/2" L)	1	1
22		PINBOW516-NL A	NLA - Pin, Cotter, Bowtie	1	1
23		GLDBEDFOOT	Foot, Bed - Rubber Santoprene	1	1
24		1186862	Plug, Under Bed Foot	1	1
25		MATTKPRKIT	Kit, Mattress Keeper, Left and Right Corners Mattress Keeper, Left Corner Mattress Keeper, Right Corner	1 1 1	1 2 2
26	C	IHECTA-NLA	NLA - Trapeze Adapter	1	1
27		FRENCHERMAL E-NLA	NLA - Plate, Frencher - Male	1	1
28	A,D	KIT19EARRO-N LA	NLA - Extension Kit - 4"	1	1
29	E	GLDBUMPER	Rubber Bed Foot & Wall Bumper (Optional, 2 per Wireform)	1	1
30		WBBSOLO76P- NLA	NLA - Wireform, 76"	1	1

## Steel and Frame Parts - BS1180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
31		WBBSOLO80P- NLA	NLA - Wireform, 80"	1	1
NOTE: A - Order through SFI Medical Solutions B - KIT85 Foot Motor Mount Bracket is No Longer Available. C - Trapeze Adapter comes with Frame, Mounting Hardware and 2 Male French Plates. D - Does not include Bolster Pad, must use 84" Mattress. E - Optional (2 per Wireform)					