
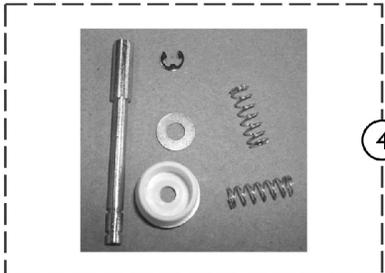


Mechanical Hardware Kits - IH6074 Series

Models 6074A

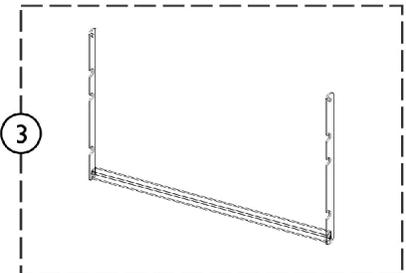

Mechanical Hardware Kits - IH6074 Series Models 6074A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1126608	Bar, Push, Back	1	1
2	B	1126609	Reclining Mechanism Assembly w/ Hardware	1	1
3		1163960	Ratchet Assembly, Grey - 6065A, 6074A	1	1
4	D	1163840	Pin, Plunger w/ Hardware	1	1
5	C	1126607-NLA	NLA - Kit, Hardware	1	1

NOTE: A - Includes (2) Hex Head Screws (M6 x 55).
 B - Kit includes: (2) Reclining Mechanisms, (2) Finger Guards, (10) Phillips Head Screws (1/4-20 x 1) and (2) Hex Nuts (1/4-20).
 C - Kit includes: (2) Phillips Head Screws (M6 x 35) with (2) Washers (6mm) and (2) Hex Head Bolts (M6.35 x 50) with (2) Washers (6.35mm).
 D - Kit includes Plunger Pin (1), Plunger Cap (1), Spring (2) and Washer (1).

The Hardware Kit (Item 5) is used to attach the Plastic Skirt to the Chair Frame.