
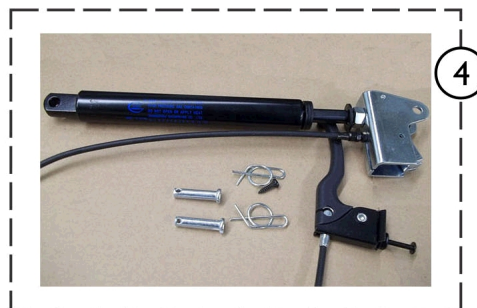
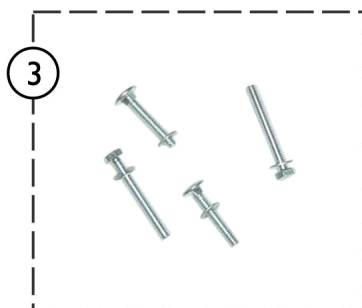


# Mechanical Hardware Kits - IH6077 Series

## Models 6077A

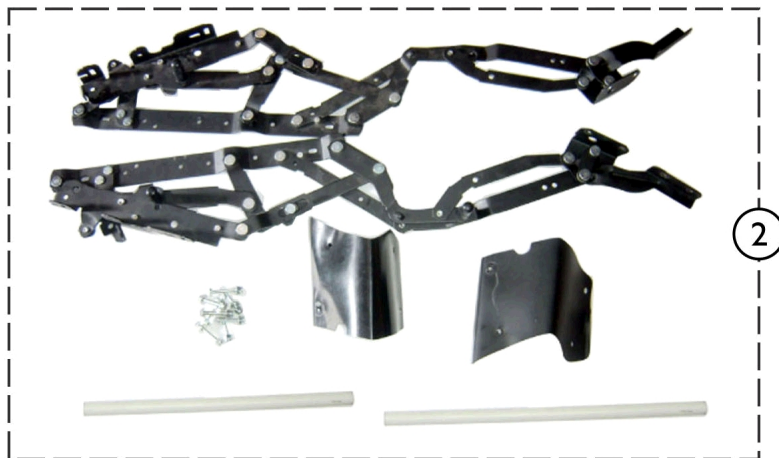

SPT\_030A

## Mechanical Hardware Kits - IH6077 Series

### Models 6077A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1126730-NLA	NLA - Bar, Push, Back	1	1
2	B	1126729	Kit, Recliner Mechanism with Finger Guards	1	1
3	C	1126607-NLA	NLA - Kit, Hardware	1	1
4	D,H	1126731	Kit, Gas Spring w/ Handle	1	1
<p>NOTE: A - Includes: Push Bar with Screws and Back.</p> <p>B - Kit includes: (2) Recliner Mechanism with Finger Guards and (10) Phillips Head Screws (1/4-20 x 1) and (2) Hex Nuts (1/4-20).</p> <p>C - Kit includes: (2) Phillips Head Screws (M6 x 35), (2) Washers (6mm), (2) Hex Head Bolts (M6.35 x 50) and (2) Washers (6.35mm).</p> <p>D - Kit includes: Gas Spring with Handle, Cable Bracket and Lock Pin.</p> <p>H - Part is no longer available for recliners built before 12/1/08. For use on recliners built after 11/30/08 only.</p> <p>The Hardware Kit (Item 3) is used to attach the Plastic Skirt to the Chair Frame.</p>					