



Electronics and Pendants - BE1180/76-BE2180/76



1 Shaft Approx. 3 Ft. - Red Plug

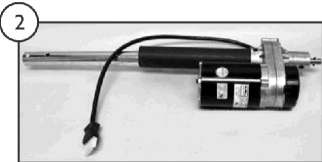


2 Shaft Approx. 1 Ft. - Blue Plug

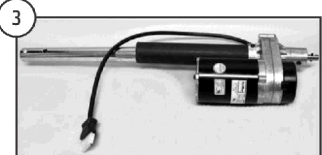


3 Shaft Approx. 4-1/2 Ft. - White Plug





5 SICC_100A

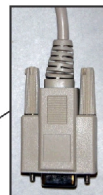


6 DB9 Connector

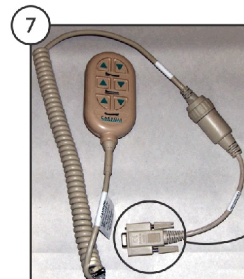





7 DB9 Connector

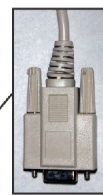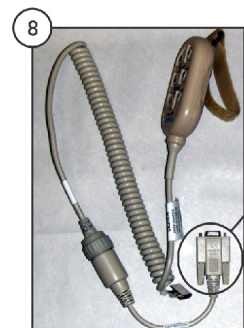


8 DB9 Connector

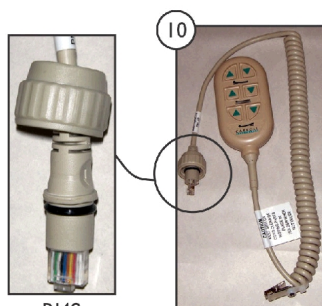


10 RJ48 Connector

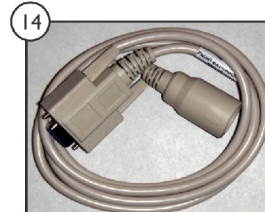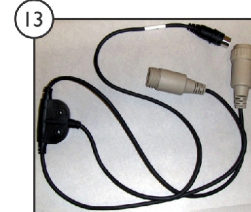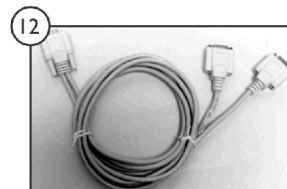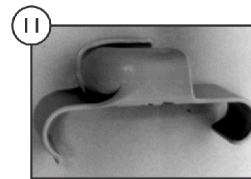

15 Old Keypad

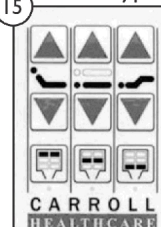

16 New Keypad

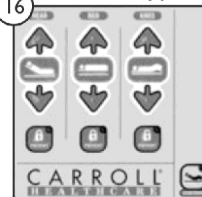

Electronics and Pendants - BE1180/76-BE2180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	MTRM2103	Actuator, Head (Red Plug) 115V ~ 60Hz	1	1
2	B	MTRM2102-NLA	NLA - Actuator, Foot (Blue Plug) 115V ~ 60Hz	1	1
3	C	MTRM2150	Actuator, HILO (White Plug) 115V ~ 60Hz	1	1
4	D	JNCTNBOX-NLA	NLA - Junction Control Box	1	1
5	E	JNCTNBOXMEM -NLA	NLA - Junction Box Membrane	1	1
6	F	PNDNT2FNCIV	Pendant, 2 Function	1	1
7	F	1190227	Pendant, Six Function (w/ Loop Adhesive Back Strip and DB9 Connector) Linak # PNDNT-6FNC-1-2	1	1
8	F	1190232	Pendant, Six Function (w/ Strap and DB9 Connector) Linak # PNDNT-6FNC-1-2	1	1
9	F	1190225	Cable, Conversion	1	1
10	G	PNDNT6FNCIV3	Pendant, Six Function w/ RJ48 Connector) Linak # PNDNT-6FNC-1-2	1	1
11		1186847	Holster, Pendant	1	1
12	F	PNDNTYCABLE	Cable, Y, Pendant	1	1
13	G,H	JNCTNDC40	Cable, Y, Pendant	1	1
14	G,H	PNDNTEXTCABLE	Cable, Pendant	1	1
15		OLD KEYPAD-NLA	NLA - Membrane Pad for 151 Only	1	1
16	I	JNCTNDC20C	Membrane Pad for 151 Only	1	1

NOTE: A - Reference #61833-1253270-01
 B - Reference #61833-0703270-01
 C - Reference #61833-1955270-02. Beds manufactured between 8/03/04-10/02/07 requires TBR118164112P (Order through SFI Medical Equipment Solutions).
 D - Reference #NAX5105-3A29MPZ
 E - For use on Membrane Beds only After 8/16/05
 F - Beds manufactured Before 11/22/07
 G - Beds manufactured After 11/21/07
 H - Item numbers 13 and 14 are required to be used together.
 I - Beds built Before 8/16/04 need to replace Head and Footboard with New Membrane Pad.