


Electronics and Pendants - BD2380/76


NOTE: This Bed is obsolete and only parts listed are currently available.




1 Shaft Approx. 3 Ft. - Red Plug

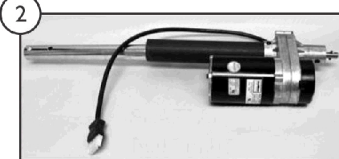


2 Shaft Approx. 1 Ft. - Blue Plug

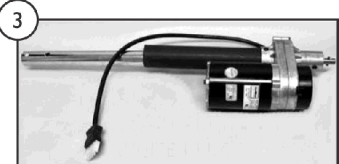


3 Shaft Approx. 1-1/2 Ft. - White Plug

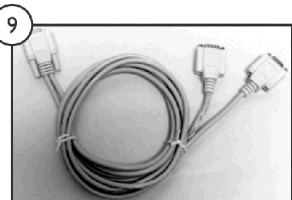


10 SICC_099A

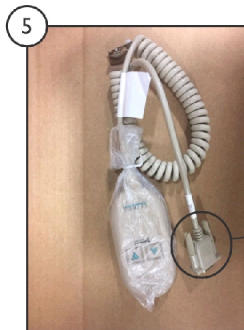


5 DB9 Connector

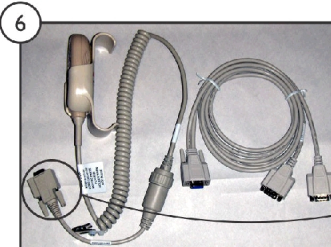



6 DB9 Connector

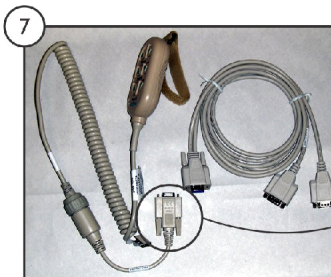

7 DB9 Connector

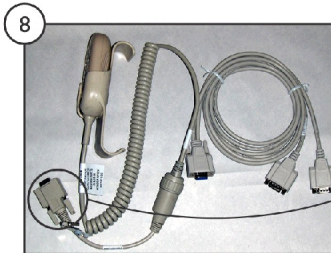

8 DB9 Connector

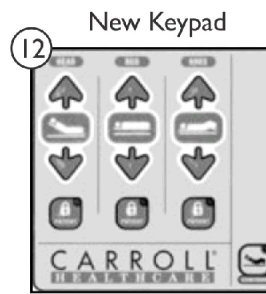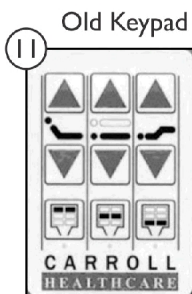

Electronics and Pendants - BD2380/76

NOTE: This Bed is obsolete and only parts listed are currently available.

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | MTRM1703 | Head Actuator (Red Plug) | 1 | 1 |
| 2 | B | MTRM1702 | Foot Actuator (Blue Plug) | 1 | 1 |
| 3 | D | MTRM2150 | Actuator, HILO (White Plug) 115V ~ 60Hz | 1 | 1 |
| | E | KIT82-NLA | NLA - HiLo Actuator (White Plug) (Includes Jagaer Hilo Motor and Drive Tube w/ Hardware) | 1 | 1 |
| 4 | F | JNCTNBOX-NLA | NLA - Junction Control Box | 1 | 1 |
| 5 | | PNDNT2FNCIV | Pendant, 2 Function | 1 | 1 |
| 6 | C | 1190228 | Kit, Pendant, New Assist Rails, Linak # PNDNT-6FNC-1-2 | 1 | 1 |
| 7 | C | 1190233 | Kit, Pendant, Full/Partial Rails, Linak # PNDNT-6FNC-1-2 (With DB9 Connector) | 1 | 1 |
| 8 | C | PNDNTKIT3-NL A | NLA - Kit, Pendant, Assist Rails (Old Square), Linak # PNDNT-6FNC-1-2 (With DB9 Connector) (May 01 - July 02) | 1 | 1 |
| 9 | | PNDNTYCABLE | Cable, Y, Pendant | 1 | 1 |
| 10 | | 1190225 | Cable, Conversion | 1 | 1 |
| 11 | | OLD KEYPAD-NLA | NLA - Membrane Pad for 151 Only | 1 | 1 |
| 12 | G | JNCTNDC20C | Membrane Pad for 151 Only | 1 | 1 |

NOTE: A - Reference #V07508AW76U
 B - Reference #V07508AX76U
 C - Reference Linak #PNDNT-6FNC-1-2
 D - Used After 4/19/00. Must use second hole on shaft.
 E - Used Before 4/20/00 White Plug. Ref. 6183M-1905220-01
 F - Used After 11/29/01. Ref. NAX5105-3A29MPZ
 G - Beds built Before 8/16/04 need to replace Head and Footboards with New Membrane Pad.