

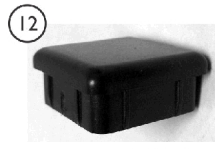
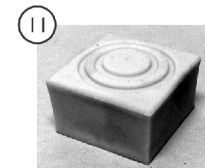
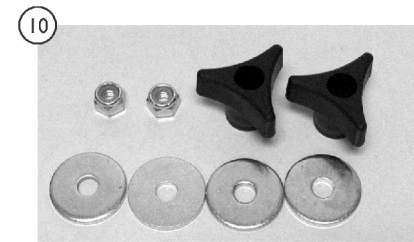
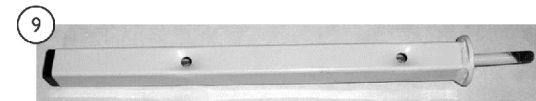
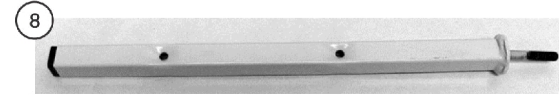
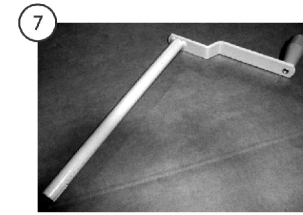
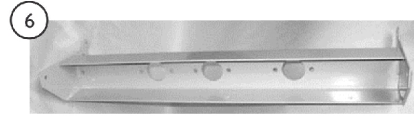
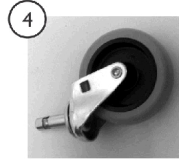
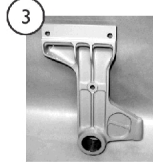
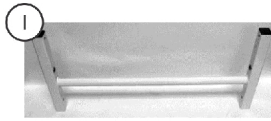
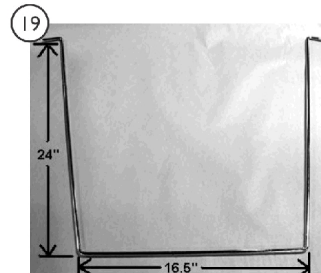
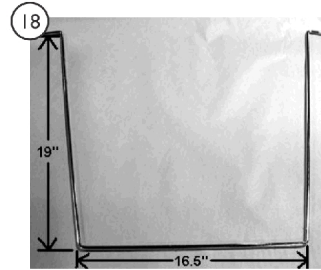
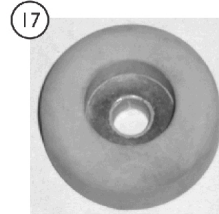
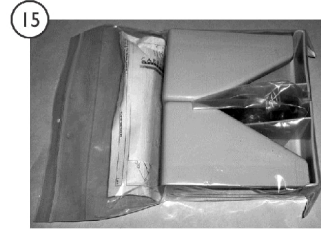


Steel and Frame Parts - BC1180/76 - (continued)

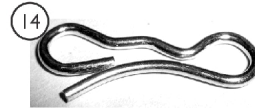

SICC_112A

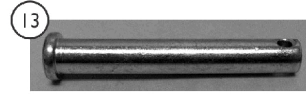

Steel and Frame Parts - BC1180/76 - (continued)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	LEG0005PNT-N LA	NLA - HILO Leg	1	1
2		LEG0014LPNT- NLA	NLA - Leg, Die Cast - Left Side	1	1
3		LEG0014RPNT- NLA	NLA - Die Cast Leg - Right Side	1	1
4		WHLEW1520R	Caster, Non-Locking (3")	1	1
5		WHLEW1520RR B	Caster, Locking	1	1
6	A	MNT0018PNT	Crank Mount Plate, Painted	1	1
7		CRKMTRHILOA SY-NLA	NLA - Handle, Crank - Emergency	1	1
8		1186525	Post, Head	1	1
9		1186524	Post, Foot	1	1
10		KIT78	Kit, Head & Foot Board Mounting Hardware	1	1
			Knob, Three Arm	1	2
11		GLDBEDFOOT	Foot, Bed - Rubber Santoprene	1	1
12		1186862	Plug, Under Bed Foot	1	1
13		PIN380212	Clevis Pin (3/8 x 2-1/2" L)	1	1
14		PINBOW516-NL A	NLA - Pin, Cotter, Bowtie	1	1
15		MATTKPRKIT	Kit, Mattress Keeper, Left and Right Corners	1	1
			Mattress Keeper, Left Corner	1	2
			Mattress Keeper, Right Corner	1	2
16		KIT19EECHOE- NLA	NLA - Kit, Extension - Electric (4") (Includes one Emergency Crank Handle. Must order DECK00-05K-IT or Kit will not work)	1	1
17	B	GLDBUMPER	Rubber Bed Foot & Wall Bumper (Optional, 2 per Wireform)	1	2
18		WB00076ECHO P-NLA	NLA - Wireform Wall Bumper, 76"	1	1
19		WBB80ECHOP- NLA	NLA - Wire Form Wall Bumper, 80"	1	1

NOTE: A - Order through SFI Medical Solutions.
B - Rubber Bumper for Wireforms are optional. (2 per Wireform).