
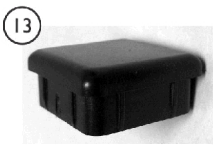
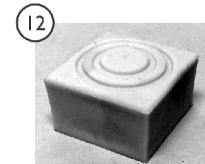
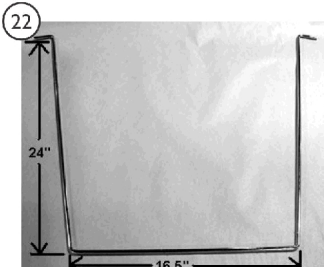
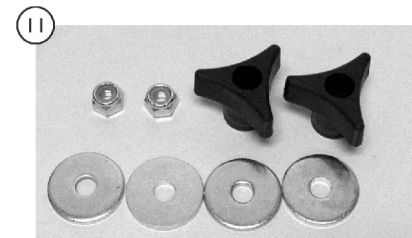
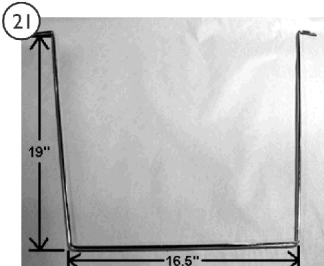
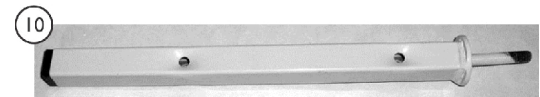
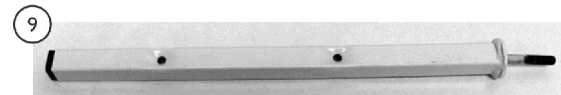
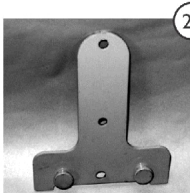
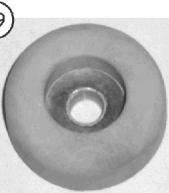
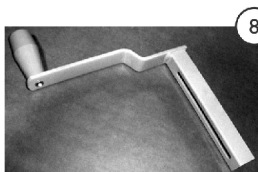
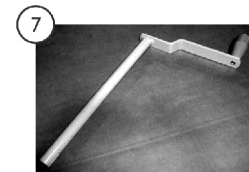
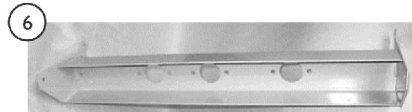
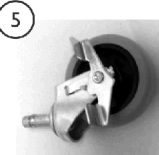
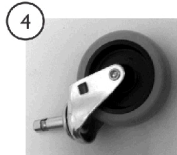
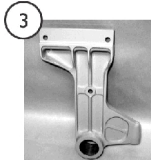


Steel and Frame Parts - BD2380/76 - (continue)

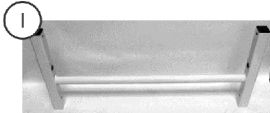

SICC_113A

Steel and Frame Parts - BD2380/76 - (continue)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	LEG0005ASSY-NLA	NLA - Hilo Leg ("H" Leg)	1	1
2	A	LEG0014LPNT-NLA	NLA - Leg, Die Cast - Left Side	1	1
3	A	LEG0014RPNT-NLA	NLA - Die Cast Leg - Right Side	1	1
4		WHLEW1520RR B	Caster, Locking	1	1
5		WHLEW1520R	Caster, Non-Locking (3")	1	1
6	B	MNT0018PNT	Crank Mount Plate, Painted	1	1
7		CRKMTRHILOASY-NLA	NLA - Handle, Crank - Emergency	1	1
8		1186516	Crank Handle	1	1
9		1186525	Post, Head	1	1
10		1186524	Post, Foot	1	1
11		KIT78	Kit, Head & Foot Board Mounting Hardware	1	1
			Knob, Three Arm	1	2
12		GLDBEDFOOT	Foot, Bed - Rubber Santoprene	1	1
13		1186862	Plug, Under Bed Foot	1	1
14		PIN380212	Clevis Pin (3/8 x 2-1/2" L)	1	1
15		PINBOW516-NLA	NLA - Pin, Cotter, Bowtie	1	1
16		MATTKPRKIT	Kit, Mattress Keeper, Left and Right Corners	1	1
			Mattress Keeper, Left Corner	1	2
			Mattress Keeper, Right Corner	1	2
17	C	IHECTA-NLA	NLA - Trapeze Adapter	1	1
18	D	KIT19EECHOM-NLA	NLA - Kit, Extension - Manual (4") (Includes three Extension Crank Handles. Must order DECK00-05K-IT or Kit will not work.)	1	1
18	E	KIT19EECHOE-NLA	NLA - Kit, Extension - Electric (4") (Includes one Emergency Crank Handle. Must order DECK00-05K-IT or Kit will not work)	1	1
19	F	GLDBUMPER	Rubber Bed Foot & Wall Bumper (Optional, 2 per Wireform)	1	1
20		FRENCHERMAL E-NLA	NLA - Plate, Frencher - Male	1	1
21		WBB00076ECHOP-NLA	NLA - Wireform, 76"	1	1
22		WBB80ECHOP-NLA	NLA - Wire Form Wall Bumper, 80"	1	1

NOTE: A - After 4/20/00
 B - Order through SFI Medical Solutions.
 C - Includes Adapter, Frame, Mounting Hardware and 2 Male Frencher Plates.
 D - Includes 3 Extension Crank Handles. Must also order DECK00-05K-IT or kit will not work.
 E - Includes 1 Extension Override Crank Handle. Must also order DECK00-05K-IT or kit will not work.
 F - Rubber Bumper for Wireforms are optional (2 per Wireform).