


Legrest Support Assembly

T94H Series Before 5/16/19

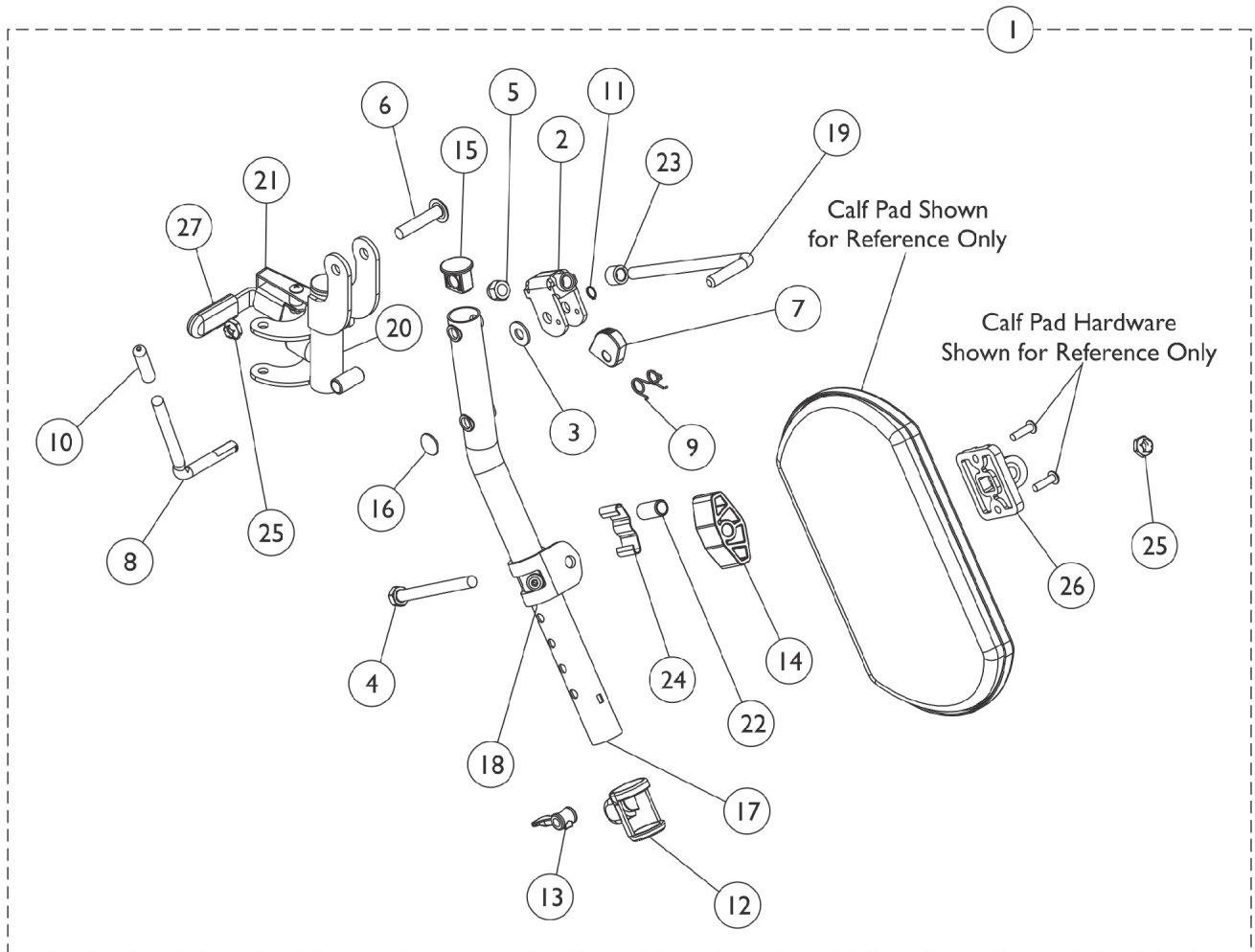

Legrest Support Assembly T94H Series Before 5/16/19

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,H	1110338	Legrest Support Assembly, Hemi, Black - Right	1	1
1	A,H	1110339	Legrest Support Assembly, Hemi, Black - Left	1	1
1	A	1110336	Legrest Support Assembly, Hemi, Black - Right	1	1
1	A	1110337	Legrest Support Assembly, Hemi, Black - Left	1	1
1	A,F	8881180761	Legrest Support Assembly, Hemi, Color - Right	1	1
1	A,F	8881180762	Legrest Support Assembly, Hemi, Color - Left	1	1
2		1110308	Bracket, Release, Black	1	1
3		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
4		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
5		10261X000	Nut, Acorn (5/16-24)	1	1
6		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
7		1037125	Cam, Legrest	1	1
8	G	1037126	Handle, Release	1	1
9		1037485	Spring, Release	1	1
10		1040060	Tip, Plastic, Black	1	1
11		1040831	Ring, Snap	1	1
12		1048085	Housing, Cam	1	1
13		1048086	Lever, Cam, Black	1	1
14		1091220	Spacer, Legrest	1	1
15		1095433	Plug Button, Black (7/8")	1	1
16		1096773	Nut, Push Dome (5/16")	1	1
17	D,G	1110165-NLA	NLA - Tube, Elevating Support, Chrome - Right	1	1
17	G	1110166-NLA	NLA - Tube, Elevating Support, Chrome - Left	1	1
17		1110163	Tube, Elevating Support, Black - Right	1	1
17	D	1110164-NLA	NLA - Tube, Elevating Support, Black - Left	1	1
18		1110208-NLA	NLA - Bracket, Calf Pad Mounting, Chrome	1	1
18		1110207	Bracket, Calf Pad Mounting, Black	1	1
19		42326X001	Bar, Ratchet	1	1
20	B,E,G	1110334-NLA	NLA - Legrest Support Tube Assembly, Hemi, Black - Right	1	1
20	B,E,G	1110335	Legrest Support Tube Assembly, Hemi, Black - Left	1	1
20	B,D,E	1110332-NLA	NLA - Legrest Support Tube Assembly, Hemi, Black - Right	1	1
20	B,E	1110333	Legrest Support Tube Assembly, Hemi, Black - Left	1	1
21	C	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
21	C	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
22		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
23		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
24		48823X008	Spring, Leaf	1	1
25		60264X001	Locknut (5/16-24)	1	2
26		04526X019	Plate, Legrest Pad	1	1
27		1006490	Tip, Plastic, Black	1	1
27		1127688	Package, Plastic Tip, Black	10	1

Legrest Support Assembly

T94H Series Before 5/16/19

NOTE: A - Includes items 2-27.
B - Assembly includes: Item #27, tip, bumper, screws, plug, spring, nuts, handle release, legrest support and bushing.
C - Order this item when ordering chair with Cyclical Drive only.
D - No longer available as a service part individually. Order item #1 as a replacement.
E - Includes item 21.
F - Specify 71P Silver Vein frame finish when ordering.
G - Item #8, 17 and 20 are not available for sale in color. Order item #1 as a replacement.
H - After 10/14/14 color changed from Chrome to Black. It is recommended to order both the right and left side to match.