


Wheels - Rear - Supports - Twion

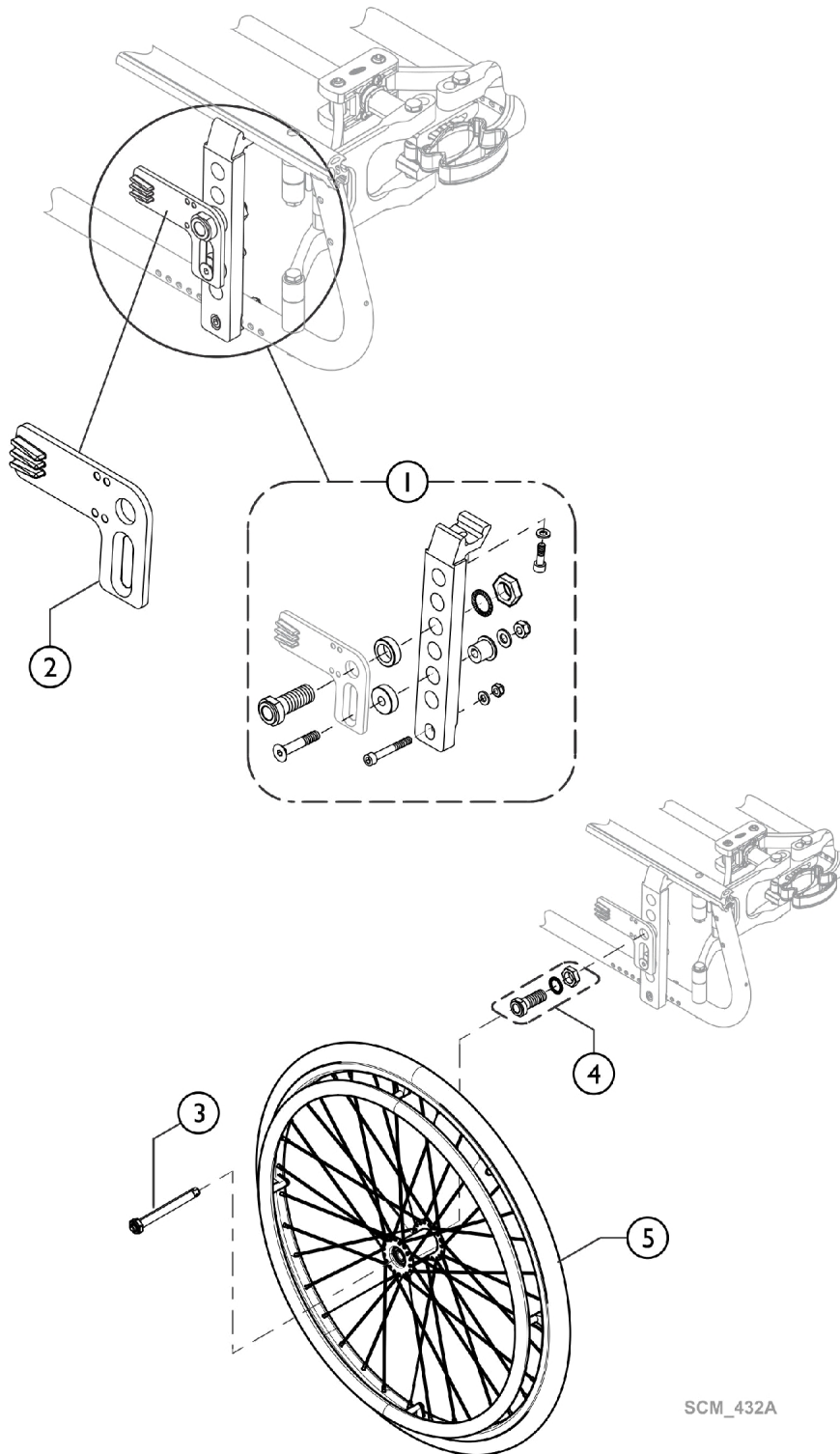

SCM_432A

Wheels - Rear - Supports - Twion

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP1566599	Kit, Twion, Bracket Set (Pair)	2	1
2		SP1565787	Kit, Torque receiver twion with angle (Pair)	2	1
3		SP1456261	Stub Axle 1/2, Le=104.05 Mm L=123.00 Mm	1	1
4		SP1463606	Kit, Axle Receiver 1/2. Length=50.8Mm Incl.	1	1
5		SP1641140	Rear Wheel 24" for 12.7 Mm Axle, Twion	1	1