
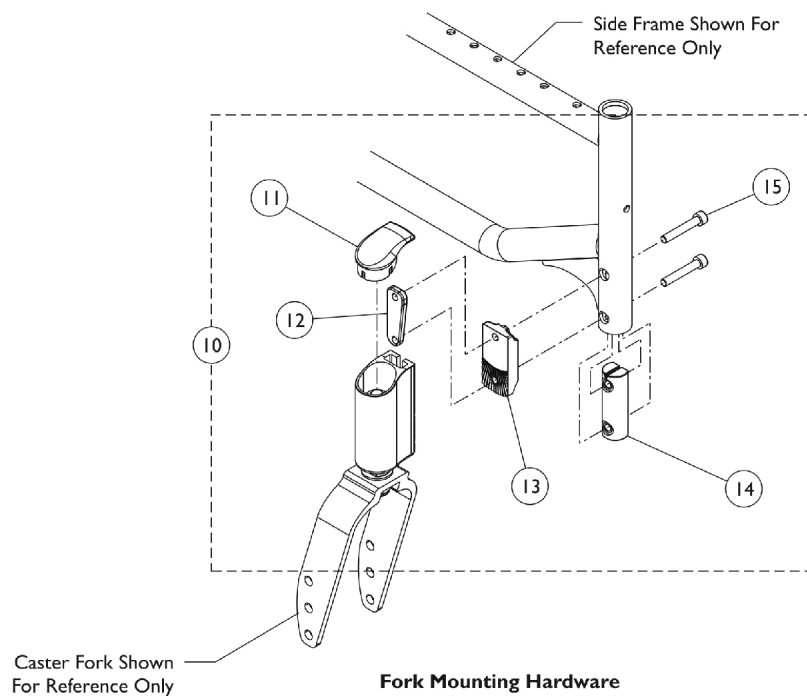


Casters - Caster, Headtube and Fork w/ Mtg. Hdwr.

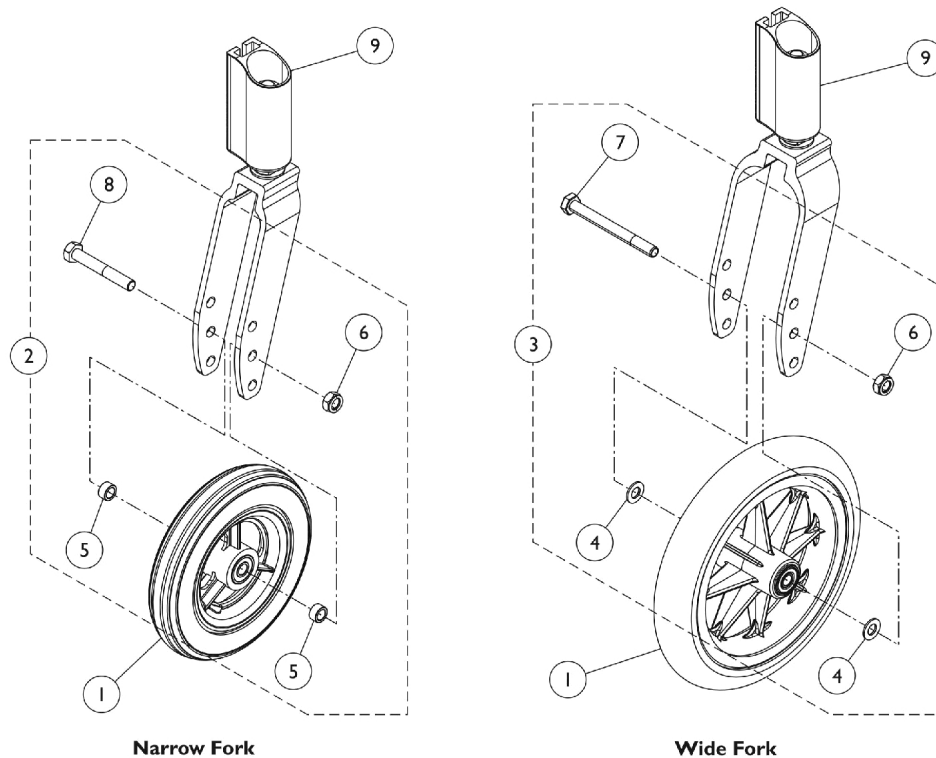

Casters - Caster, Headtube and Fork w/ Mtg. Hdwr.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Casters	1	1
2		1169075	Kit, Caster Attaching Hardware, Narrow Fork (Pair)	1	1
5			Spacer (5/16 x 15/32 x 1/4")	1	4
6			NLA - Locknut (M8 x 1.25)	1	2
8	B	1169076	Bolt, Cap, Hex Head (M8 x 55mm)	1	2
3			Kit, Hardware, Caster Attaching, Wide Fork (Pair)	1	1
4			Washer, Flat (5/16 x 5/8 x 1/16")	1	4
6			NLA - Locknut (M8 x 1.25)	1	2
7	C,D	See Chart	NLA - Screw, Hex Head (M8 x 1.25 x 80mm)	1	2
9			#9 Headtube and Fork Assembly	1	1
10			Kit, Headtube Attaching Hardware (Pair)	1	1
11			Cap, Head Tube, Caster Housing	1	2
12			Plate, Mounting Tube Head	1	2
13			Plate, Mounting Caster, Inner	1	2
14			Insert, Support Fork	1	2
15			Screw, (M6 x 1 x 37mm)	1	4
11		1540575	Cap, Head Tube, Caster Housing	1	1
NOTE: A - Includes items 5, 6, and 8. B - Includes items 4, 6, and 7. C - Includes items 11 - 15. D - Please review Owners Manual for specific installation instructions before replacing these parts on chair.					

#1 Casters

Caster Type	Wheel/ Tire Type	Tire Size	Used with Narrow Fork	Used with Wide Fork
5ALUM	Aluminum/ Urethane	5" x 1"	1026952	N/A
AT901	Composite/ Urethane	5" x 1"	1029496	N/A
AT902	Composite/ Urethane	6" x 1"	1034352 See Note(s) A	N/A
6PNEU	Composite/ Pneumatic	6" x 1-1/4"	1044040	N/A
AT903	Composite/ Urethane	8" x 1-1/4"	N/A	1166026
AT1292	Composite/ Pneumatic	8" x 1-1/4"	N/A	1166028
1492	Pneumatic Flat Free	8" x 1-1/4"	N/A	1166027
NOTE: A - Consists of (2) Casters.				

#9 Headtube and Fork Assembly

Casters - Caster, Headtube and Fork w/ Mtg. Hdwr.

#9 Headtube and Fork Assembly

Seat to Floor Height	5ALUM AT901 AT5SOFT (Narrow Fork)	AT902 6PNUE (Narrow Fork)	AT903 AT1292 1492 (Wide Fork)
FSTF14 (14" H)	1542007	N/A	N/A
FSTF15 - 16 (15 - 16" H)	1488399	N/A	N/A
FSTF16 - 19 (16 - 19" H)	N/A	1538090	N/A
FSTF18 - 19 (18 - 19" H)	N/A	N/A	1449113 See Note(s) A
FSTF18 - 20 (18 - 20" H)	N/A	N/A	1449113
NOTE: A - Used on AT903 only.			