


Casters - Caster, Headtube and Fork w/ Mtg. Hdwr.

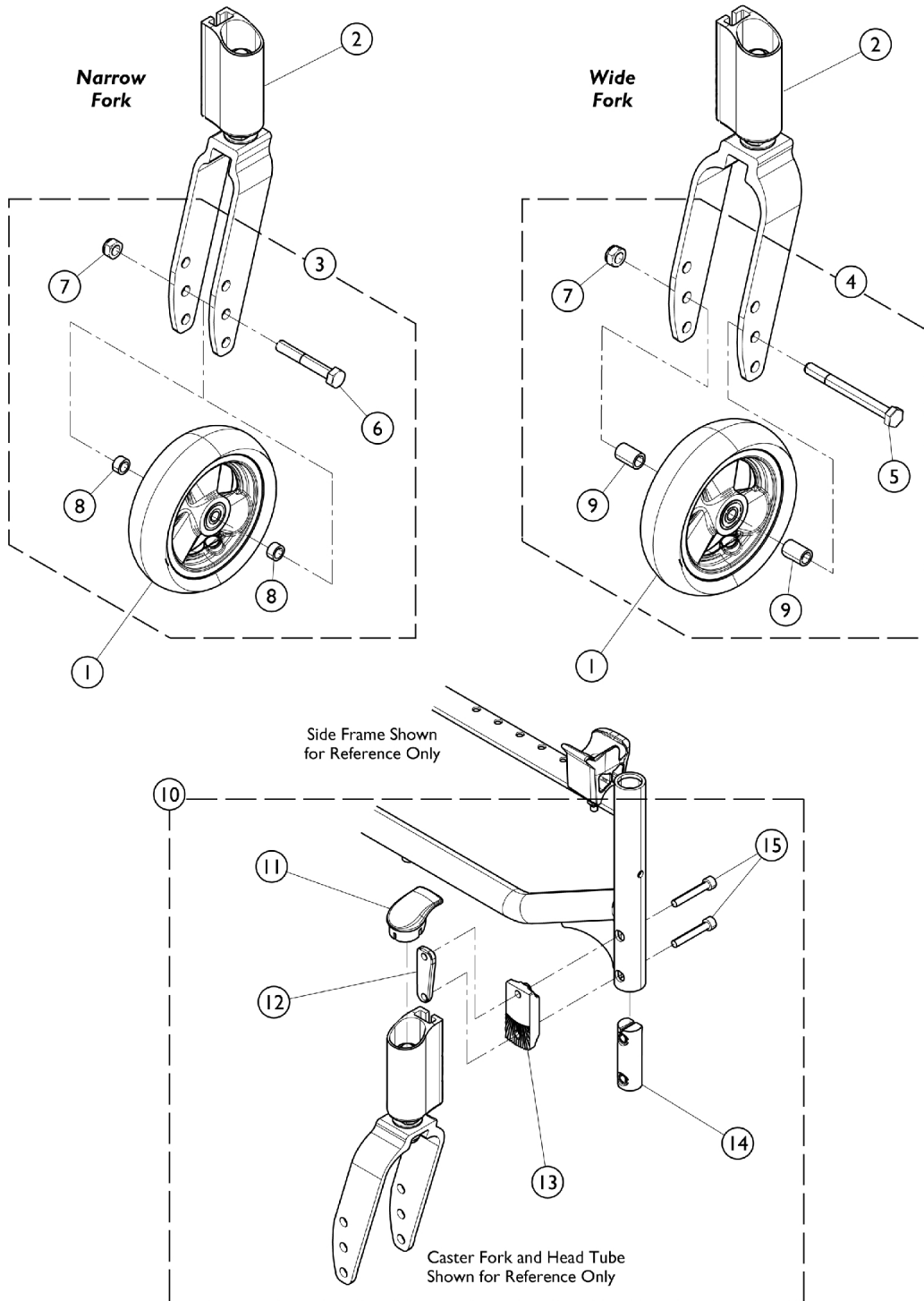

Casters - Caster, Headtube and Fork w/ Mtg. Hdwr.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Casters	1	1
2		See Chart	#2 Headtube Forks w/ Hardware	1	1
3	A	1169075	Kit, Caster Attaching Hardware, Narrow Fork (Pair)	1	1
6			Bolt, Cap, Hex Head (M8 x 55mm)	1	2
7			NLA - Locknut (M8 x 1.25)	1	2
8			Spacer (5/16 x 15/32 x 1/4")	1	4
4	B	1173020	Kit, Caster Attaching Hardware, Wide (Pair)	1	1
5			NLA - Screw, Hex Head (M8 x 1.25 x 80mm)	1	2
7			NLA - Locknut (M8 x 1.25)	1	2
9			Spacer, Caster	1	4
10	C,E	1169073	Kit, Headtube Attaching Hardware (Pair)	1	1
11			Cap, Head Tube, Caster Housing	1	2
12			Plate, Mounting Tube Head	1	2
13			Plate, Mounting Caster, Inner	1	2
14			Insert, Support Fork	1	2
15			Screw, (M6 x 1 x 37mm)	1	4
10	D	1172240	Kit, Headtube Attachment Hardware (TRBKTS/TRRO)	1	1
11			Cap, Head Tube, Caster Housing	1	2
12			Plate, Mounting Tube Head	1	2
13			Plate, Mounting Caster, Inner	1	2
14			Insert, Support Fork	1	2

NOTE: A - Includes items 6, 7, and 8.
 B - Includes items 5, 7, and 9.
 C - Includes items 11 - 15.
 D - Includes items 11 - 14.
 E - Please review your Owners Manual for Torque Specification and Torque Label when replacing parts from this kit.

#1 Casters

Caster Type	Wheel/Tire Type	Tire Size	Part No.	Mounting Hardware
5ALUM	Aluminum Urethane	5"	1026952	1169075
AT901	Composite Urethane	5"	1029496	1169075
AT5SOFT	Soft Roll	5 x 1-1/2"	1157706	1173020
AT902	Composite Urethane	6"	1029500	1169075
6PNEU	Composite Pneumatic	6 x 1-1/4"	1044040	1169075
AT903	Composite Urethane	8 x 1-1/4"	1166026	1169076
AT1292	Composite Pneumatic	8 x 1-1/4"	1166028	1169076
1492	Pneumatic w/ FF Insert	8 x 1-1/4"	1166027	1169076

Casters - Caster, Headtube and Fork w/ Mtg. Hdwr.

#2 Headtube Forks w/ Hardware

Caster Style	(FRKL1) Super Low Single Hole	(FRKS1) Normal Range Single Hole	(FRKS2) Normal Range Two Hole	(FRKS3) Normal Range Three Hole
5ALUM	1542007 See Note(s) A	1542007	1538091	1538090
AT901	1542007 See Note(s) A	1542007	1538091	1538090
AT5SOFT	1542007 See Note(s) A	1542007	1538092	1449113
AT902	1542007	1488399	1538092	1538090
6PNEU	1542007	1488399	1538092	1538090
AT903	1542007	N/A	N/A	1449113
AT1292	1542007	N/A	N/A	1449113
1492	1542007	N/A	N/A	1449113

NOTE: A - Also order Part Number 1449245 (2) when order replacement parts.