

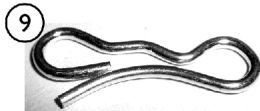
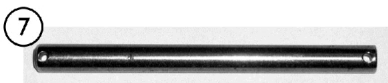
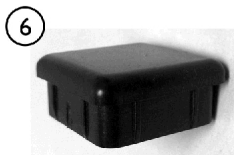
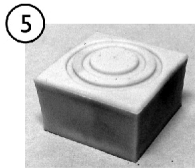
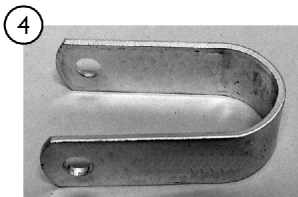
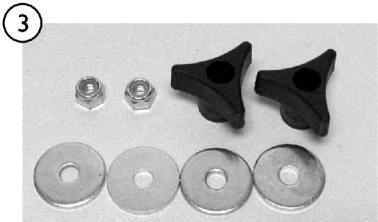
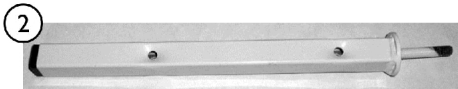
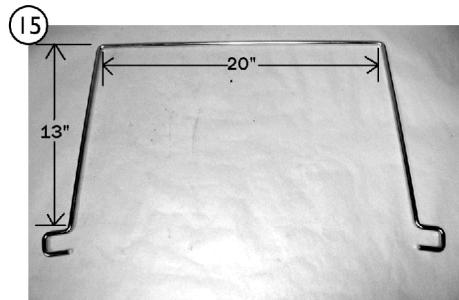
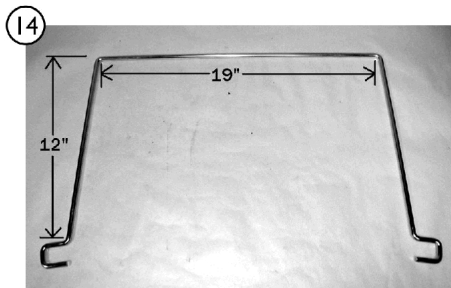


Steel and Frame Parts - BD0880/76 (continued)

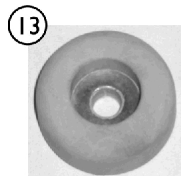

SICC_I21A

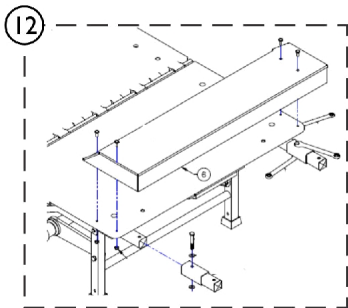

Steel and Frame Parts - BD0880/76 (continued)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1186525	Post, Head	1	1
2		1186524	Post, Foot	1	1
3		KIT78	Kit, Head & Foot Board Mounting Hardware	1	1
			Knob, Three Arm	1	2
4	A	MTRBRACKP1	Bracket, Motor Mount, Floating	1	1
5		GLDBEDFOOT	Foot, Bed - Rubber Santoprene	1	1
6		1186845	Plug, Bed Foot	1	1
7		PIN38412-NLA	NLA - Pin, 4-1/2" Straight	1	1
8		PIN380212	Clevis Pin (3/8 x 2-1/2" L)	1	1
9		PINBOW516-NLA	NLA - Pin, Cotter, Bowtie	1	1
10		MATTKPRKIT	Kit, Mattress Keeper, Left and Right Corners	1	1
			Mattress Keeper, Left Corner	1	2
			Mattress Keeper, Right Corner	1	2
11	B	IHECTA-NLA	NLA - Trapeze Adapter	1	1
12	C	KIT19EARRO-NLA	NLA - Extension Kit - 4"	1	1
12	C	DECK0005KIT-NLA	NLA - Kit, Old Style Foot Deck for DLX/ALLTO	1	1
13	D	GLDBUMPER	Rubber Bed Foot & Wall Bumper (Optional, 2 per Wireform)	1	1
14		WBB000576PL-NLA	NLA - Wireform, 76"	1	1
15		WBB000580PL-NLA	NLA - Wireform, 80"	1	1
NOTE: A - Order through SFI Medical Solutions B - Includes Frame, Mounting Hardware and 2 ea. Male Frencher Plates. C - Does not come with Bolster Pad, must use 84" Mattress. (Order from SFI Medical Solutions). D - Optional (2 per Wireform).					