

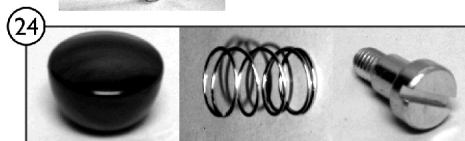
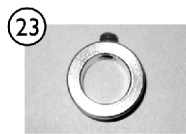
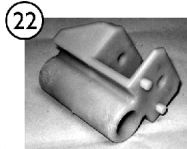
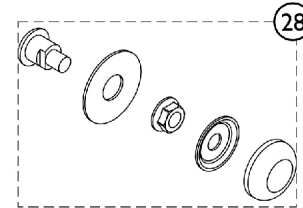
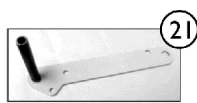
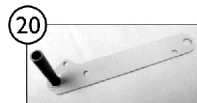
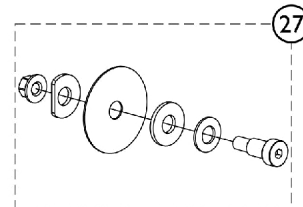
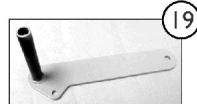
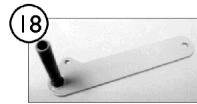
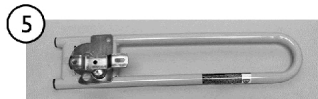
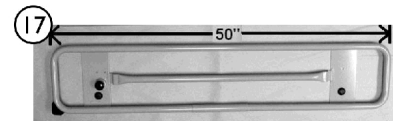
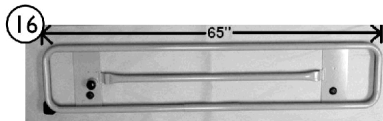
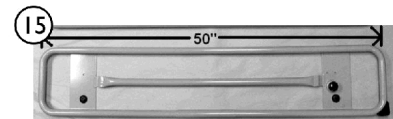
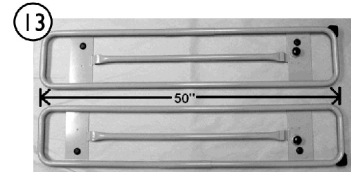
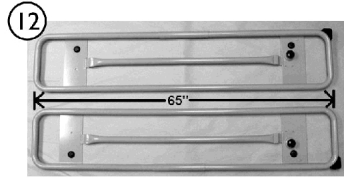
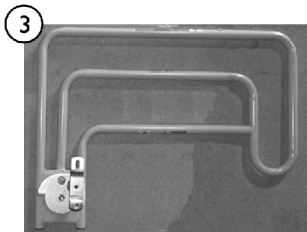
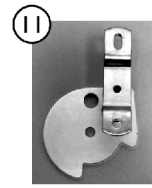
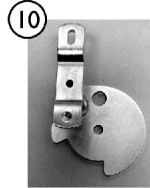
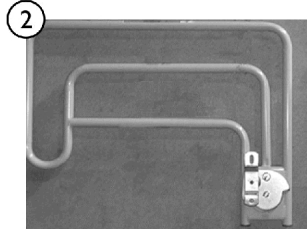
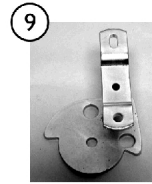


Bed Rails - BS1180/76

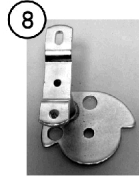

SICC_I25A

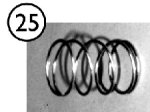

Bed Rails - BS1180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		IHECRLAS	Assist Rails (Pair)	2	1
2		RAILSAELT-NLA	NLA - Assist Rail Assembly - Left Side	1	1
3		RAILSAERT-NL A	NLA - Assist Rail Assembly - Right Side	1	1
4		IHECBAAS	Set of Assist Bars	2	1
5		RAILABLT-NLA	NLA - Assist Bar - Left Side	1	1
6		RAILABRT-NLA	NLA - Assist Bar - Right Side	1	1
7		KITASSISTR1	Kit, Assist Rail Mounting Hardware	1	1
8		RLBRACKAEP- NLA	NLA - Bracket, Mounting, Assist Rail - Left Side	1	1
9		RRBRACKAEP- NLA	NLA - Bracket, Mounting, Assist Rail - Right Side	1	1
10		RLBRACKASTR P-NLA	NLA - Punched Bracket, No Guard - Left Hand	1	1
11		RRBRACKASTR P-NLA	NLA - Punched Bracket, No Guard - Right Hand	1	1
12		IHARRLFUL-NL A	NLA - Full Rails (Set)	1	1
13		IHARRLPAR-NL A	NLA - Partial Rails (Set)	1	1
14		RAILS0080FLNT -NLA	NLA - Full Rail - Left Hand	1	1
15		RAILS0080PLNT -NLA	NLA - Partial Rail - Left Hand	1	1
16		RAILS0080FRN T-NLA	NLA - Full Rail - Right Hand	1	1
17		RAILS0080PRN T-NLA	NLA - Partial Rail - Right Hand	1	1
18		SRA001LA-NLA	NLA - Short Rail Arm - Left	1	1
19		SRA001RA-NLA	NLA - Short Rail Arm - Right	1	1
20		LRA001LA-NLA	NLA - Long Rail Arm - Left	1	1
21		LRA001RA-NLA	NLA - Long Rail Arm - Right	1	1
22		RAILBRACKNT- NLA	NLA - Rail Bracket	1	1
23		COL34R-NLA	NLA - Locking Set Collar	1	1
24		1189259	Kit, Knob - Latch	1	1
			Knob, LATCH-0002 (3/8-16 x 1-3/8")	1	1
			Spring, Compression (3/4 x 1 x 3/64")	1	1
			Screw, Shoulder Slotted Head (1/2 x 1/2", 3/8-16)	1	1
25		1186856	Spring, Compression (3/4 x 1 x 3/64")	1	2
26		1165068	Package, End Plug	10	1
27		1189260	Kit, Assist Rail Mounting	1	1
			Bolt, Shoulder (3/8-16 x 3/4")	1	1
			Washer, Belleville (33/64 x 1-1/64 x 3/32")	1	1
			Washer, Flat Edge (17-32 x 1-1/4 x 1/8")	1	1
			Washer, Nylon (33/64 x 2-1/2 x 3/64")	1	1
			Locknut (3/8-16)	1	1
			Washer, Plain (17/32 x 1-1/4 x 1/8")	1	1
28		1189261	Kit, Rail Mounting Hardware	1	1

Bed Rails - BS1180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
			Pin, Rail (5/16-18)	1	4
			Washer, Nylon (33/64 x 1-29/64 x 3/64")	1	4
			Washer, Retaining (11/32 x 1-3/32 x 3/64")	1	4
			NLA - Nut, Hex Head, Flange (5/16-18)	1	4
			Cap, Bearing	1	4