
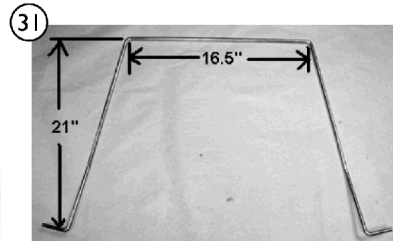
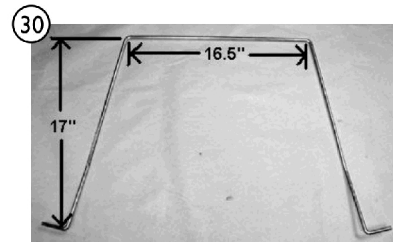
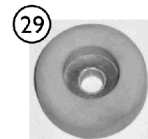
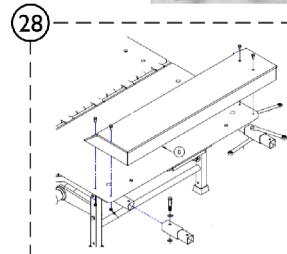
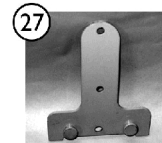
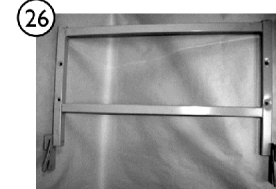
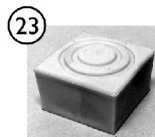
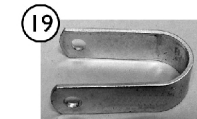
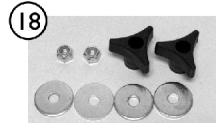
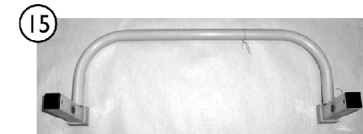
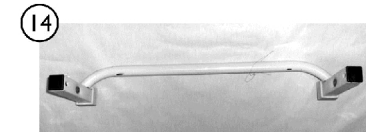
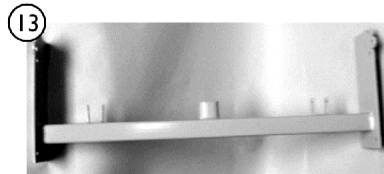
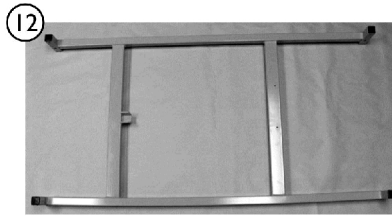
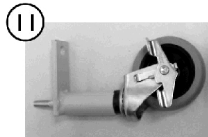
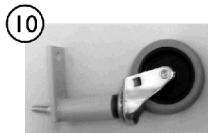
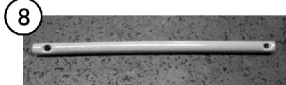
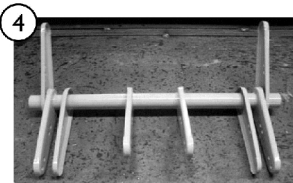
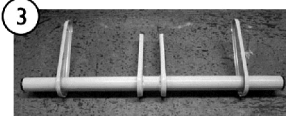
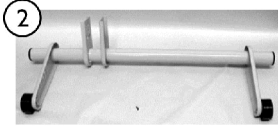


# Steel and Frame Parts - BS2180/76

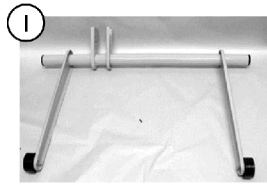

SICC\_I27A

## Steel and Frame Parts - BS2180/76

| Item Number | Note | Part Number           | Description                                                 | Quantity Per |          |
|-------------|------|-----------------------|-------------------------------------------------------------|--------------|----------|
|             |      |                       |                                                             | Package      | Assembly |
| 1           |      | 1185149-NLA           | NLA - Head Gatch Assembly<br>(Black Insert w/ Steel Pin)    | 1            | 1        |
| 2           |      | GATFOOTASSY<br>1-NLA  | NLA - Foot Gatch<br>(Black Insert with Steel Pin)           | 1            | 1        |
| 3           | A    | HILOUPPERAS<br>SY-NLA | NLA - Upper Gatch "4Arm"<br>(Black Insert w/ Steel Pin)     | 1            | 1        |
| 4           | A    | HILOLOWERAS<br>Y1-NLA | NLA - Gatch, Lower 8 Arm Head                               | 1            | 1        |
| 5           | A    | HILOLOWERAS<br>Y2-NLA | NLA - Gatch, Lower 6 Arm Foot                               | 1            | 1        |
| 6           |      | DRVTUBESOLO<br>P-NLA  | NLA - Tube, Hilo Drive                                      | 1            | 1        |
| 7           |      | LNK0017PL-NLA         | NLA - Linkage, Drive Tube                                   | 1            | 1        |
| 8           |      | TBR78161812P<br>N-NLA | NLA - Arm, Solo Control                                     | 1            | 1        |
| 9           | A    | LNK0013PNT-N<br>LA    | NLA - Linkage, Hilo Head/Foot                               | 1            | 1        |
| 10          | A    | WHLBRACKASS<br>Y3-NLA | NLA - Caster Assembly (Non-Lock)                            | 1            | 1        |
| 11          | A    | WHLBRACKASS<br>Y4-NLA | NLA - Caster Assembly, Lock                                 | 1            | 1        |
| 12          |      | LWRFRASSY1-<br>NLA    | NLA - Frame, Lower                                          | 1            | 1        |
| 13          | A    | MNT0018BPNT           | Plate, 80" Crank Mount                                      | 1            | 1        |
| 14          |      | LEG0004BASSY<br>-NLA  | NLA - Leg, Head End Hoop                                    | 1            | 1        |
| 15          |      | LEG0003CASSY<br>-NLA  | NLA - Leg, Foot End Hoop                                    | 1            | 1        |
| 16          |      | 1186525               | Post, Head                                                  | 1            | 1        |
| 17          |      | 1186524               | Post, Foot                                                  | 1            | 1        |
| 18          |      | KIT78                 | Kit, Head & Foot Board Mounting Hardware                    | 1            | 1        |
|             |      |                       | Knob, Three Arm                                             | 1            | 2        |
| 19          | A,B  | MTRBRACKP1            | Bracket, Motor Mount, Floating                              | 1            | 1        |
| 20          |      | PIN38412-NLA          | NLA - Pin, 4-1/2" Straight                                  | 1            | 1        |
| 21          |      | PIN380212             | Clevis Pin (3/8 x 2-1/2" L)                                 | 1            | 1        |
| 22          |      | PINBOW516-NL<br>A     | NLA - Pin, Cotter, Bowtie                                   | 1            | 1        |
| 23          |      | GLDBEDFOOT            | Foot, Bed - Rubber Santoprene                               | 1            | 1        |
| 24          |      | 1186862               | Plug, Under Bed Foot                                        | 1            | 1        |
| 25          |      | MATTKPRKIT            | Kit, Mattress Keeper, Left and Right Corners                | 1            | 1        |
|             |      |                       | Mattress Keeper, Left Corner                                | 1            | 2        |
|             |      |                       | Mattress Keeper, Right Corner                               | 1            | 2        |
| 26          | C    | IHECTA-NLA            | NLA - Trapeze Adapter                                       | 1            | 1        |
| 27          |      | FRENCHERMAL<br>E-NLA  | NLA - Plate, Frencher - Male                                | 1            | 1        |
| 28          | A,D  | KIT19EARRO-N<br>LA    | NLA - Extension Kit - 4"                                    | 1            | 1        |
| 29          | E    | GLDBUMPER             | Rubber Bed Foot & Wall Bumper<br>(Optional, 2 per Wireform) | 1            | 1        |
| 30          |      | WBBSOLO76P-<br>NLA    | NLA - Wireform, 76"                                         | 1            | 1        |

## Steel and Frame Parts - BS2180/76

| Item Number                                                                                                                                                                                                                                                                                     | Note | Part Number    | Description         | Quantity Per |          |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|----------------|---------------------|--------------|----------|
|                                                                                                                                                                                                                                                                                                 |      |                |                     | Package      | Assembly |
| 31                                                                                                                                                                                                                                                                                              |      | WBBSOLO80P-NLA | NLA - Wireform, 80" | 1            | 1        |
| NOTE: A - Order through SFI Medical Solutions<br>B - KIT85 Foot Motor Mount Bracket is No Longer Available.<br>C - Trapeze Adapter comes with Frame, Mounting Hardware and 2 Male Frencher Plates.<br>D - Does not include Bolster Pad, must use 84" Mattress.<br>E - Optional (2 per Wireform) |      |                |                     |              |          |