

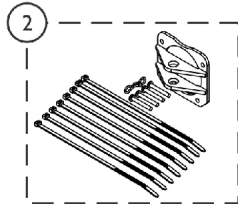
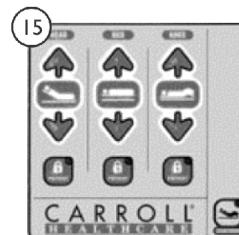
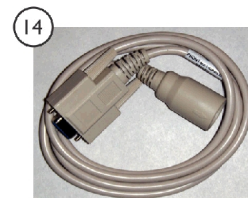
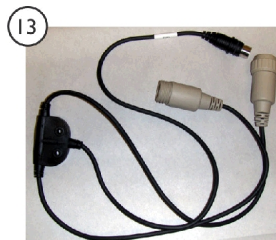
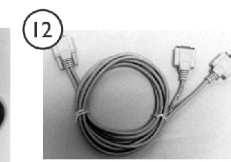
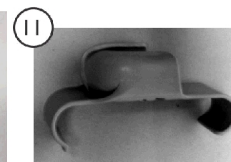
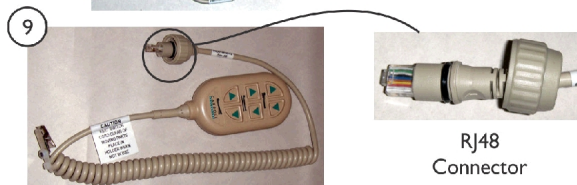


Electronics & Pendants - BS3180/76

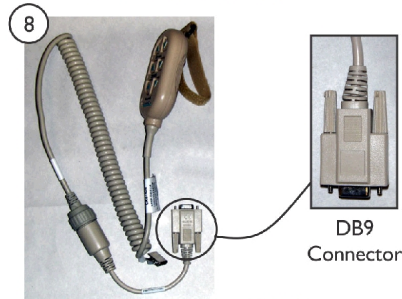

SICC_123A

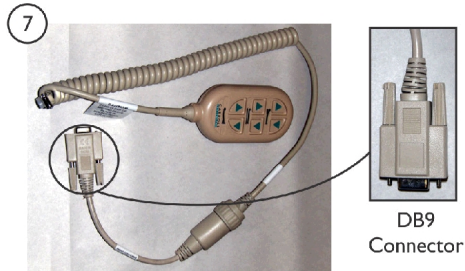

Electronics & Pendants - BS3180/76

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		MTRM2133-NLA	NLA - Actuator, Head (Red Plug) rEF. #61832-1005270-01	1	1
2		KIT80-NLA	NLA - Kit, Bracket Trunion for Head Motor	1	1
			Pin, Cotter Bow-Tie (1/4 x 3/32 x 5/16")	1	1
			Screw, Torx HD Taptite LG (8-32 x 1-1/4")	1	4
			Bracket, Trunion Plated	1	1
			Clevis Pin (3/8 x 2-1/2" L)	1	2
			NLA - Wrap, Tie (8")	1	8
			Sheet, Instruction, Bracket Solo Head/HiLo Motors	1	1
3	A	MTRM2134-NLA	NLA - Foot Actuator (Blue Plug) Ref. #61831-0625270-01	1	1
4		MTRM2478-NLA	NLA - Actuator, HiLo, Solo (White Plug)	1	1
5		JNCTNBOXKIT2-NLA	NLA - Junction Box Ref. #NAX5105-3A29MPZ	1	1
6		JNCTNBOXMEM-NLA	NLA - Junction Box Membrane	1	1
7	B	1190227	Pendant, Six Function (w/ Loop Adhesive Back Strip and DB9 Connector) Linak # PNDNT-6FNC-1-2	1	1
8		1190232	Pendant, Six Function (w/ Strap and DB9 Connector) Linak # PNDNT-6FNC-1-2	1	1
9	C	PNDNT6FNCIV3	Pendant, Six Function w/ RJ48 Connector) Linak # PNDNT-6FNC-1-2	1	1
10		1190225	Cable, Conversion	1	1
11		1186847	Holster, Pendant	1	1
12	B	PNDNTYCABLE	Cable, Y, Pendant	1	1
13	C	JNCTNDC40	Cable, Y, Pendant	1	1
14	C	PNDNTEXTCABLE	Cable, Pendant	1	1
15		JNCTNDC20C	Membrane Pad for 151 Only	1	1
NOTE: A - Foot Actuator Kit (KIT84) is No Longer Available for Beds built prior to 12/15/08. Part #MTRM2134 used for Brds built as of 12/08. B - Beds built prior to 11/22/07. C - Beds built after 11/22/07.					