


IH3602GXL, IH3752G, IH3802G - Door Hinge Assembly

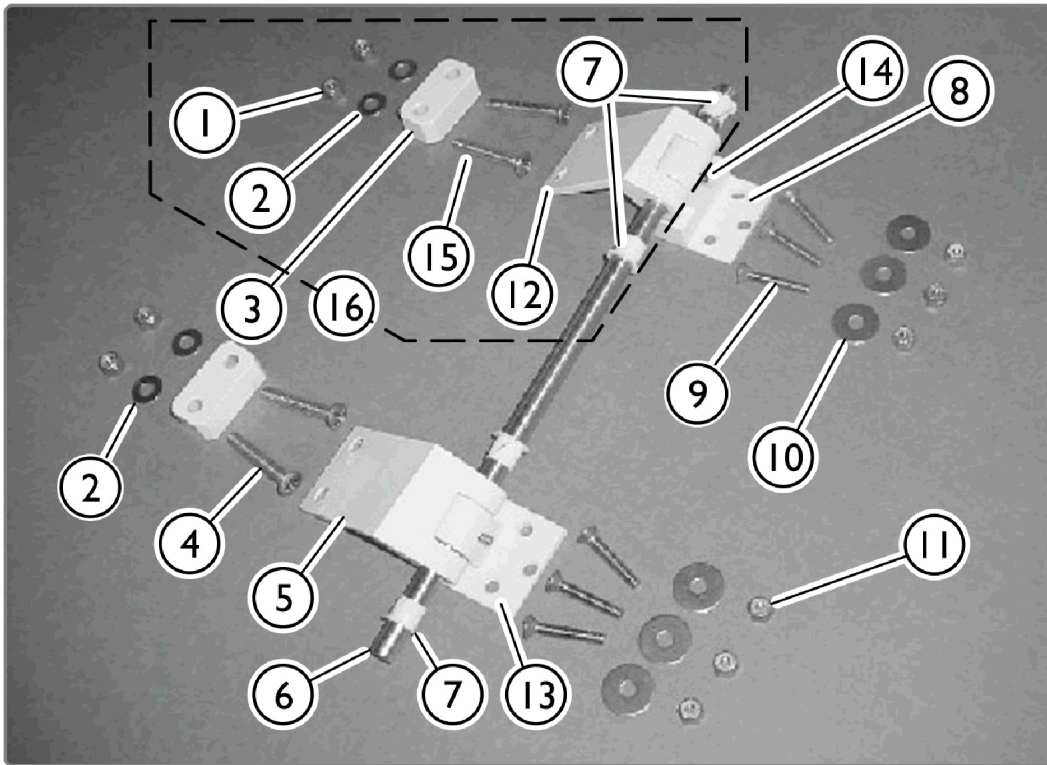

SPC_045A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		PAR9921	Loc-Nut	1	4
2		PAR9919	Washer	1	4
3		IHPAR3610W	Spacer, Block, White (3/4")	1	2
3		IHPAR3611W	Spacer, Block, White (5/8")	1	2
3		IHPAR3612W-NLA	NLA - Spacer, Block, White (11/16")	1	2
4		PAR9920	Bolt, Door Hinge, Lower	1	2
5		IHPAR3622LB	Hinge, Lower Door, Bisque	1	1
6	A	IHPAR3615A	Pin, Door Hinge	1	1
7		IHPAR3609	Bushing, Hinge	1	4
8		IHPAR3623B	Bracket, Upper Door Hinge, Off White	1	1
9		1127641	Screw, Phillips, Flat Head (1/4-20 x 2")	1	6
10		PAR9918	Washer	1	6
11		PAR9916	Nut (1/4-20)	1	6
13		IHPAR3624B	Bracket, Lower Door Hinge, Off White	1	1
14		IHPAR3625	Screw, Set, Hinge Bracket	1	1
16	B	60001324	Kit, Upper Door Hinge, Off White	1	1

NOTE: A - IHPAR3615A (one piece hinge pin) replaces 2 of IHPAR3615 (single hinge pin).
 B - If Tub was manufactured before 09/22/17, entire Kit item #16 part number 60001324 must be ordered. Includes items 1, 2, 7, 12 and 15. Item 12 and 15 are not sold separately. Item 15 Screws should be torqued to 45-60 in-lbs.

For item #12 and 15 order item #16 part number 60001324 as a replacement.