
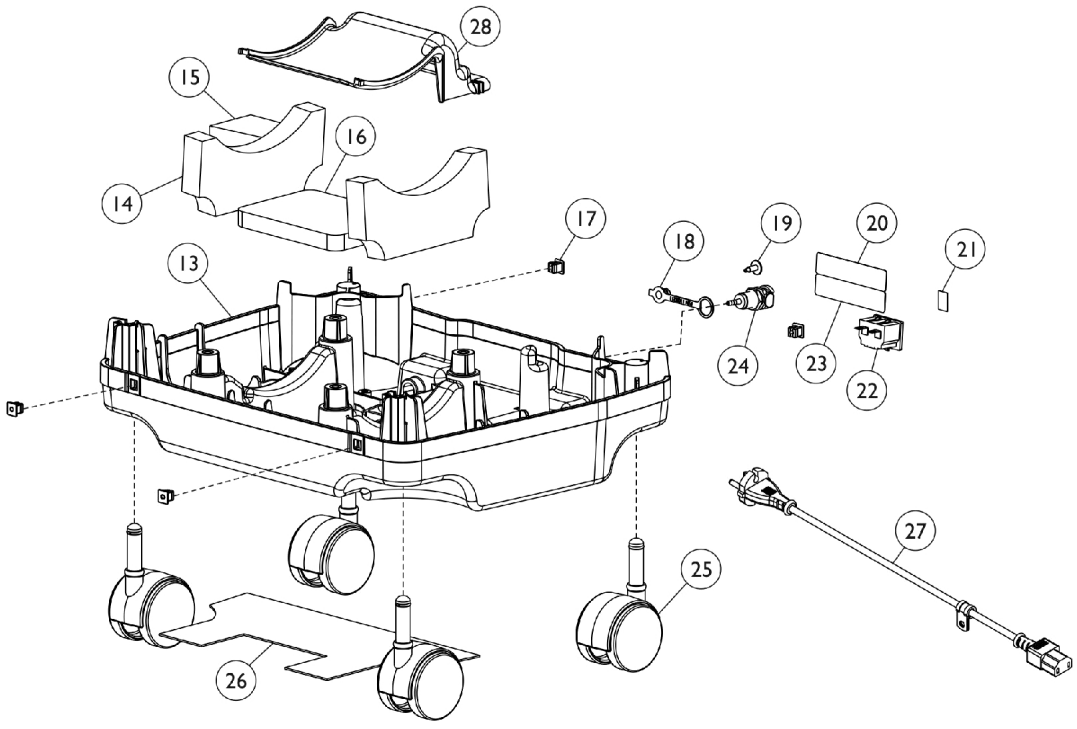


Cabinet and Base

Perfecto2 AW

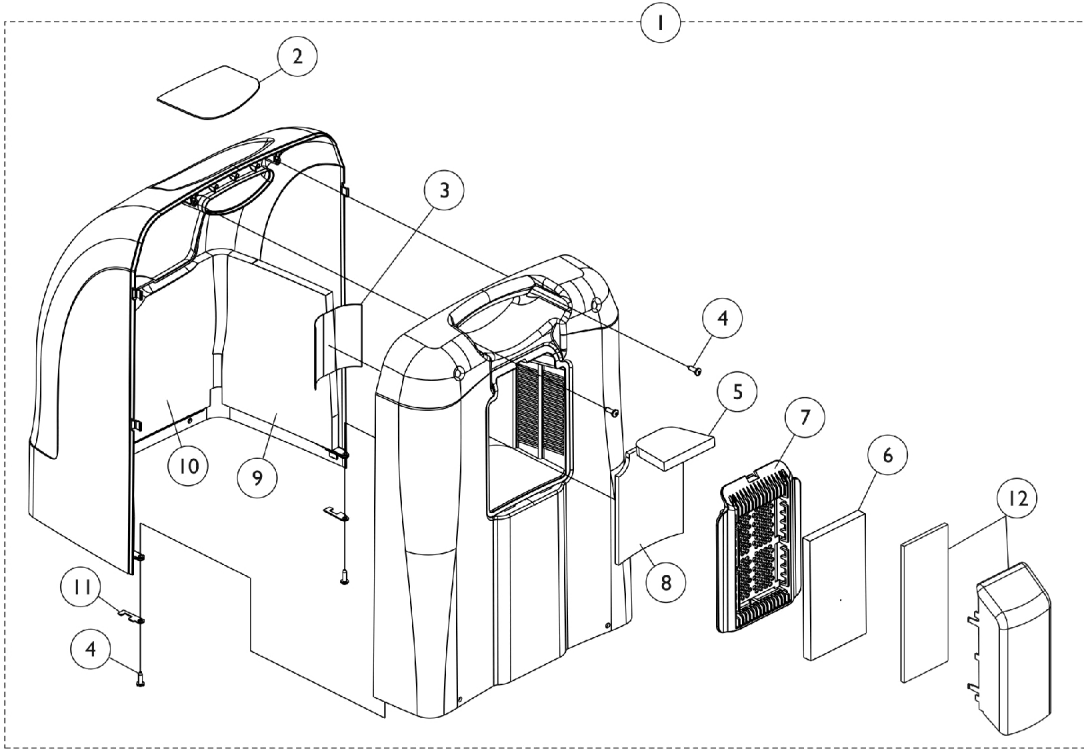

Cabinet and Base Perfecto2 AW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E,A	1154095-NLA	NLA - Tech Only - Top Shroud Assembly, Beige	1	1
2		1150725-NLA	NLA - Label, Text, English	1	1
3		1137267	Label, Diagnostic	1	1
4		1108810	Screw, Hi-Lo Six Lobe Drive (#10-16 x 5/8")	1	4
5		1153695	Foam, Inlet - Ceiling	1	1
6		1143492	Filter, Cabinet	1	1
7	E,F	1161831	Door, Cabinet Access, Beige	1	1
8		1153700	Foam, Inlet - Wall	1	1
9		1143465	Foam, Shroud - Side	1	2
10		1143464	Foam, Shroud - Front	1	1
11		1143550	Clip, Retaining	1	2
12	B	1156532	Inlet Air Scoop Assembly	1	1
13	C	1143451-NLA	NLA - Tech Only - Base, Concentrator	1	1
14		1153706	Foam, Damping - Base	1	2
15		1154501	Foam, Resonator	1	1
16		1153705	Foam, Damping Outlet Base	1	1
17		2000360	Grommet, Plastic	1	4
18		1108211	Cap, End, Dust, Green	1	1
19		1156530	Screw, Phillips Round Washer Head Tap (8-18 x 1/2")	1	1
20		1153702-NLA	NLA - Label, Spec (230V)	1	1
21		1136841-NLA	NLA - Label, Recycling	1	1
22		1105515-NLA	NLA - Tech Only - Module, Inlet (15A/ 250 VAC)	1	1
23		1143250	Label, European Rep Address	1	1
24		1104151	Fitting, Hose Barb with Nut (1/8")	1	1
25		1110426	Caster Assembly (2")	1	4
26		1153704	Damper, Sound Base	1	1
27		1156533-NLA	NLA - Tech Only - Power Cord Assembly w/ Clip	1	1
28		1143444-NLA	NLA - Tech Only - Baffle, Exhaust	1	1

NOTE: A - Includes Cabinet and Items 2-11. This item is No Longer Available in the US. This item comes from China only.
 B - Includes Inlet Foam and Scoop.
 C - Includes Base Only.
 E - Color was "Graphite" before 1/16/2009.
 F - Cabinet Access Door was revised 10/16/09 to allow the Cabinet Filter to be assembled to the outside of the Cabinet Access Door. The Cabinet Access Door is backward compatible in the field for models manufactured before 10/16/09.