


# Utility Tray

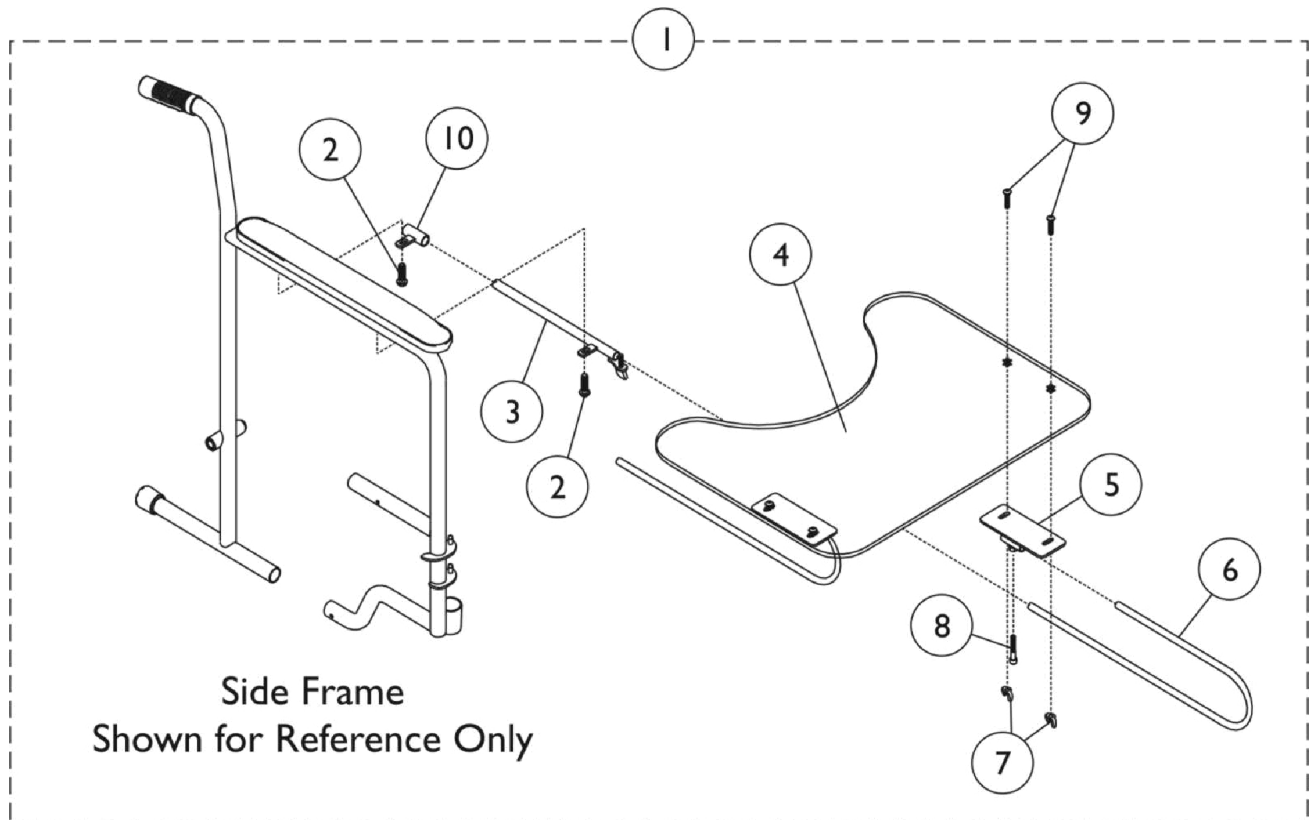

## Utility Tray

| Item Number                                                                                                                                                                                                                                                                                                  | Note | Part Number   | Description                                     | Quantity Per |          |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|---------------|-------------------------------------------------|--------------|----------|
|                                                                                                                                                                                                                                                                                                              |      |               |                                                 | Package      | Assembly |
| 1                                                                                                                                                                                                                                                                                                            | A    | 1035965-NLA   | NLA - Tray, Utility, Full Length Arm            | 1            | 1        |
| 1                                                                                                                                                                                                                                                                                                            | A    | 1035966-NLA   | NLA - Tray, Utility, Desk Length Arm            | 1            | 1        |
| 2                                                                                                                                                                                                                                                                                                            |      | 11008X011     | Screw, Phillips Round Head (#10-32 x 1-3/8")    | 1            | 2        |
| 3                                                                                                                                                                                                                                                                                                            |      | 04570X009-NLA | NLA - Tube, Slide Lock, Full Length Arm - Right | 1            | 1        |
| 3                                                                                                                                                                                                                                                                                                            |      | 04570X010-NLA | NLA - Tube, Slide Lock, Full Length Arm - Left  | 1            | 1        |
| 3                                                                                                                                                                                                                                                                                                            |      | 04570X011-NLA | NLA - Tube, Slide Lock, Desk Length Arm - Right | 1            | 1        |
| 3                                                                                                                                                                                                                                                                                                            |      | 04570X012-NLA | NLA - Tube, Slide Lock, Desk Length Arm - Left  | 1            | 1        |
| 4                                                                                                                                                                                                                                                                                                            |      | 47469X006-NLA | NLA - Tray                                      | 1            | 1        |
| 5                                                                                                                                                                                                                                                                                                            |      | 1009341-NLA   | NLA - Bracket, Tray                             | 1            | 2        |
| 6                                                                                                                                                                                                                                                                                                            |      | 1010922-NLA   | NLA - Tube, Support                             | 1            | 2        |
| 7                                                                                                                                                                                                                                                                                                            |      | 20148X004-NLA | NLA - Nut, Wing (1/4-20)                        | 1            | 4        |
| 8                                                                                                                                                                                                                                                                                                            |      | 60284X001     | Screw, Socket Head (1/4-20 x 1/2")              | 1            | 2        |
| 9                                                                                                                                                                                                                                                                                                            |      | 10123X006     | Bolt, Carriage (1/4-20 x 3/4")                  | 1            | 4        |
| 10                                                                                                                                                                                                                                                                                                           |      | 04570X002-NLA | NLA - Tube, Support, Slide, Chrome              | 1            | 2        |
|                                                                                                                                                                                                                                                                                                              | B,C  | 1023704-NLA   | NLA - Instruction Sheet                         | 1            | 1        |
| NOTE: A - Includes items 2-9 and an instruction sheet.<br>B - Not Shown.<br>C - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy. |      |               |                                                 |              |          |