
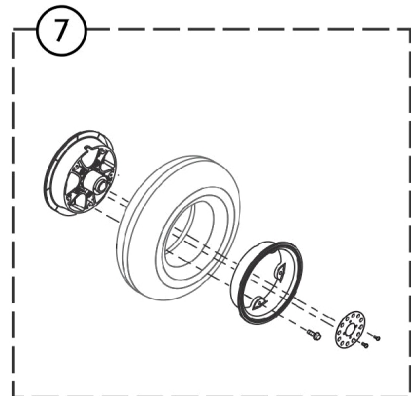
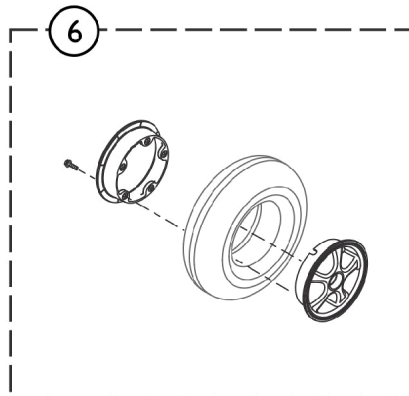
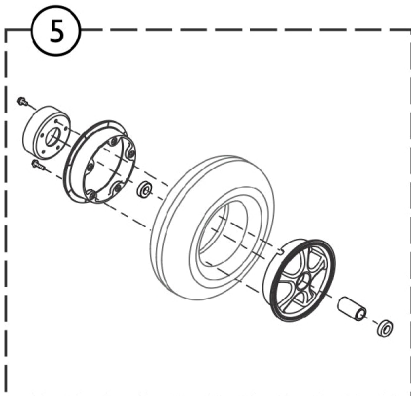
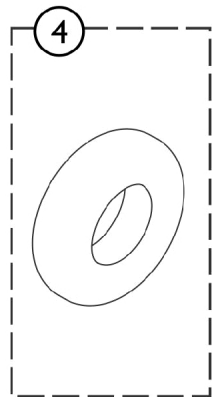
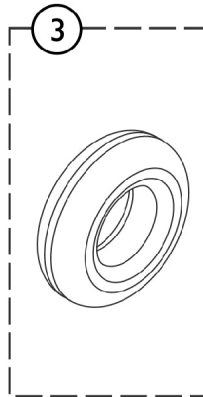
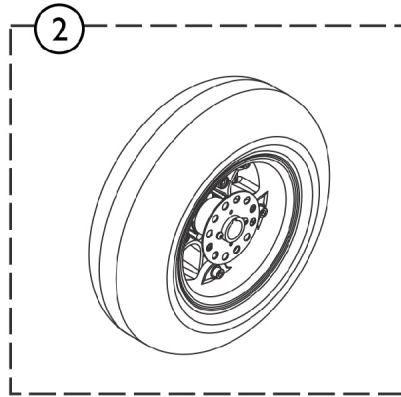


Wheels - 6" Rim

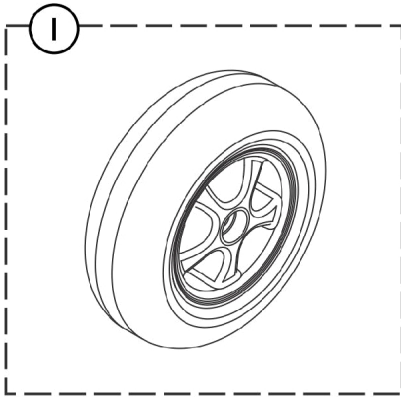

SCP_112A

Wheels - 6" Rim

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1608776	Front Wheel 11", pneumatic, Low Profile, 6" rim, Orion Metro 3-Wheel	--	1
1		1608794	Front Wheel 11", pneumatic, Low Profile, 6" rim, Orion Metro 4-Wheel	--	1
1		1608799	Rear Wheel 11", pneumatic, Low Profile, 6" rim - Orion Metro	--	1
1		1608796	Front Wheel 12", pneumatic, Low Profile, 6" rim - Orion Pro 3 & 4-Wheel	--	1
1		1637432	Rear Wheel 12", pneumatic, Low Profile, 6" rim, RH - Orion Pro	--	1
2		1608812	Rear Wheel 12", pneumatic, Low Profile, 6" rim, LH - Orion Pro	--	1
3		1638977	Tire 11x3.50-6	--	1
3		1638974	Tire 12x4.00-6	--	1
4		1638978	Tube 11x3.50-6	--	1
4		1638975	Tube 12x4.00-6	--	1
5		1638835	6" Rim Kit, Front - Metro 3-Wheel	--	1
5		1638840	6" Rim Kit, Front - Metro 4-Wheel	--	1
6		1638836	6" Rim Kit, Rear - Metro LH/RH & Pro RH	--	1
7		1638861	6" Rim Kit, Rear - Pro LH	--	1