


Steering Complete

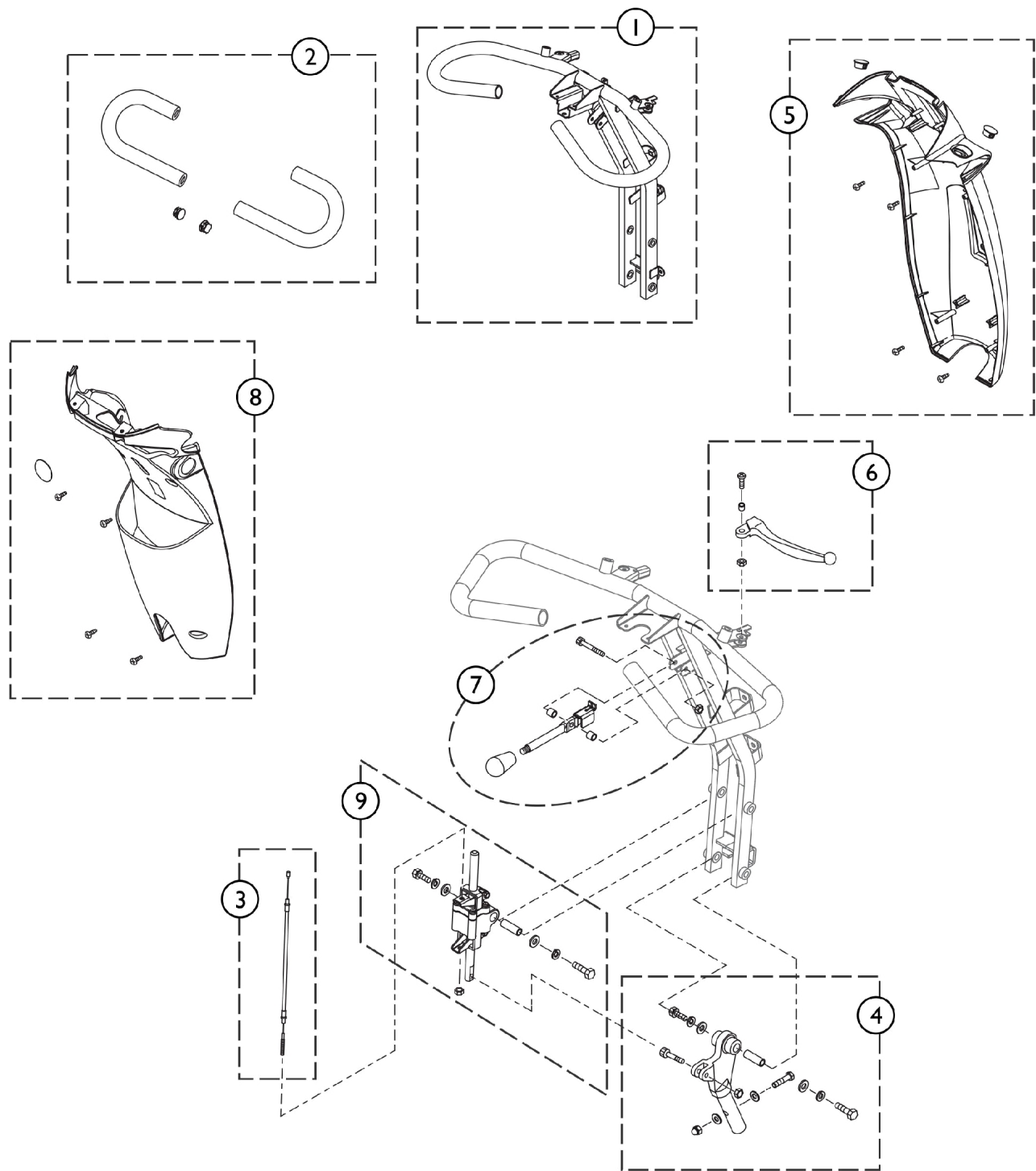

SCP_120A

Steering Complete

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1608184	Steering	--	1
2		1636767	Grip, Handle Bar, cpl.	--	1
3		1608201	Cable, steering adj.	--	1
4		1636812	Joint, steering adjustment, cpl.	--	1
5		1608205	Front steering cover	--	1
6		1636804	Lever, Brake - RH	--	1
7		1636813	Lever, steering adjustment	--	1
8		1636814	Rear steering cover	--	1
9		1608202	Steering adjustment Assembly, cpl.	--	1