


Backrest Mounting Hardware - No Arms or One Arm

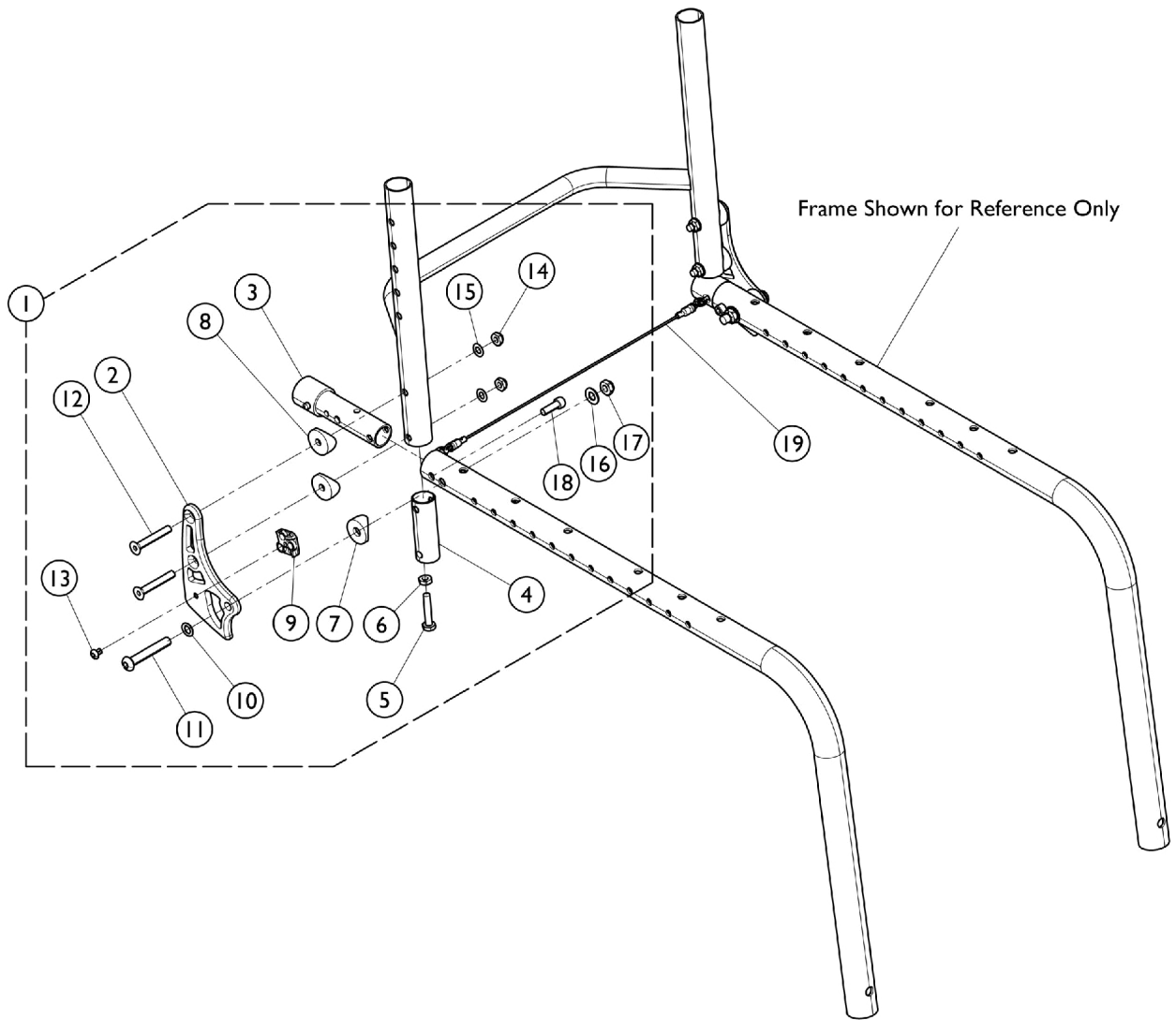

Backrest Mounting Hardware - No Arms or One Arm

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1172685-NLA	NLA - Kit, Hardware, Backrest Mounting w/o Arms	1	1
2			NLA - Bracket, Back, Black - Right	1	1
2			Bracket, Back, Black - Left	1	1
3			Insert, Backstop	1	2
4			Insert, Cane, Rear	1	2
5			Screw, Hex (1/4-28 x 1-1/4")	1	2
6			Nut, Hex, Jam (1/4-28)	1	2
7			Spacer, Coved (3/4 x 1 x 1/2")	1	2
8			Spacer, Coved (1/4 x 1 x 9/16")	1	4
9			Square, Adjustment	1	2
10			Washer, Nylon (5/16 x 17/32 x 1/32")	1	2
11			Screw, Socket Button Head (5/16-18 x 2")	1	2
12			Screw, Socket, Flat Head, (1/4-28 x 1-3/4")	1	4
13			Screw, Socket, Button Head w/ Patch (10-32 x 1/4")	1	2
14			Locknut (1/4-28)	1	4
15			NLA - Washer, Steel (1/4 x 1/2 x 3/64")	1	4
16			Washer, Steel (21/64 x 5/8 x 3/64")	1	2
17			Locknut (5/16-18)	1	2
18			NLA - Screw, Socket Head, Cap (1/4-28 x 3/4")	1	2
19	B	1159762-NLA	NLA - Cord, Back Release (12" W)	1	1
19	B	1159763-NLA	NLA - Cord, Back Release (13" W)	1	1
19	B	1159764-NLA	NLA - Cord, Back Release (14" W)	1	1
19	B	1159765-NLA	NLA - Cord, Back Release (15" W)	1	1
19	B	1159766-NLA	NLA - Cord, Back Release (16" W)	1	1
19	B	1159767-NLA	NLA - Cord, Back Release (17" W)	1	1
19	B	1159768-NLA	NLA - Cord, Back Release (18" W)	1	1
19	B	1159769-NLA	NLA - Cord, Back Release (19" W)	1	1
19	B	1159789-NLA	NLA - Cord, Back Release (20" W)	1	1
		1165867	Kit, Plunger Pin w/ Spring	1	1
			Pin, Plunger	1	1
			Spring, Pin, Plunger (1/4 x 13/16 x .020)	1	1
		See Chart	Item 2. Back Brackets - Right/Left	1	1

NOTE: A - Includes items 2 - 18.
 B - The Back Release Cord includes a Split Ring on each end but DOES NOT INCLUDE the Plunger Pins or Springs. It is recommended when ordering a new Back Release Cord, 1165867 (Kit, Plunger Pin w/ Spring) be ordered also.

Item 2. Back Brackets - Right/Left

Color	Black	Silver	Blue	Red
Right	1167083-NLA	1167081	1167084-NLA	1167082-NLA
Left	1167087	1167086-NLA	1167088	1167085