
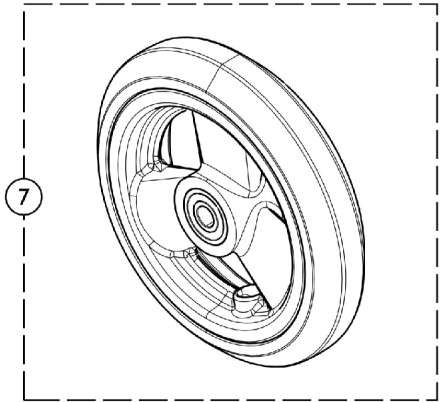
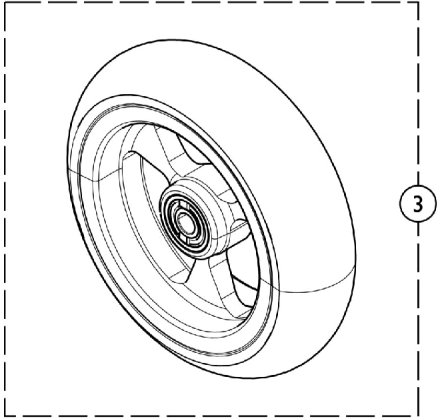
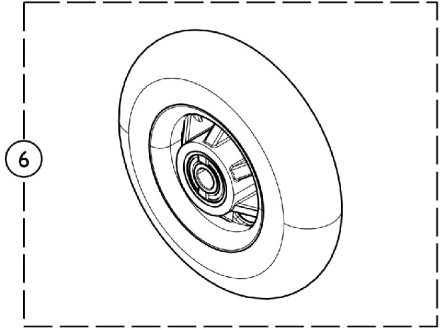
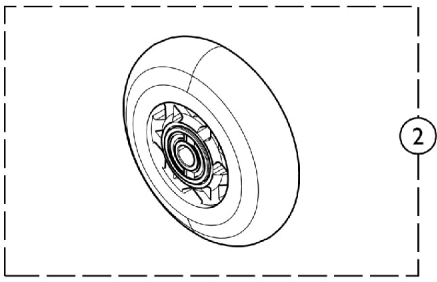
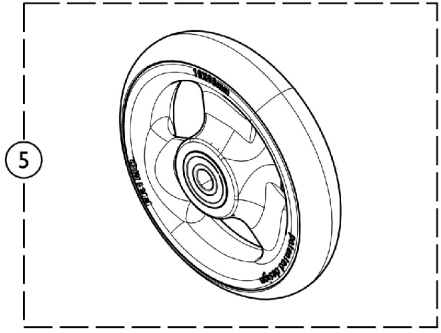
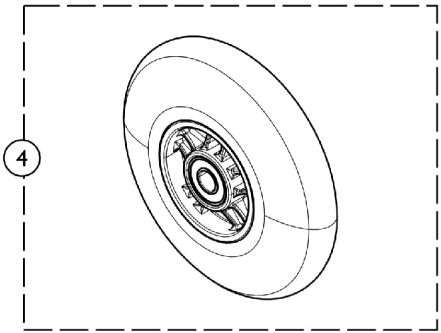


Casters - Front and Mounting Hardware

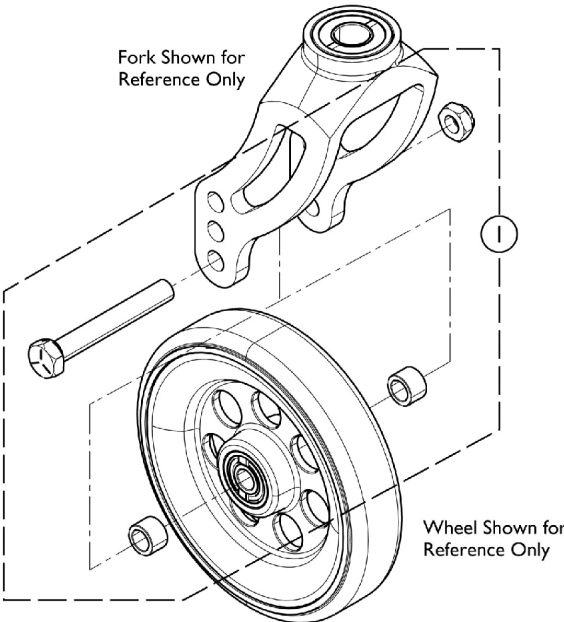

Casters - Front and Mounting Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1159772	Kit, Front Wheel Hardware	1	1
			Spacer (5/16 x 7/16)	1	2
			Locknut (5/16-24)	1	1
			Screw, Hex Head (5/16-24 x 2-1/4")	1	1
		See Chart	Caster Styles	1	1
2		1157282-NLA	NLA - Wheel Assembly, Soft Roll (3" x 1-1/2")	1	1
2		1128070	Wheel Assembly, Soft Roll (4" x 1")	1	1
2		1128071	Wheel Assembly, Soft Roll (5" x 1-1/2")	1	1
2		1169260	Wheel Assembly, Composite, Urethane (5" x 1")	1	1
3		1032190-NLA	NLA - Wheel Assembly (3" x 1")	1	1
4		1060910	Wheel Assembly, Composite Urethane, Light Up (4" x 1")	1	1
5		1164317-NLA	NLA - Wheel Assembly, Composite, Urethane (4 x 1")	1	1
5		1164318-NLA	NLA - Wheel Assembly, Composite, Urethane (4 x 1-1/2")	1	1
6		1114142	Wheel Assembly, Semi-Pneumatic, Black (4" x 1")	1	1
7		1164311	Wheel Assembly, Composite, Urethane (5" x 1")	1	1

NOTE: A - Includes Screw, Locknut, and (2) Spacers.

Caster Styles

Caster Style	Size	Tire Type	Part Number
41DC	4 x 1"	Urethane	1164317-NLA
415DC	4 x 1-1/2"	Urethane	1164318-NLA
51DC	5 x 1"	Urethane	1169260
515DC	5 x 1-1/2"	Urethane	1164312
61DC	6 x 1"	Urethane	1164314
615DC	6 x 1-1/2"	Urethane	1164313
CRF654	4"	Semi-Pneumatic	1114142
CRF655	5"	Semi-Pneumatic	1164311
CRFA654	4"	Aluminum Soft Roll	1128070
CRFA655	5"	Aluminum Soft Roll	1128071
CRF844	4"	Lighted	1060910