
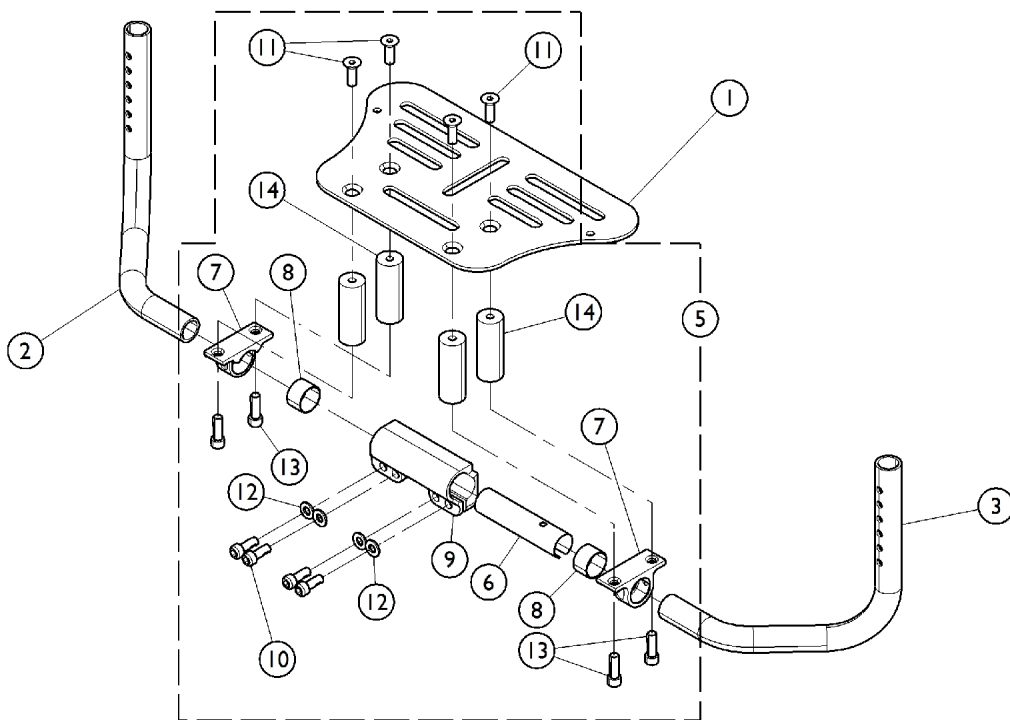


Front Riggings - Footrest and Hardware (FLPFP and FLPFP2)

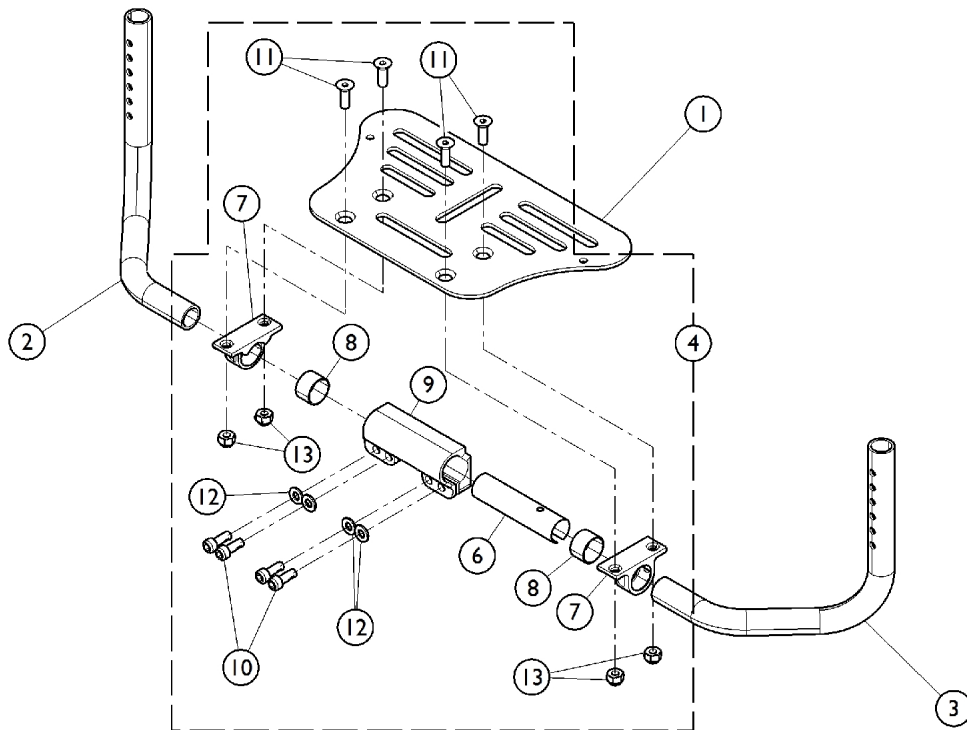

Front Riggings - Footrest and Hardware (FLPFP and FLPFP2)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1184073	Footplate, Small, Black (10")	1	1
1		1184074	Footplate, Large, Black (12")	1	1
2		See Chart	#2. Front Rigging Tubes - Right	1	1
3		See Chart	#3. Front Rigging Tubes - Left	1	1
4	A	1184114	Kit, Hardware, Flip Up Footplate	1	1
6			Sleeve, Spacer, Footrest	1	1
7			Bracket, Flip Up, Footrest	1	2
8			Sleeve, Footrest Bracket	1	2
9			Clamp, Footrest, Flip Up	1	1
10			NLA - Screw, Socket Head (M6 x 1 x 16mm)	1	4
11			Screw, Socket Flat Head (1/4-28 x 3/4")	1	4
12			NLA - Washer, Steel (1/4 x 1/2 x 3/64")	1	4
13			Locknut (1/4-28)	1	4
5	B	1184115	Kit, Hardware, Flip Up Footplate w/ Spacers	1	1
6			Sleeve, Spacer, Footrest	1	1
7			Bracket, Flip Up, Footrest	1	2
8			Sleeve, Footrest Bracket	1	2
9			Clamp, Footrest, Flip Up	1	1
10			NLA - Screw, Socket Head (M6 x 1 x 16mm)	1	4
11			Screw, Socket Flat Head (1/4-28 x 3/4")	1	4
12			NLA - Washer, Steel (1/4 x 1/2 x 3/64")	1	4
13			NLA - Screw, Socket Head, Cap (1/4-28 x 3/4")	1	4
14			Spacer (16/64 x 3/4 x 2")	1	4

NOTE: A - Kit includes items 6 - 13.
B - Kit includes items 6 - 14.

#2. Front Rigging Tubes - Right

Chair Width	Part Number
14" Wide	1184097-NLA
15" Wide	1184099-NLA
16" Wide	1184101-NLA
17" Wide	1184103-NLA
18" Wide	1184105-NLA

#3. Front Rigging Tubes - Left

Chair Width	Part Number
14" Wide	1184098-NLA
15" Wide	1184100-NLA
16" Wide	1184102-NLA

Front Riggings - Footrest and Hardware (FLPFP and FLPFP2)

#3. Front Rigging Tubes - Left

Chair Width	Part Number
17" Wide	1184104-NLA
18" Wide	1184106-NLA