


Drop Hook Solid Folding Seats

NOTE: Solid Folding Seats are 2" narrower than actual chair width and are not available on chairs over 20" wide.

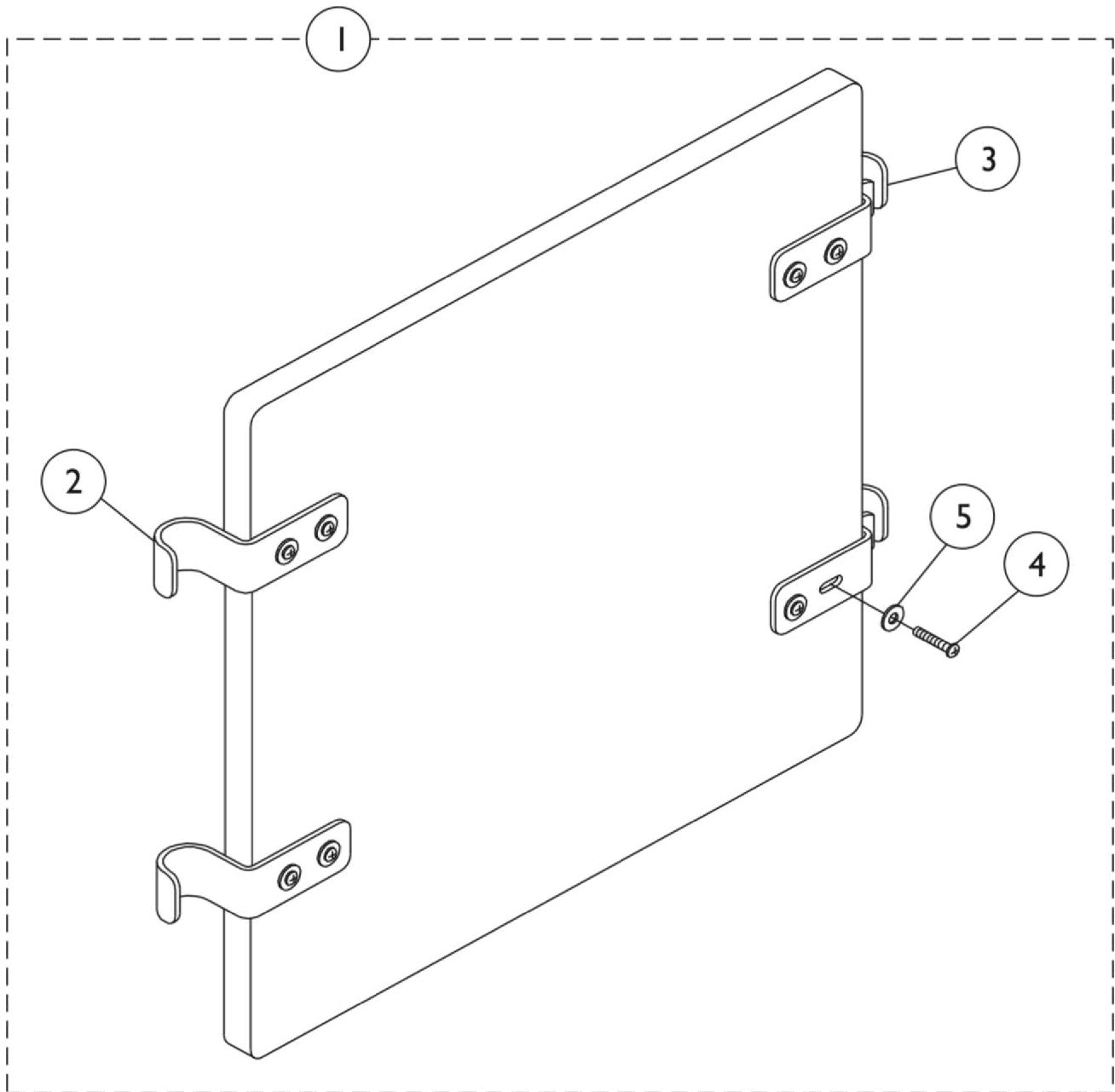

Drop Hook Solid Folding Seats

NOTE: Solid Folding Seats are 2" narrower than actual chair width and are not available on chairs over 20" wide.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	See Chart	Solid Folding Seats - 9000XDT	1	1
1	A,B	See Chart	Solid Folding Seats - 9000XT / Recliner	1	1
2		1023986	Hook, Seat	1	2
3		1023985	Bracket, Hinge, Seat	1	2
4		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	8
5		1013218	Washer (#10 Flat)	1	8
	C,D	1035940-NLA	NLA - Solid Folding Seat with Hardware (14" W x 16" D)	1	1
	C,D	1035941-NLA	NLA - Solid Folding Seat with Hardware (14" W x 17" D)	1	1
	C,D	1035942-NLA	NLA - Solid Folding Seat with Hardware (14" W x 18" D)	1	1
	C,D	1035943-NLA	NLA - Solid Folding Seat with Hardware (16" W x 16" D)	1	1
	C,D	8881035944-NL A	NLA - Solid Folding Seat w/ Hardware (16" W x 17" D)	1	1
	C,D	8881035945-NL A	NLA - Solid Folding Seat w/ Hardware (16" W x 18" D)	1	1
	C,D	8881035946-NL A	NLA - Solid Folding Seat w/ Hardware (18" W x 16" D)	1	1
	C,D	8881035947-NL A	NLA - Solid Folding Seat w/ Hardware (18" W x 17" D)	1	1
	C,D	8881035948-NL A	NLA - Solid Folding Seat w/ Hardware (18" W x 18" D)	1	1
NOTE: A - Includes items 2-5. B - For use on chairs manufactured after 5/25/03. C - Specify upholstery color when ordering, see category titled, "Frame Finishes, Upholstery, and Packaging." D - For use on Adult, Hemi and Recliner chairs manufactured before 5/26/03.					

Solid Folding Seats

9000XDT

Width X Depth	Nylon Upholstery	Vinyl Upholstery
16" x 16"	8881110370-NLA	8881110378-NLA
	See Note(s) A	See Note(s) A
16" x 18"	8881110374-NLA	8881110382-NLA
	See Note(s) A	See Note(s) A
18" x 16"	8881110371-NLA	8881110379
	See Note(s) A	See Note(s) A
18" x 18"	8881110375-NLA	8881110383-NLA
	See Note(s) A	See Note(s) A
20" x 16"	8881110372	8881110380-NLA
	See Note(s) A	See Note(s) A

Drop Hook Solid Folding Seats

NOTE: Solid Folding Seats are 2" narrower than actual chair width and are not available on chairs over 20" wide.

Solid Folding Seats

9000XDT

Width X Depth	Nylon Upholstery	Vinyl Upholstery
20" x 18"	8881110376-NLA	8881110384-NLA
	See Note(s) A	See Note(s) A
NOTE: A - Specify upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors and Packaging."		

Solid Folding Seats

(9000XT & XT Recliner)

Width X Depth	Nylon Upholstery	Vinyl Upholstery
14" x 16"	8881110369-NLA	8881110377-NLA
	See Note(s) A	See Note(s) A
14" x 18"	8881110373	8881110381-NLA
	See Note(s) A	See Note(s) A
16" x 16"	8881110370-NLA	8881110378-NLA
	See Note(s) A	See Note(s) A
16" x 18"	8881110374-NLA	8881110382-NLA
	See Note(s) A	See Note(s) A
18" x 16"	8881110371-NLA	8881110379
	See Note(s) A	See Note(s) A
18" x 18"	8881110375-NLA	8881110383-NLA
	See Note(s) A	See Note(s) A
20" x 16"	8881110372	8881110380-NLA
	See Note(s) A	See Note(s) A
20" x 18"	8881110376-NLA	8881110384-NLA
	See Note(s) A	See Note(s) A
NOTE: A - Specify upholstery color when ordering, see category titled, "Frame Finishes, Upholstery Colors and Packaging."		