


Cabinet and Base

Perfecto2V AW

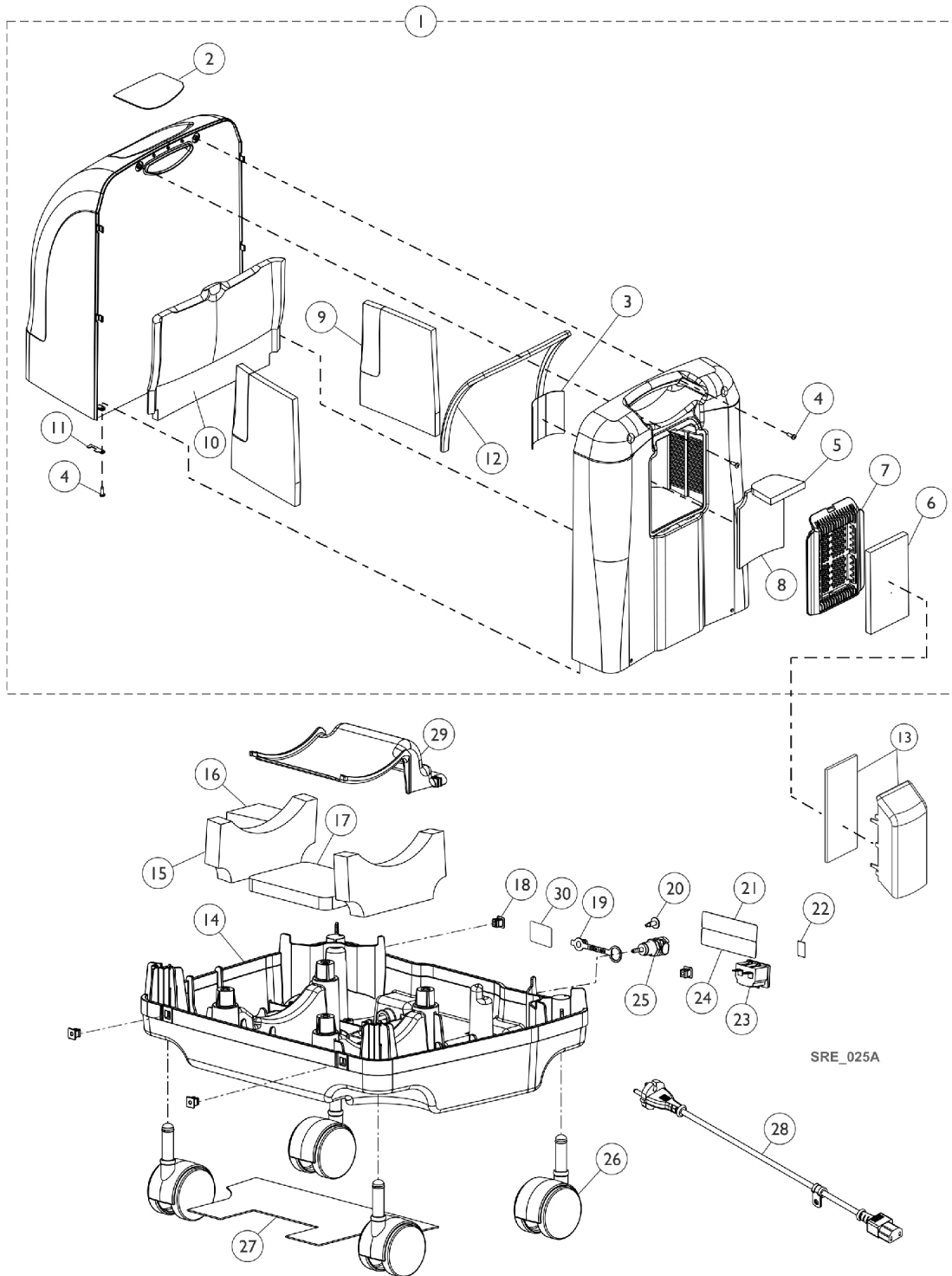

Cabinet and Base Perfecto2V AW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1156046-NLA	NLA - Tech Only - Top Shroud Assembly	1	1
2		1167517	Label, Warning, Perfecto2VAW, English	1	1
2		1167518	Label, Warning, Perfecto2VAW, French	1	1
2		1167519	Label, Warning, Perfecto2VAW, Italian	1	1
2		1167520	Label, Warning, Perfecto2VAW, German	1	1
2		1167521	Label, Warning, Perfecto2VAW, Polish	1	1
2		1167522	Label, Warning, Perfecto2VAW, Dutch	1	1
2		1167523	Label, Warning, Perfecto2VAW, Czech	1	1
2		1167524	Label, Warning, Perfecto2VAW, Portuguese	1	1
2		1167525	Label, Warning, Perfecto2VAW, Danish	1	1
2		1167526	Label, Warning, Perfecto2VAW, Norwegian	1	1
2		1167527	Label, Warning, Perfecto2VAW, Spanish	1	1
2		1167528	Label, Warning, Perfecto2VAW, Greek	1	1
2		1167529	Label, Warning, Perfecto2VAW, Finnish	1	1
2		1171757	Label, Warning, Perfecto2VAW, Swedish	1	1
2		1193251	Label, Warning, Perfecto2VAW, Croatian	1	1
2		1193330	Label, Warning, Perfecto2VAW, Estonian	1	1
2		1193333	Label, Warning, Perfecto2VAW, Lithuanian	1	1
2		1194981	Label, Warning, Perfecto2VAW, Bulgarian	1	1
2		1194983	Label, Warning, Perfecto2VAW, Romanian	1	1
2		1194959	Label, Warning, Perfecto2VAW, Russian	1	1
3		1137267	Label, Diagnostic	1	1
4		1108810	Screw, Hi-Lo Six Lobe Drive (#10-16 x 5/8")	1	4
5		1153695	Foam, Inlet - Ceiling	1	1
6		1143492	Filter, Cabinet	1	1
7		1161831	Door, Cabinet Access, Beige	1	1
8		1153700	Foam, Inlet - Wall	1	1
9		1175250	Foam, Shroud - Side	1	2
10		1175251	Foam, Shroud - Front	1	1
11		1143550	Clip, Retaining	1	2
12		1186157	Foam, Gasket	1	1
13		1156532	Inlet Air Scoop Assembly	1	1
14	B	1143451-NLA	NLA - Tech Only - Base, Concentrator	1	1
15		1153706	Foam, Damping - Base	1	2
16		1154501	Foam, Resonator	1	1
17		1153705	Foam, Damping Outlet Base	1	1
18		2000360	Grommet, Plastic	1	4
19		1108211	Cap, End, Dust, Green	1	1
20		1156530	Screw, Phillips Round Washer Head Tap (8-18 x 1/2")	1	1
21		1167533	Label, PO2VAW Spec 230V	1	1
22		1136841-NLA	NLA - Label, Recycling	1	1
23		1105515-NLA	NLA - Tech Only - Module, Inlet (15A/ 250 VAC)	1	1
24		1143250	Label, European Rep Address	1	1
25		1104151	Fitting, Hose Barb with Nut (1/8")	1	1
26		1110426	Caster Assembly (2")	1	4

Cabinet and Base

Perfecto2V AW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
27		1153704	Damper, Sound Base	1	1
28		1156533-NLA	NLA - Tech Only - Power Cord Assembly w/ Clip	1	1
29		1143444-NLA	NLA - Tech Only - Baffle, Exhaust	1	1
30		1171740	Label, PHTHALATE	1	1
NOTE: A - Includes Cabinet and Items 2-12. B - Includes Base Only.					