


Compressor and Mounting Hardware Between 7/24/14-1/25/19 Perfecto2VAW

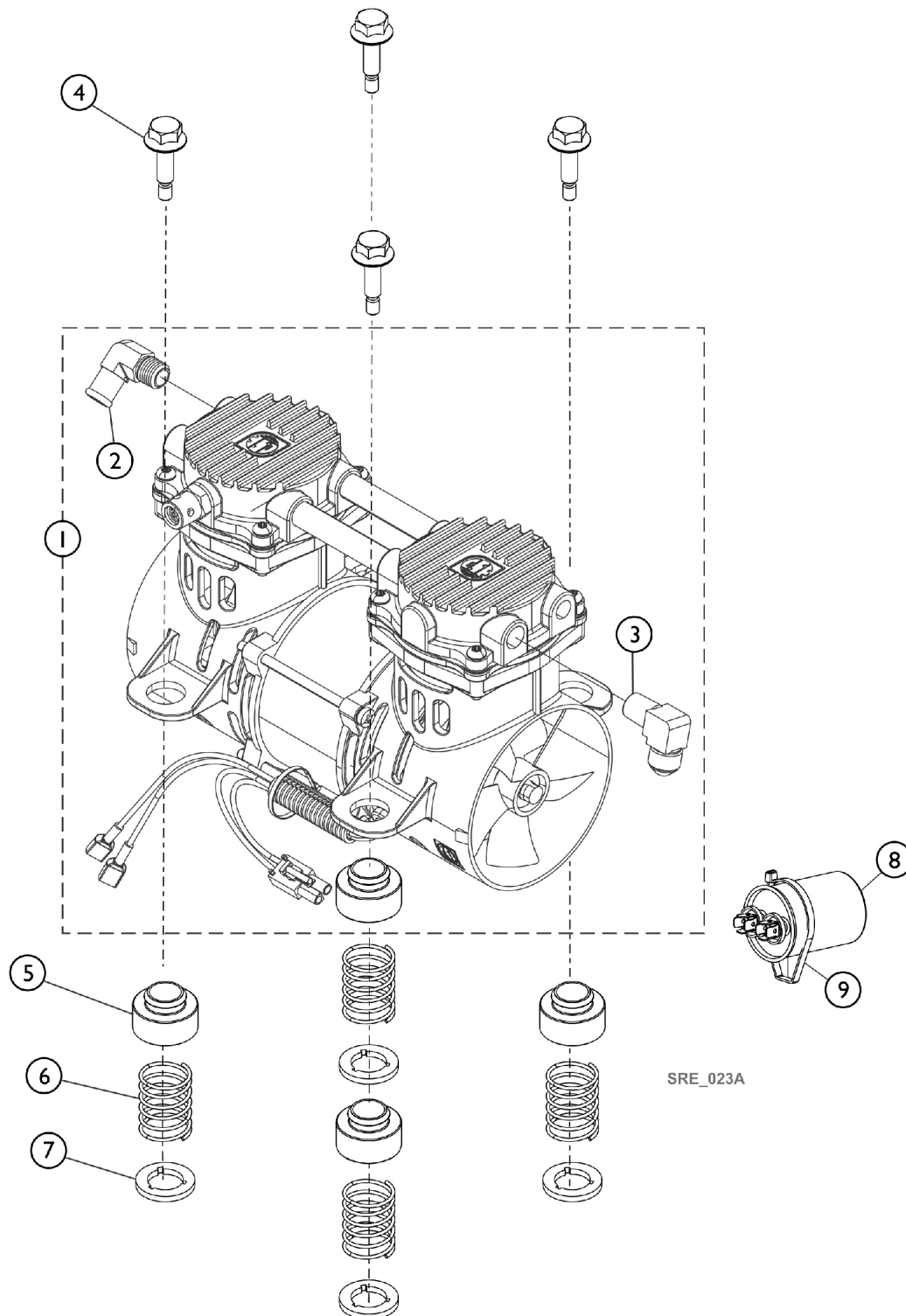

Compressor and Mounting Hardware Between 7/24/14-1/25/19 Perfecto2VAW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1156043-NLA	NLA - Tech Only - Compressor Assembly, PO2V AW	1	1
2		1108819	Fitting, Elbow (1/2" Beaded Hose x 1/4" NPT)	1	1
3		1143489	Elbow, Male Flare	1	1
4		1143461	Screw, Hex Head Flange (1/4-20 x 7/8")	1	4
5		1171968-NLA	NLA - Tech Only - Bushing, Mounting	1	4
6		1185400-NLA	NLA - Tech Only - Spring, Compression (1" x 1-11/32 x 1/16")	1	4
7		1185398-NLA	NLA - Washer, Lower Rubber	1	4
8	B	1131250KIT	Kit, Capacitor (10 MFD)	1	1
			NLA - Tech Only - Capacitor, 10 MFD	1	1
			Tie Wrap (17-1/2" L)	1	1
			Tie Wrap (8" L)	1	2
			Instruction Sheet, Capacitor Replacement	1	1
9		1018239	Tie Wrap (17-1/2" L)	1	1
	1171179	PO2VAW Compressor Rebuild Kit w/ Instructions	1	1	
NOTE: A - Includes (1) Compressor, and Items 2, 3. You will also need to order (1) Tie Wrap Part #1018239. B - Includes (1) 10MFD Capacitor and (2) Tie Wraps and instructions.					