


Sound Box Assembly Before 7/25/14

Perfecto2VAW

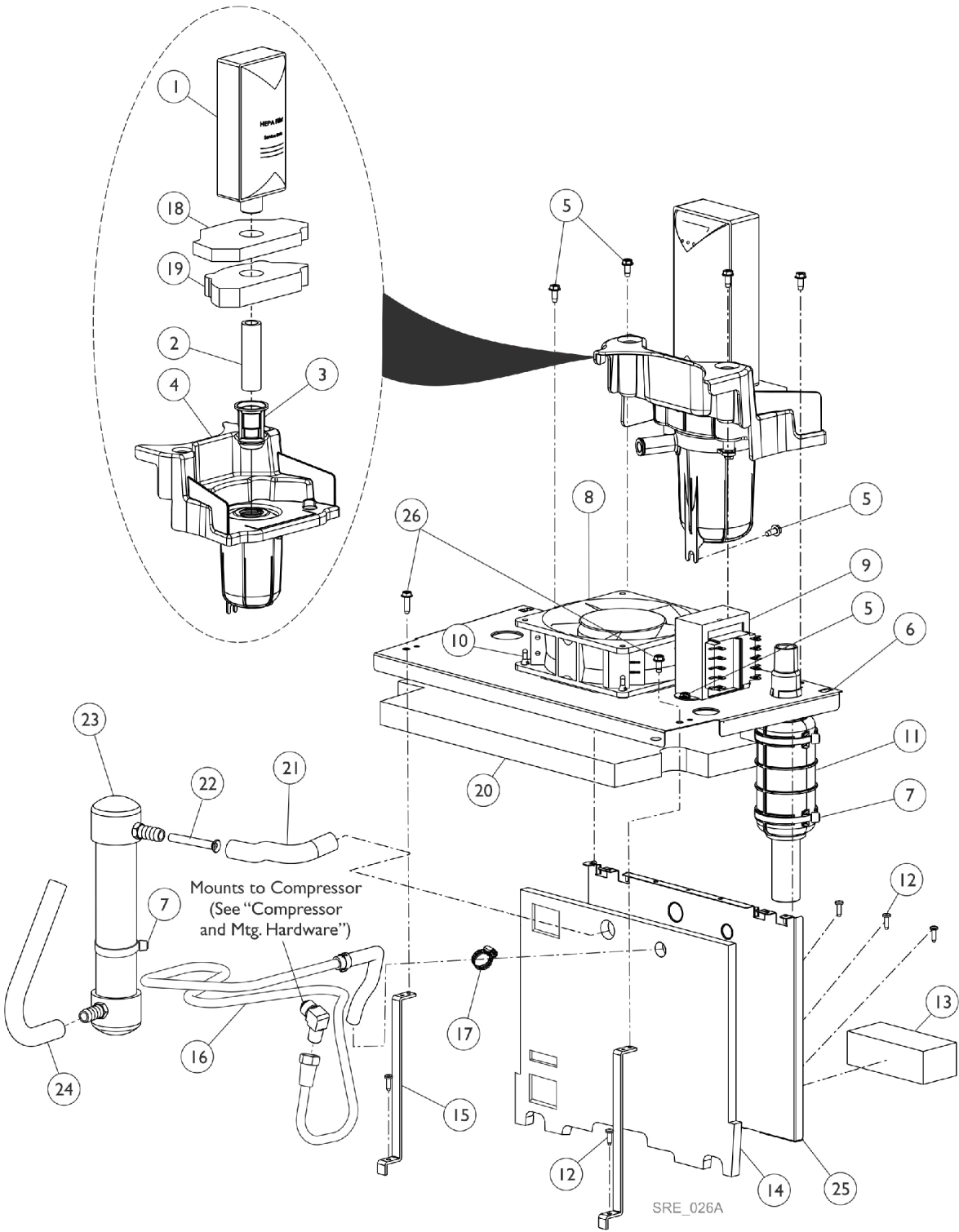

Sound Box Assembly Before 7/25/14 Perfecto2VAW

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1131249	Filter, Inlet Sound Reduced	1	1
2		1153703-NLA	NLA - Tube, Clear (1/2 x 2-3/4")	1	1
3		1106675	Coupling, Inlet Filter	1	1
4		1143435	Resonator Assembly	1	1
5		1128404	Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	7
6		1143473	Soundbox, Horizontal	1	1
7		2001682	Tie Wrap (8" L)	1	4
8		1136943-NLA	NLA - Tech Only - Fan, Exhaust	1	1
9		1153690-NLA	NLA - Tech Only - Transformer Assembly, 20V - PO2AW	1	1
10		1143491	Rivet, Isolator Mount	1	4
11		1143554	Muffler Assembly	1	1
12		1108810	Screw, Hi-Lo Six Lobe Drive (#10-16 x 5/8")	1	5
13		1147917	Mount, Foam Valve 4-Way	1	1
14		1154030	Foam, Soundbox - Vertical	1	1
15		1143474	Gusset, Soundbox	1	2
16		1186169	Heat Exchanger Assembly	1	1
17		1143582	Clamp, Hose Adjustable (3/8")	1	1
18		1153693	Foam, Resonator - Top	1	1
19		1153694	Foam, Resonator - Bottom	1	1
20		1153699	Foam, Soundbox - Ceiling	1	1
21		1153707	Tube, Resonator Inlet Preformed	1	1
22		1153692-NLA	NLA - Tube, Inlet Silencer	1	1
23		1153788	Resonator, Inlet	1	1
24		1153708-NLA	NLA - Tube, Resonator Outlet Preformed	1	1
25		1143445-NLA	NLA - Soundbox, Vertical	1	1
26		60127442	Screw, Slot Hex (10-16)	1	2