


Arms - Adjustable Height, Swingaway

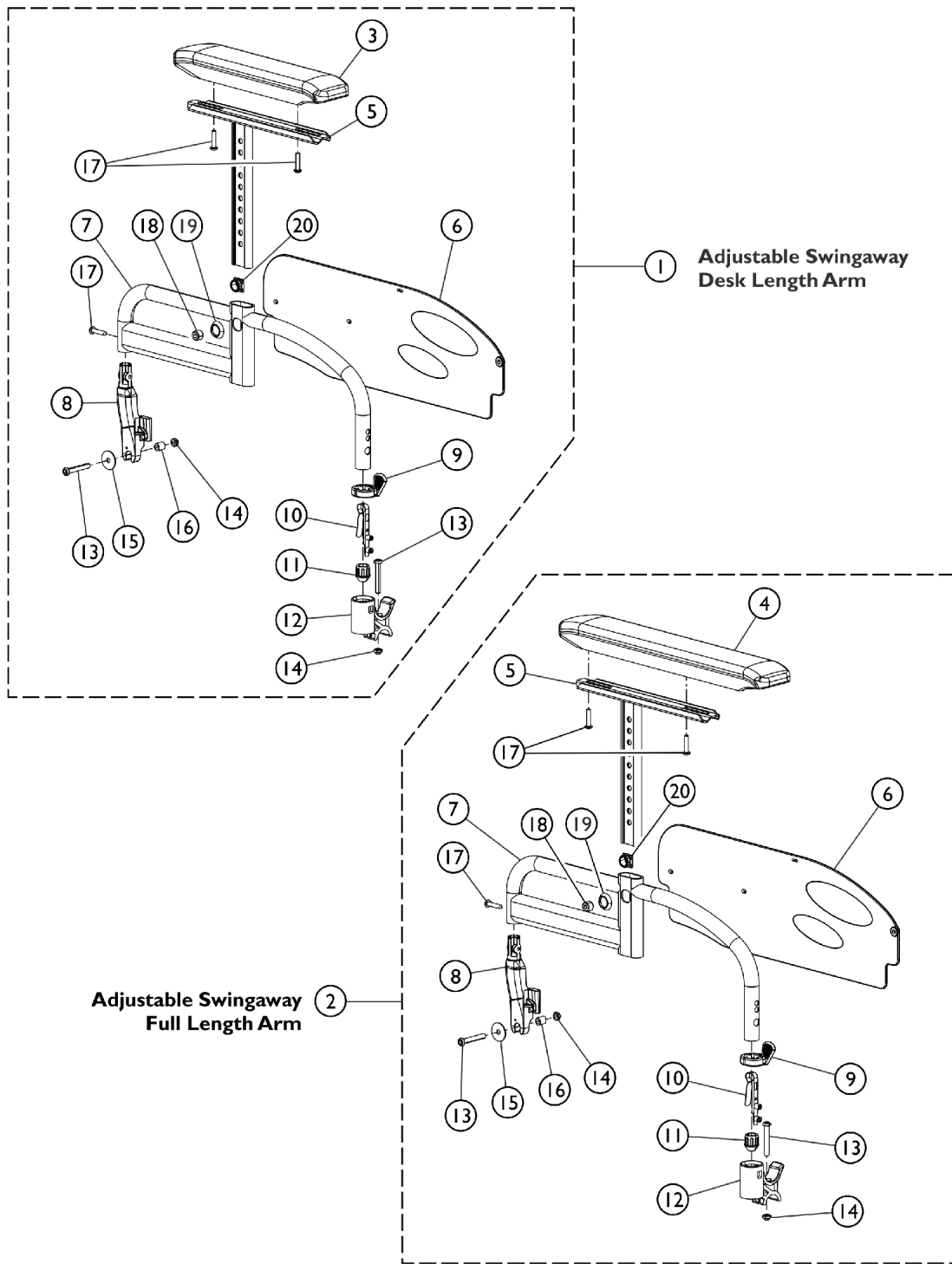

Arms - Adjustable Height, Swingaway

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881169069	Armrest Assembly, Swingaway, Adjustable Height, Desk Length, Color - Right	1	1
3			Pad, Arm, Desk Length	1	1
5			Arm Pad Mounting Assembly - Right	1	1
6			Guard, Clothing	1	1
7			Tube, Armrest, Color - Right	1	1
8			Support, Armrest - Right	1	1
9			Lever, Release	1	1
10			Button, Spring	1	1
11			Plug, Armrest	1	1
12			Support, Armrest, Front	1	1
13			Screw (M6 x 45mm)	1	2
14			NLA - Locknut (M6 x 1)	1	2
15			Washer (6.6 x 22 x 2)	1	1
16			Bushing, Black	1	1
17			Screw, Countersunk (M4.8)	1	1
18			Screw, Socket Head (M12 x 12.5)	1	1
19			Bushing, Armrest	1	1
20			Nut, Square, Armrest	1	1
8		1425466	Support, Armrest - Right	1	1
1	A	8881169070	Armrest Assembly, Swingaway, Adjustable Height, Desk Length, Color - Left	1	1
3			Pad, Arm, Desk Length	1	1
5			Arm Pad Mounting Assembly - Left	1	1
6			Guard, Clothing	1	1
7			Tube, Armrest, Color - Left	1	1
8			Support, Armrest - Left	1	1
9			Lever, Release	1	1
10			Button, Spring	1	1
11			Plug, Armrest	1	1
12			Support, Armrest, Front	1	1
13			Screw (M6 x 45mm)	1	2
14			NLA - Locknut (M6 x 1)	1	2
15			Washer (6.6 x 22 x 2)	1	1
16			Bushing, Black	1	1
17			Screw, Countersunk (M4.8)	1	2
18			Screw, Socket Head (M12 x 12.5)	1	1
19			Bushing, Armrest	1	1
20			Nut, Square, Armrest	1	1
8		1425466	Support, Armrest - Right	1	1
2	B	8881169071	Armrest Assembly, Swingaway, Adjustable Height, Full Length, Color - Right	1	1
4			Arm Pad, Full Length	1	1
5			Arm Pad Mounting Assembly - Right	1	1
6			Guard, Clothing	1	1
7			Tube, Armrest, Color - Right	1	1
8			Support, Armrest - Right	1	1
9			Lever, Release	1	1

Arms - Adjustable Height, Swingaway

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
10			Button, Spring	1	1
11			Plug, Armrest	1	1
12			Support, Armrest, Front	1	1
13			Screw (M6 x 45mm)	1	2
14			NLA - Locknut (M6 x 1)	1	2
15			Washer (6.6 x 22 x 2)	1	1
16			Bushing, Black	1	1
17			Screw, Countersunk (M4.8)	1	3
18			Screw, Socket Head (M12 x 12.5)	1	1
19			Bushing, Armrest	1	1
20			Nut, Square, Armrest	1	1
8		1425466	Support, Armrest - Right	1	1
2	B	8881169072	Armrest Assembly, Swingaway, Adjustable Height, Full Length, Color - Left	1	1
4			Arm Pad, Full Length	1	1
5			Arm Pad Mounting Assembly - Left	1	1
6			Guard, Clothing	1	1
7			Tube, Armrest, Color - Left	1	1
8			Support, Armrest - Left	1	1
9			Lever, Release	1	1
10			Button, Spring	1	1
11			Plug, Armrest	1	1
12			Support, Armrest, Front	1	1
13			Screw (M6 x 45mm)	1	2
14			NLA - Locknut (M6 x 1)	1	2
15			Washer (6.6 x 22 x 2)	1	1
16			Bushing, Black	1	1
17			Screw, Countersunk (M4.8)	1	3
18			Screw, Socket Head (M12 x 12.5)	1	1
19			Bushing, Armrest	1	1
20			Nut, Square, Armrest	1	1
8		1425466	Support, Armrest - Right	1	1
8		1426617	Support, Armrest - Left	1	1
NOTE: A - Includes items 3, 5, and 6 - 20. B - Includes items 4 - 20.					