
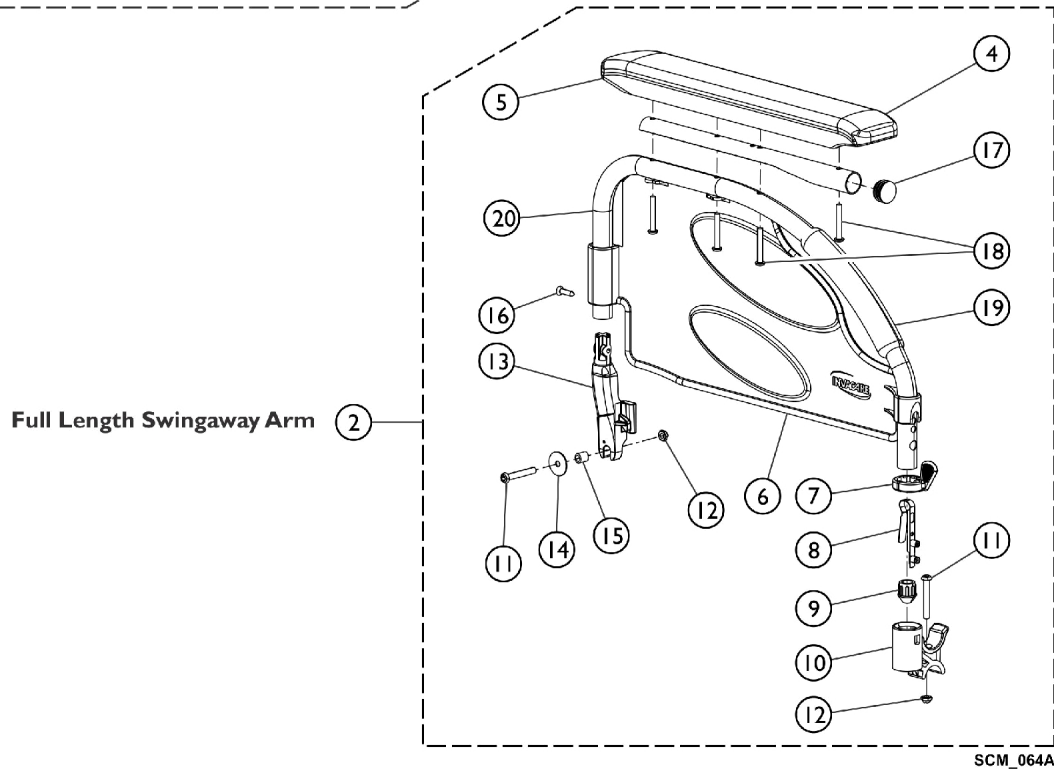


# Arms - Fixed Height, Swingaway

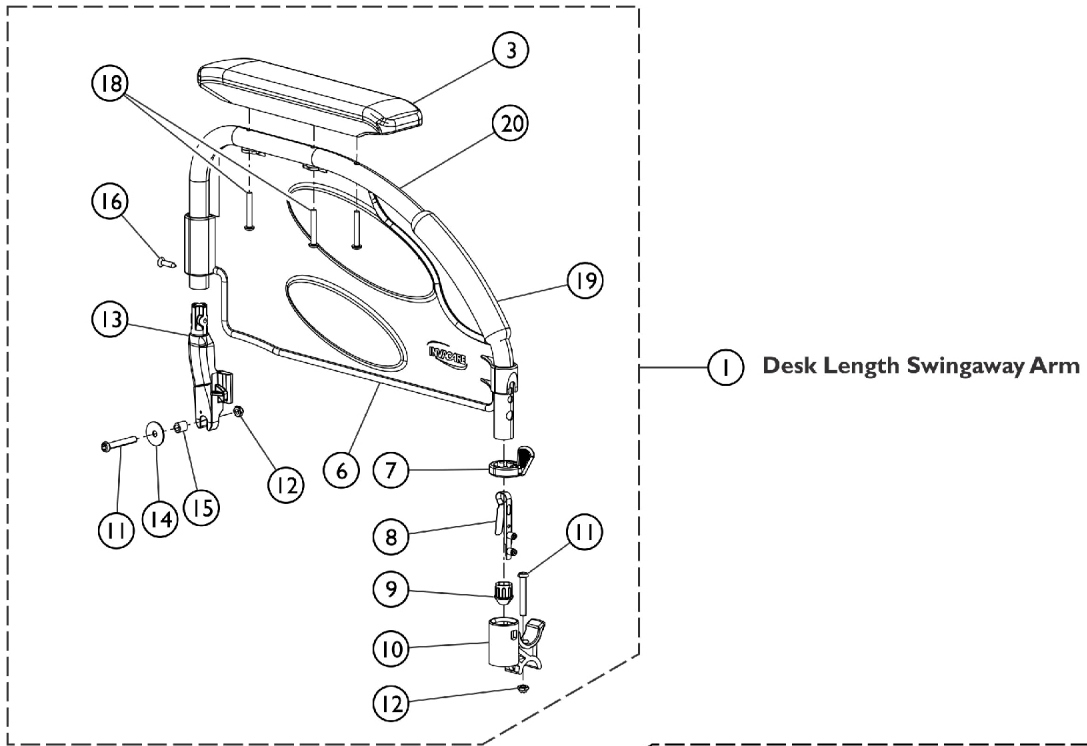

## Arms - Fixed Height, Swingaway

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881169065	Armrest Assembly, Swingaway, Fixed Height, Desk Length, Color - Right	1	1
3			Arm Pad, Desk Length	1	1
6			Guard, Clothing, Desk Length - Right	1	1
7			Lever, Release	1	1
8			Button, Spring	1	1
9			Plug, Armrest	1	1
10			Support, Armrest, Front	1	1
11			Screw (M6 x 45mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2
13			Support, Armrest - Right	1	1
14			Washer (6.6 x 22 x 2)	1	1
15			Bushing, Black	1	1
16			Screw, Countersunk (M4.8)	1	1
18			Screw, Socket, Button Head (M5 x 35mm)	1	3
19			Grip, Foam Armrest	1	1
20			Tube, Armrest, Color	1	1
20		1169061	Tube, Armrest, Color	1	1
1	A	8881169066	Armrest Assembly, Swingaway, Fixed Height, Desk Length, Color - Left	1	1
3			Arm Pad, Desk Length	1	1
6			Guard, Clothing, Desk Length - Left	1	1
7			Lever, Release	1	1
8			Button, Spring	1	1
9			Plug, Armrest	1	1
10			Support, Armrest, Front	1	1
11			Screw (M6 x 45mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2
13			Support, Armrest - Left	1	1
14			Washer (6.6 x 22 x 2)	1	1
15			Bushing, Black	1	1
16			Screw, Countersunk (M4.8)	1	1
18			Screw, Socket, Button Head (M5 x 35mm)	1	3
19			Grip, Foam Armrest	1	1
20			Tube, Armrest, Color	1	1
13		1426617	Support, Armrest - Left	1	1
20		1169061	Tube, Armrest, Color	1	1
2	B	8881169067	Armrest Assembly, Swingaway, Fixed Height, Full Length, Color - Right	1	1
4			Arm Pad, Full Length	1	1
5			Tube, Support, Armpad, Full Length	1	1
6			Guard, Clothing, Desk Length - Right	1	1
7			Lever, Release	1	1
8			Button, Spring	1	1
9			Plug, Armrest	1	1
10			Support, Armrest, Front	1	1
11			Screw (M6 x 45mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2

## Arms - Fixed Height, Swingaway

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
13			Support, Armrest - Right	1	1
14			Washer (6.6 x 22 x 2)	1	1
15			Bushing, Black	1	1
16			Screw, Countersunk (M4.8)	1	1
17			Button, Plug, Reflector (22mm)	1	1
18			Screw, Socket, Button Head (M5 x 35mm)	1	4
19			Grip, Foam Armrest	1	1
20			Tube, Armrest, Color	1	1
13		1425466	Support, Armrest - Right	1	1
20		1169061	Tube, Armrest, Color	1	1
2	B	8881169068	Armrest Assembly, Swingaway, Fixed Height, Full Length, Color - Left	1	1
4			Arm Pad, Full Length	1	1
5			Tube, Support, Armpad, Full Length	1	1
6			Guard, Clothing, Desk Length - Left	1	1
7			Lever, Release	1	1
8			Button, Spring	1	1
9			Plug, Armrest	1	1
10			Support, Armrest, Front	1	1
11			Screw (M6 x 45mm)	1	2
12			NLA - Locknut (M6 x 1)	1	2
13			Support, Armrest - Left	1	1
14			Washer (6.6 x 22 x 2)	1	1
15			Bushing, Black	1	1
16			Screw, Countersunk (M4.8)	1	1
17			Button, Plug, Reflector (22mm)	1	1
18			Screw, Socket, Button Head (M5 x 35mm)	1	4
19			Grip, Foam Armrest	1	1
20			Tube, Armrest, Color	1	1
6		1491219	Guard, Clothing, Desk Length - Right	1	1
6		1491220	Guard, Clothing, Desk Length - Left	1	1
13		1425466	Support, Armrest - Right	1	1
13		1426617	Support, Armrest - Left	1	1
19		0572-02851	Grip, Foam Armrest	1	1
20		1169061	Tube, Armrest, Color	1	1

NOTE: A - Includes items 3, 6 - 16, 18-20.  
 B - Includes items 4-20.