

Transport Option Clarification Reference

TRRO Option is an On-Chair option only.

About Transport Ready Package

TRRO includes four factory-installed transport brackets and a wheelchair anchored pelvic belt. **TRRO** has been crash-tested in accordance with ANSI/RESNA WC-4:2017, Section 19 Frontal Impact Test requirements for wheelchairs with a 225 lb. crash dummy, which corresponds to a person with weight of over 300 lbs. for Adult seat sizes.

TRBKTS includes four factory-installed wheelchair transport brackets. **TRBKTS** has not been crash-tested in accordance with ANSI/RESNA WC-4:2017 Section 19 Frontal Impact Test requirements for wheelchairs.

Only use the transport brackets included with **TRRO** for the purpose described in the appropriate Owner's Operator and Maintenance Manual.

Battery support brackets **MUST** be installed at all times. Otherwise, the wheelchair will not be ANSI/RESNA WC-4:2017 Section 19 compliant. Refer to Removing/Installing the Batteries From/Into the Wheelchair in the appropriate Owner's Operator and Maintenance Manual.

As of this date, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type. It is Invacare's position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the auto industry. Invacare cannot and does not recommend any wheelchair transportation systems.

NOTE: ANSI = American National Standards Institute, RESNA = Rehabilitation Engineering and Assistive Technology Society of North America.

NOTE: For additional information refer to appropriate Owner's Operator and Maintenance Manual.

SHEP_038C