
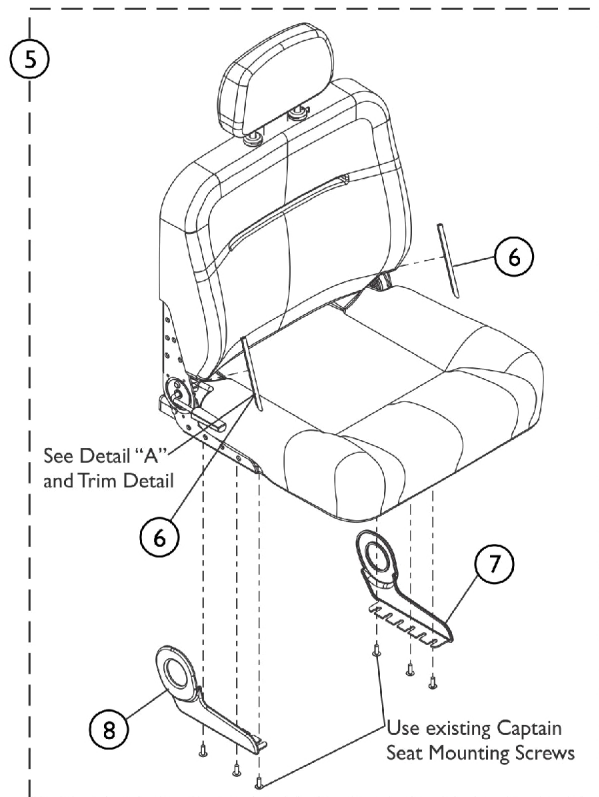
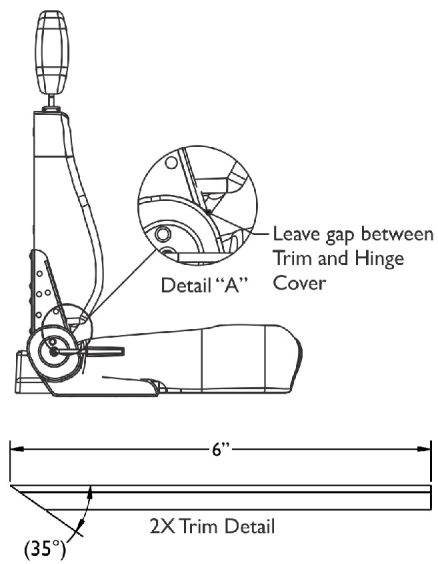
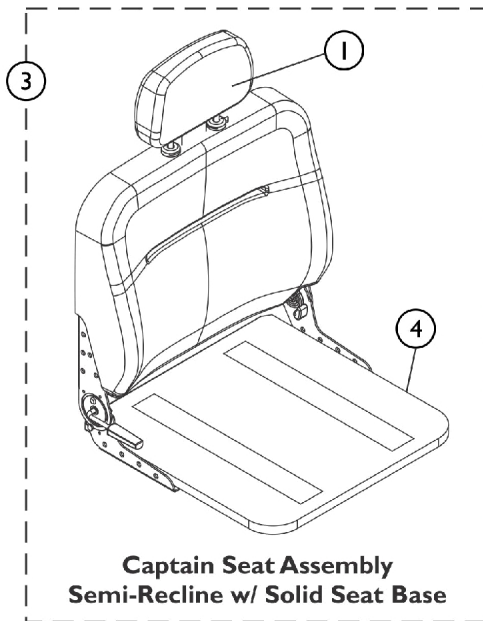


Captain Seat Assembly

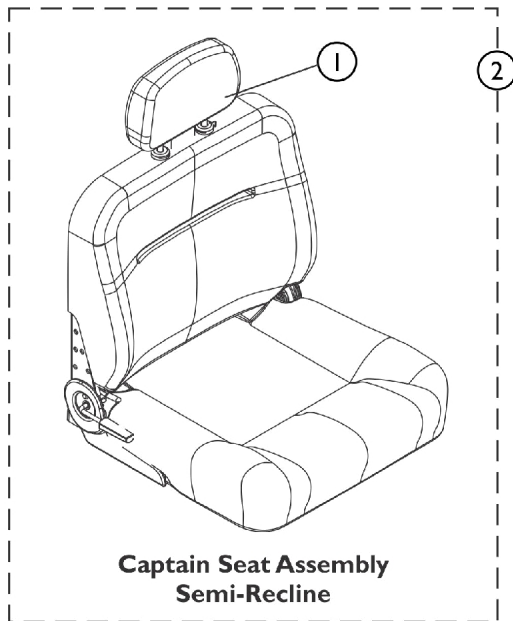

SHEP_302A

Captain Seat Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1108408	Extension, Backrest, Captain Seat, Grey Vinyl	1	1
2		See Chart	Captain Seat (Semi-Recline) Assembly	1	1
3		See Chart	Captain Seat (Semi-Recline) w/ Solid Seat Base Assembly	1	1
4	E	1159739	Base, Solid, Captain Seat, Grey Vinyl (18" W)	1	1
4	E	1159740	Base, Solid, Captain Seat, Grey Vinyl (20" W)	1	1
4	E	1159741	Base, Solid, Captain Seat, Grey Vinyl (22" W)	1	1
4	E,F	TBD24	Base, Solid, Captain Seat, Grey Vinyl (24" W)	1	1
	A,B	1116634	Bracket, Hinge w/ Lever, Semi-Recline - Right	1	1
	A,B	1116635	Bracket, Hinge w/o Lever, Semi-Recline - Left	1	1
	A,C	5000136	Screw, Phillips Truss Head (1/4-20 x 3/4")	1	14
5	D	60104868	Kit, Hinge Cover Assembly, Captain Seat	1	1
6			Trim, Edge w/ Adhesive (12" L)	1	1
7			Cover, Hinge, Captain Seat, Left	1	1
8			Cover, Hinge, Captain Seat, Right	1	1
NOTE: A - Not Shown B - Hinge Brackets used for ALL Semi-Recline Seat models. For attaching Hinge Bracket, see part number 5000136. C - Screw used for attaching Hinge Bracket(s) 1116634 and 1116635. Total of (7) required for each Bracket. D - Includes items 4-6. Each Hinge uses 6" Long Edge Trim. Cut Edge Trim before application. Hinge Cover Assembly comes standard on TRRO Option only. E - Seat Hinges are not included. F - TBD24 = To Be Determined. Currently not available.					

Item #2 Captain Seat (Semi-Recline) Assembly

(Includes Item #1 Backrest Extension)

Upholstery Type	Captain Seat with Backrest 16" Wide	Captain Seat with Backrest 18" Wide	Captain Seat with Backrest 20" Wide	Captain Seat with Backrest 22" Wide	Captain Seat with Backrest 24" Wide
All Grey Vinyl	1150810	1109242	1159891	1109244	1123360
NOTE: Semi-recline Captain Seat reclines approximately 90-114 degrees. Adjustable depths are (16, 17, 18" deep for 16 & 18" wide seats) and (18, 19, 20" deep for 20, 22 and 24" wide seats). Recline release levers are standard on the right side. Left side release levers can be ordered as a special with a new Captain Seat.					

#3 Captain Seat (Semi-Recline) w/ Solid Seat Base Assembly

(Includes Item #1 Backrest Extension)

Upholstery Type	Captain Seat w/ Solid Seat and Backrest 16" Wide	Captain Seat w/ Solid Seat and Backrest 18" Wide	Captain Seat w/ Solid Seat and Backrest 20" Wide	Captain Seat w/ Solid Seat and Backrest 22" Wide	Captain Seat w/ Solid Seat and Backrest 24" Wide
All Grey Vinyl	1150819	1150820	1159886	1150822	TBD See Note(s) A
NOTE: A - TBD = To Be Determined. Currently not available. NOTE: Semi-recline Captain Seat reclines approximately 90-114 degrees. Adjustable depths are (16, 17, 18" deep for 16 & 18" wide seats) and (18, 19, 20" deep for 20 & 22" wide seats). Recline release levers are standard on the right side. Left side release levers can be ordered as a special with a new Captain Seat.					