


Assist Rail

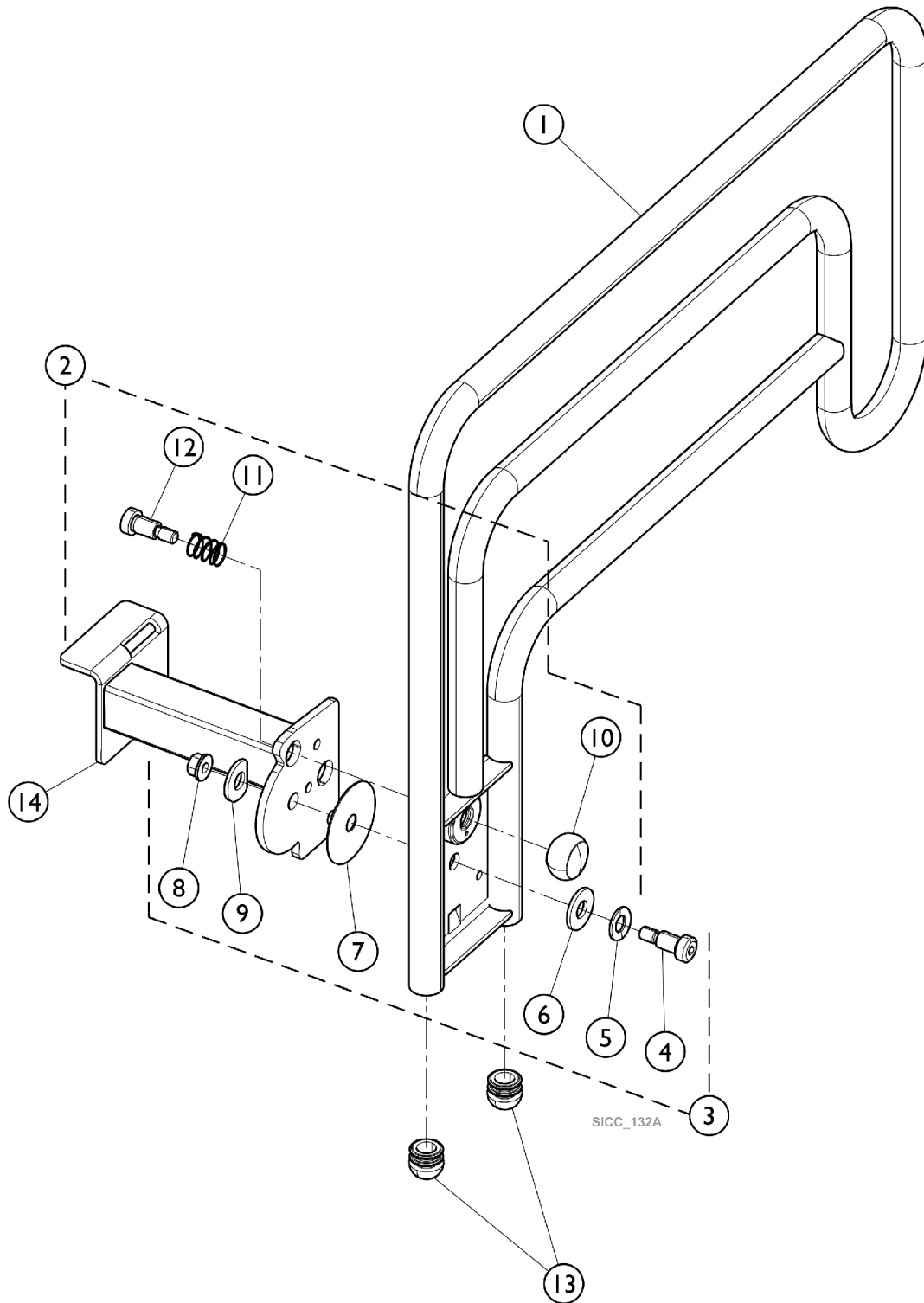

Assist Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHCSRLAS-QSP	Assist Rail Assembly (36") (Pair)	1	1
1	A	IHCSRLAS-39	Assist Rail Assembly (39") (Pair)	1	1
1	A	IHCSRLAS-42	Assist Rail Assembly (42") (Pair)	1	1
2	B	1189259	Kit, Knob - Latch	1	1
10			Knob, Black Ball	1	1
11			Spring, Compression	1	1
12			Bolt, Shoulder, Socket Head (1/2-12)	1	1
3	C	1189260	Kit, Assist Rail Mounting	1	1
4			Bolt, Shoulder, Socket (1/2-34)	1	1
5			Washer, Bevel	1	1
6			Washer, Flat	1	1
7			Washer, Nylon (1/2" x 2-1/2")	1	1
8			Nut, Flanged (3/8)	1	1
9			Washer, Flat Edge	1	1
11		1186856	Spring, Compression (3/4 x 1 x 3/64")	1	2
13		1165068	Package, End Plug	10	2

NOTE: A - Includes items 2-14 Right and Left Assist Rail Assemblies.
 B - Includes items 10, 11, and 12.
 C - Includes items 4 - 9.

Item #14 highlighted in the illustration is non-saleable. Order Item #1 Assist Rail Assemblies.