



Fork Assembly


(Options 4SPLT, 4ALUM2, 51DC, 5ALUM2, 5SPLT, 61DC, 6ALUM2)

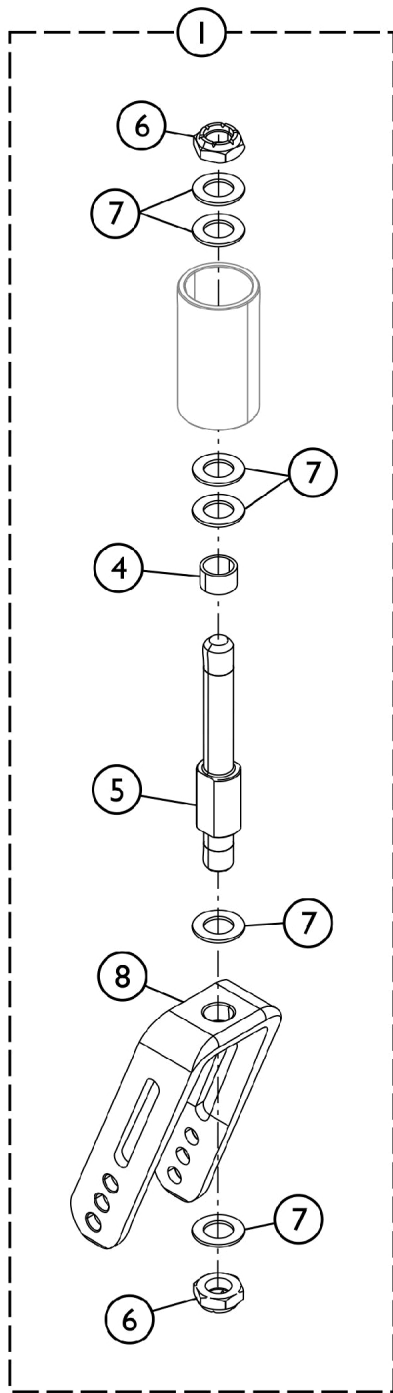

Fork and Stem
5" Fork w/ Long Stem

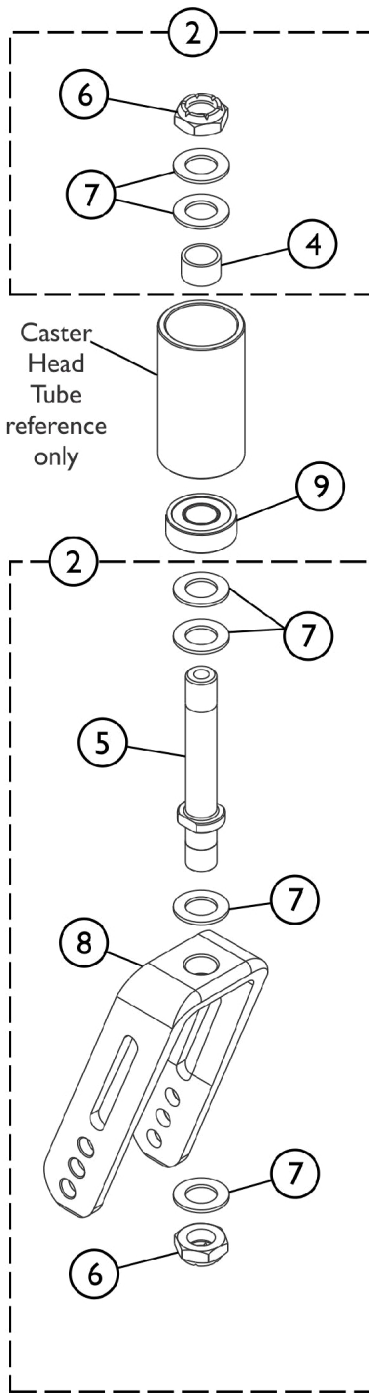

Fork and Stem
5" Fork w/ Short Stem

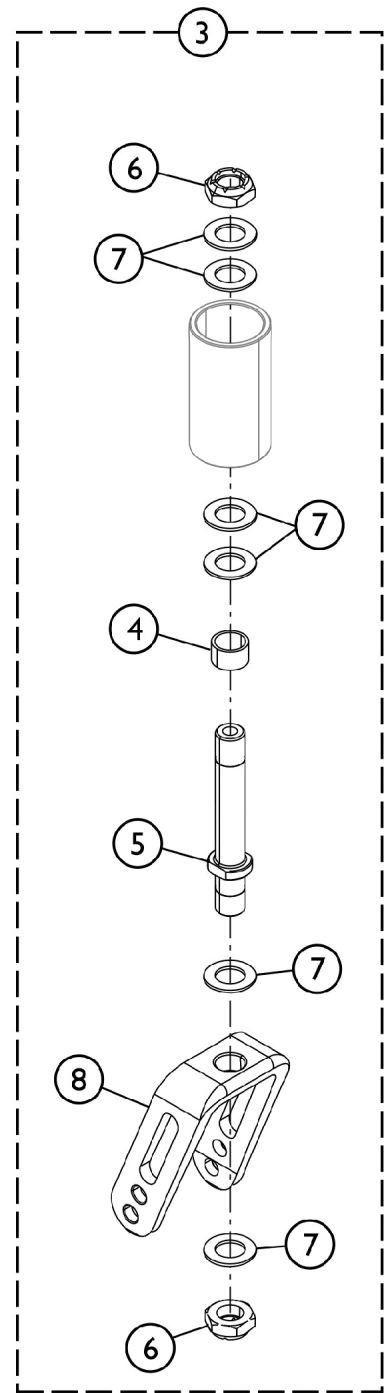

Fork and Stem
3" Fork w/ Short Stem

Fork Assembly

(Options 4SPLT, 4ALUM2, 51DC, 5ALUM2, 5SPLT, 61DC, 6ALUM2)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		See Chart	Fork Assembly - For 4", 5", 6" Casters - Where Used	1	1
1	A	1192573	Kit, Fork (5") w/ Long Stem	1	1
4			Spacer (1/2 x 5/8 x 3/8")	1	1
5			Stem, Fork, Long	1	1
6			Locknut (1/2-20)	1	2
7			Washer, Plain (1/2 x 7/8 x 1/16)	1	6
8			NLA - Fork, Caster	1	1
2	C	1118545	Kit, Fork (5") w/ Short Stem	1	1
4			Spacer (1/2 x 5/8 x 3/8")	1	1
5			Stem, Fork, Short	1	1
6			Locknut (1/2-20)	1	2
7			Washer, Plain (1/2 x 7/8 x 1/16)	1	6
8			NLA - Fork, Caster	1	1
9		40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	1
3	A	1192574	Kit, Fork (3") w/ Short Fork	1	1
4			Spacer (1/2 x 5/8 x 3/8")	1	1
5			Stem, Fork, Short	1	1
6			Locknut (1/2-20)	1	2
7			Washer, Plain (1/2 x 7/8 x 1/16)	1	6
8			Fork, Caster	1	1
	B	1088523	Plug Button, Head Tube, Black	1	1

NOTE: A - Kit includes items 4, 5, 6, 7 and item 8.
 B - Not Shown in illustration.
 C - Kit includes items 4, 5, 6, 7 and item 8. Must order item #9 as well to make up a complete Fork Kit (5") w/ Short Fork Stem.

Fork Assembly - For 4", 5", 6" Casters - Where Used

Caster Option	Rear Wheel Option	Seat to Floor Dimensions	Stem Length	Part Number	Part Number
4SPLT 4ALUM2	12", 16", 18" (All Tires)	12-1/2, 13"	3" Fork w/ Short Stem	1192574	N/A
4SPLT 4ALUM2	12", 16", 18" (All Tires)	13-1/2, 14-1/2"	5" Fork w/ Short Stem	1118545 See Note(s) A	40117X021 See Note(s) A
4SPLT 4ALUM2	12", 16", 18" 20", 22" (All Tires)	15", 15-1/2", 16"	5" Fork w/ Long Stem	1192573	N/A
51DC 5ALUM2 5SPLT	12", 16", 18" (All Tires)	13-1/2"	3" Fork w/ Short Stem	1192574	N/A
51DC 5ALUM2 5SPLT	12", 16", 18" (All Tires)	14", 15"	5" Fork w/ Short Stem	1118545 See Note(s) A	40117X021 See Note(s) A
51DC 5ALUM2 5SPLT	12", 18", 20" 22", 24" (All Tires)	15-1/2", 16", 16-1/2"	5" Fork w/ Long Stem	1192573	N/A

Fork Assembly

(Options 4SPLT, 4ALUM2, 51DC, 5ALUM2, 5SPLT, 61DC, 6ALUM2)

Fork Assembly - For 4", 5", 6" Casters - Where Used

Caster Option	Rear Wheel Option	Seat to Floor Dimensions	Stem Length	Part Number	Part Number
61DC 6ALUM2	12", 18", 20" 22", 24" (All Tires)	14-1/2", 16"	5" Fork w/ Short Stem	1118545 See Note(s) A	40117X021 See Note(s) A
61DC 6ALUM2	12", 18", 20" 22", 24" (All Tires)	16-1/2", 17"	5" Fork w/ Long Stem	1192573	N/A

NOTE: A - To order a complete Fork Assembly, order both part numbers 1118545 and 40117X021.