


Fork Support, Swing Away Frame

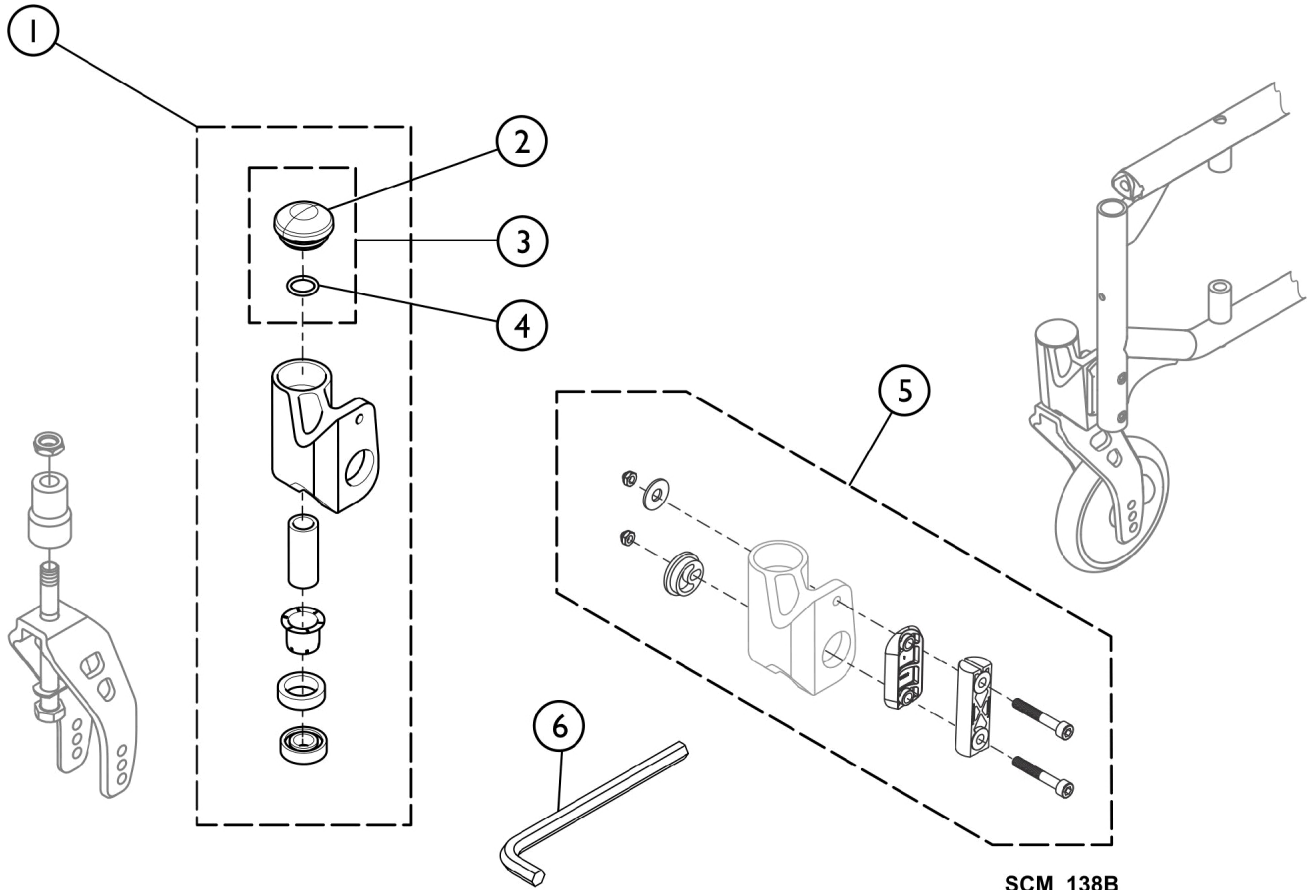

SCM_138B

Fork Support, Swing Away Frame

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	SP1584023-000 5	Kit, Fork Support, w/o Accent, Swing Away Frame - Left	1	1
1	A	SP1586261-000 5	Kit, Fork Support, w/o Accent, Swing Away Frame - Right	1	1
1	A	SP1584023-064 1	Kit, Fork Support, Blue Accent, Swing Away Frame - Left	1	1
1	A	SP1586261-064 1	Kit, Fork Support, Blue Accent, Swing Away Frame - Right	1	1
1	A	SP1584023-064 2	Kit, Fork Support, Purple Accent, Swing Away Frame - Left	1	1
1	A	SP1586261-064 2	Kit, Fork Support, Purple Accent, Swing Away Frame - Right	1	1
2		SP1589193	Caster Housing Cap, Black	1	1
3		SP1585342-064 1	Caster Housing Cap, Blue Accent	1	1
4		SP1483872	O-Ring (15,6 x 2,4 NITRILE 70 SH)	1	1
5		SP1584024	Kit, Adjustment Fork Support, Swing Away Frame	1	1
6		SP1586506	Short Metric Hex Key (5 mm)	1	1

NOTE: A - Includes items 2, 3, 4, and 5.