


Front Riggings - Footrest Support Assembly

T93H Series After 5/15/19

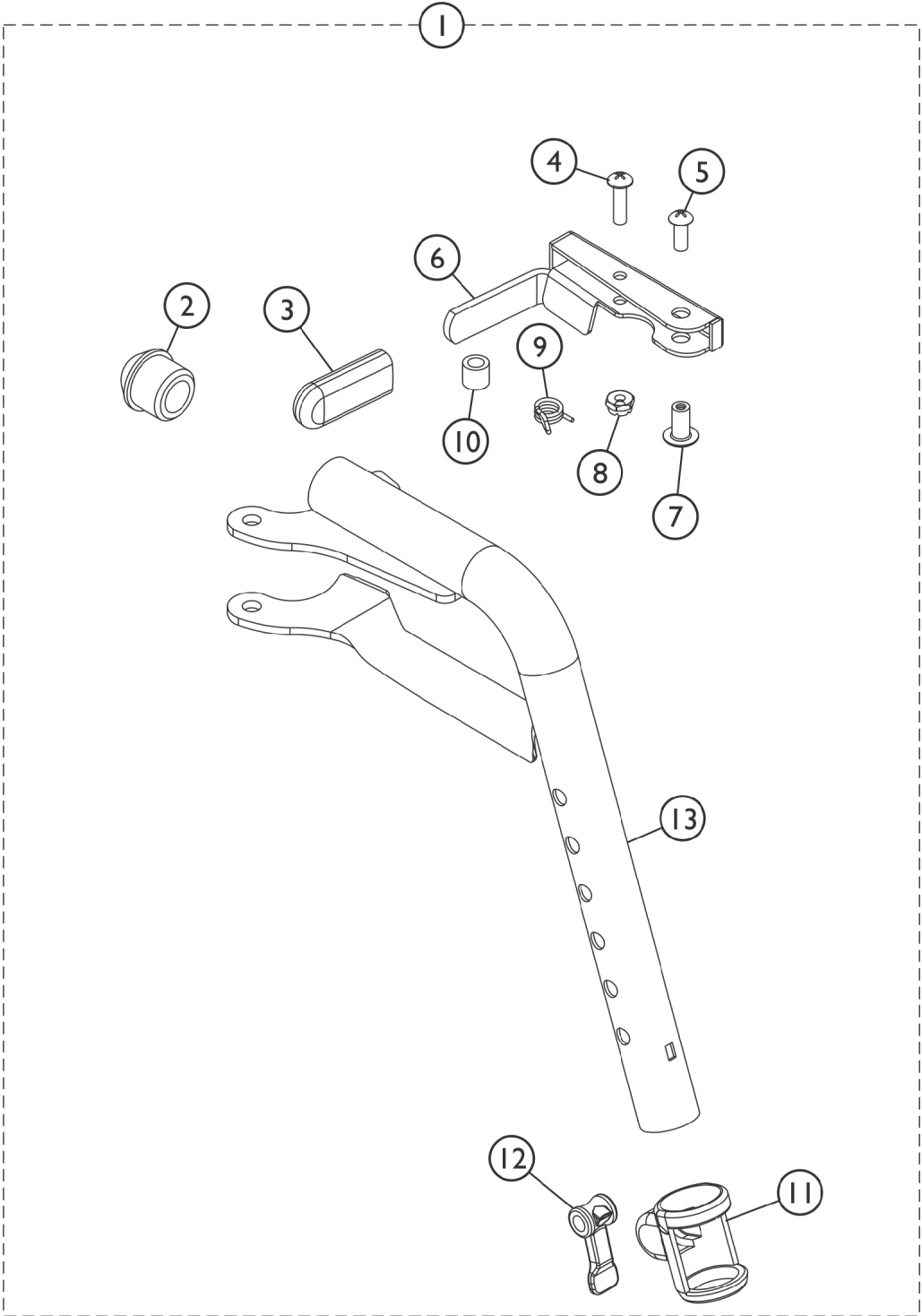

SSM_038A

Front Riggings - Footrest Support Assembly

T93H Series After 5/15/19

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1192525	Footrest Support Assembly, Hemi, Black - Right	1	1
1	A,B	1192526	Footrest Support Assembly, Hemi, Black - Left	1	1
2		1020719	Anti-Rattle, Black	1	1
3		1006490-NLA	NLA - Tip, Plastic, Black	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
6		1180757	Handle, Release, Black - Right	1	1
6		1180758	Handle, Release, Black - Left	1	1
7		1028846	NLA - Rivnut (#10-32)	1	1
8		1025735	Locknut (#10-32)	1	1
10		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
11		1048085	Housing, Cam	1	1
12		1048086	Lever, Cam, Black	1	1
NOTE: A - Includes Items 2 thru 13. Item #9 is not available. Order item #1 as a replacement.					
B - NOTE: Footrest Support Tube item #13 is not available, order item #1 as a replacement.					