



Drive Wheels for Non-Free Wheel Hubs

4 Pole Motors (US and Canada After 8/16/10)

NOTE: New drive wheel mounting hardware is required when converting from (14" x 3") to (14" x 4") drive wheels.

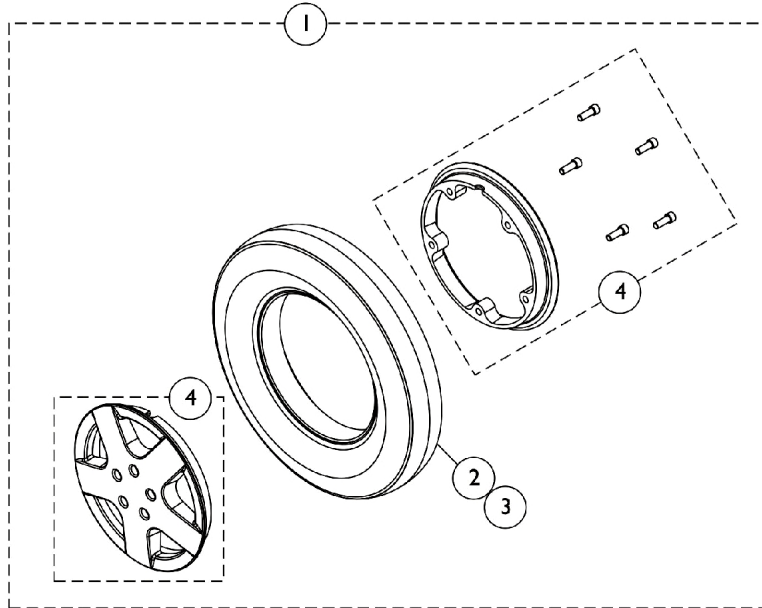

14" x 3" Drive Wheel Assembly

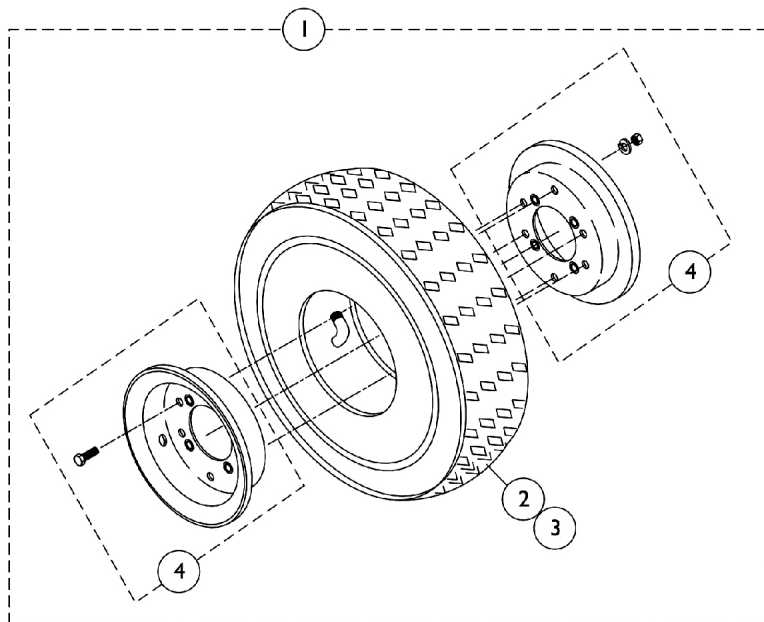

14" x 4" Drive Wheel Assembly

Drive Wheels for Non-Free Wheel Hubs

4 Pole Motors (US and Canada After 8/16/10)

NOTE: New drive wheel mounting hardware is required when converting from (14" x 3") to (14" x 4") drive wheels.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-3		See Chart	#'s 1, 2 and 3 Drive Wheels for Non-Free Wheel Hubs	1	1
4	A	1121707-NLA	NLA - Kit, 8" Rim w/ Hardware (14" x 3" Wheel)	1	1
4	B	1042453-NLA	NLA - Kit, 6" Rim w/ Hardware (14" x 4" Wheel)	1	1
<p>NOTE: A - Sold complete. Includes the Rims, (5) ea. Socket Head Screws w/ Patch (5/16-18 x 7/8"). Part #1121707 is used item #1 1144653 and 1144660 Wheel Assemblies only.</p> <p>B - Sold complete. Includes the Rims, (6) ea. Hex Head Screws (5/16-24 x 5/8"), (6) ea. Lock Washers (5/16 x 9/16 x 1/16") and (6) ea. Hex Nuts (5/16-24)</p>					

Item #'s 1, 2 and 3 Wheel Components

Wheel Size	Item #1 Wheel Complete Pneumatic	Item #1 Wheel Complete w/ Flat Free	Item #2 Tire	Item #2 Tire w/ Flat Free	Item #3 Tube
14" x 3"	1144653-NLA See Note(s) C	1144660-NLA See Note(s) F, A	1039652	1141342	1039651
14" x 3"	1176911 See Note(s) D	1178834	1039652	1141342 See Note(s) G	1039651
14" x 3"	1176609 See Note(s) E	1167685 See Note(s) B	1039652	1141342 See Note(s) G	1039651
14" x 3"	60102668 See Note(s) H	60102669 See Note(s) H	N/A	N/A	N/A
14" x 4"	1042476	1042477	1042474	1042478	1042475
<p>NOTE: A - Used Before 3/1/12 for 300 Lb. user weight models and used Before 4/11/12 for 301-400 Lb. user weight models.</p> <p>B - Used Between 2/29/12-07/07/19 for 300 Lb. user weight models and used Between 4/10/12-07/07/19 for 301-400 Lb. user weight models.</p> <p>C - Used Before 4/10/12. No Longer Available. Use Part #1176911 as a replacement.</p> <p>D - Used 4/10/12-6/3/12. Part #1176911 14" x 3" Wheel Assembly has Grey Tire/Black Rim and Metric Hardware. This item is sold complete w/ the Rim Hardware. Tire and Tube are still available for individual sale.</p> <p>E - Used Between 6/4/12-07/07/19. Part #1176609 14" x 3" Wheel Assembly has Grey Tire/Silver Rim and Metric Hardware. This item is sold complete w/ the Rim Hardware. Tire and Tube are still available for individual sale.</p> <p>F - Part #1144660 is No Longer Available. Use part #1178834 as a replacement. Sold complete w/ Grey Tire/Black Rim, Metric Hardware and Flat Free Gel Foam material. It is recommended to order in pairs because of the flat free foam density difference.</p> <p>G - 14" x 3" Tire w/ Soft Density Flat Free Gel foam (yellow) is not available for item #1 1178834 and 1167685. Customer can use the Tire w/ Hard Density Flat Free foam (green) part #1141342. It is recommended to order in pairs to match.</p> <p>H - Used After 07/07/19. Wheel Assembly sold complete only. Drive Wheel Tire color changed from Grey to Black.</p>					