


Permanent Arm Frames and Hardware

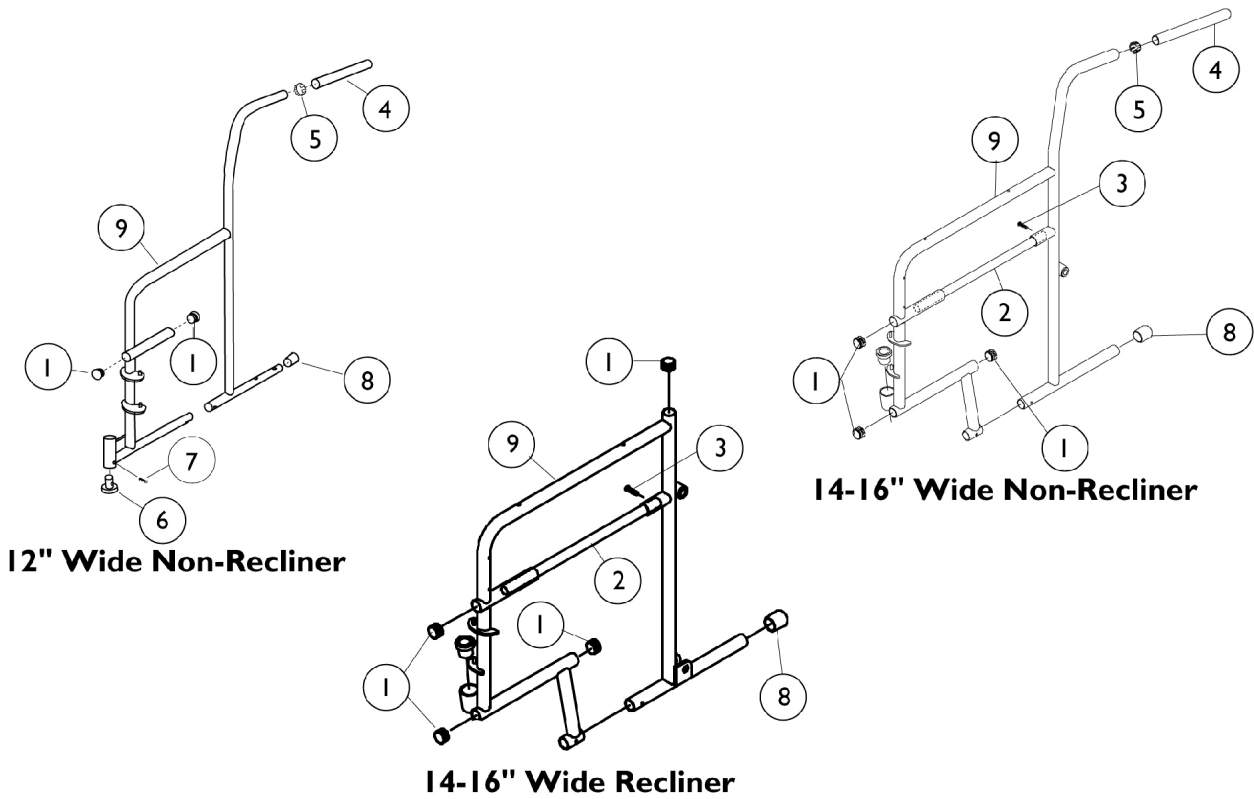

Permanent Arm Frames and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1027095	Plug Button, Black (7/8")	1	3
2		47453X019	Tube, Insert	1	1
3		21243X000-NLA	NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	1
4		1006229	Grip, Foam (8")	1	1
5		40144X000-NLA	NLA - Plug Button (7/8")	1	1
6		57768X020-NLA	NLA - Sleeve, Caster	1	1
6		1016682	Bearing, Sleeve Caster	1	1
7		40121X000-NLA	NLA - Screw, Phillips Pan Head (#6-32 x 3/16")	1	1
8	B	15680X012	Collar, Anti-Rattle (7/8")	1	1
9		1015878-NLA	NLA - Frame w/o Hardware, Chrome - Right	1	1
9		1015879-NLA	NLA - Frame w/o Hardware, Chrome - Left	1	1
9		1015876-NLA	NLA - Frame w/o Hardware, Chrome - Right	1	1
9		1015877-NLA	NLA - Frame w/o Hardware, Chrome - Left	1	1
9		1016459-NLA	NLA - Frame w/o Hardware - Right	1	1
9		1016460-NLA	NLA - Frame w/o Hardware - Left	1	1
NOTE: A - Due to product improvement when replacing 57668X020 sleeve, a pair of no. 1016682 new Caster sleeves should be ordered. B - 12" Wide, Non-Recliner C - 14-16" Wide, Non-Recliner D - 14-16" Wide, Recliner					