


Mattress Support, Head End, w/ Auto Regression

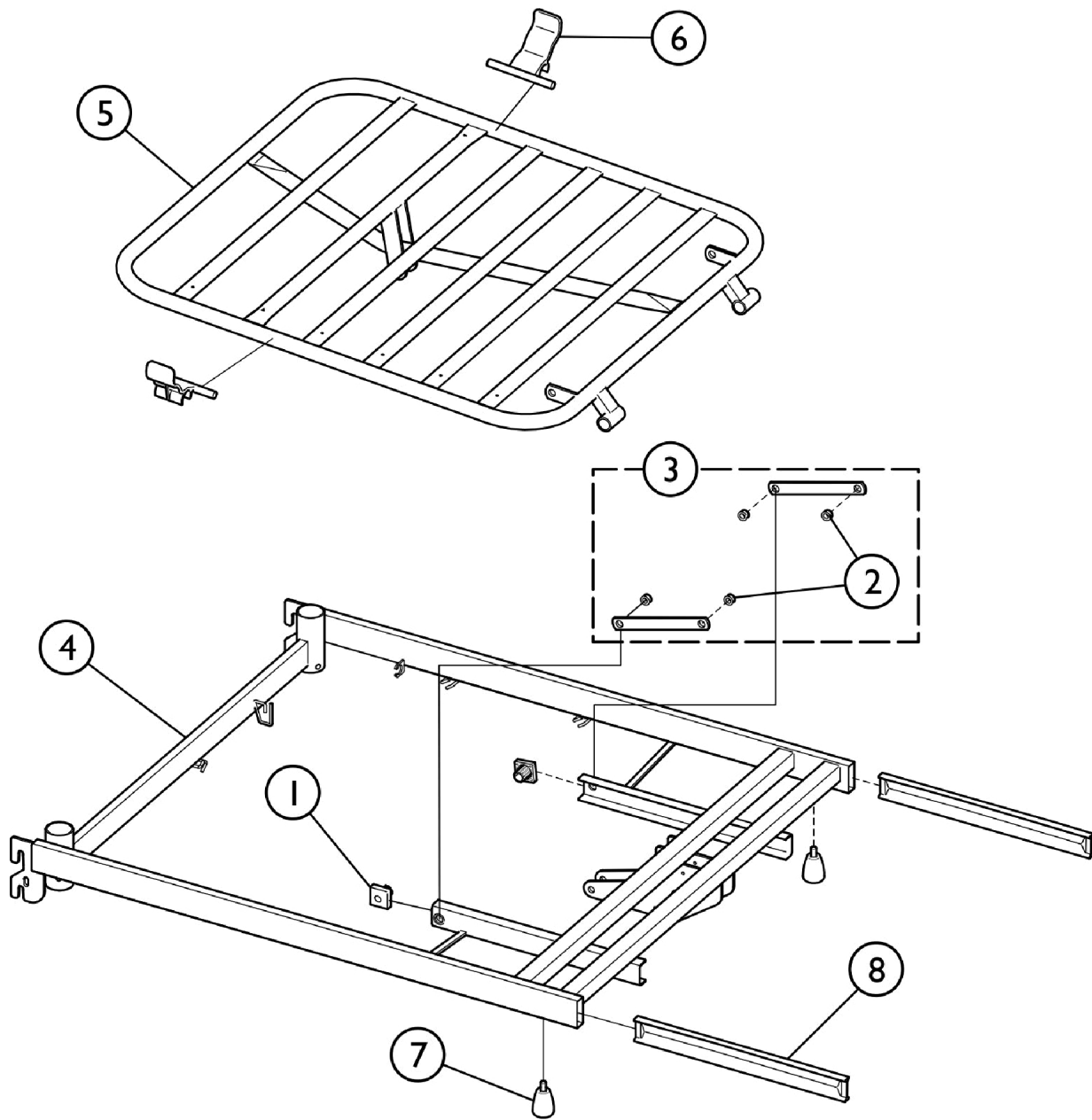

SBP_064A

Mattress Support, Head End, w/ Auto Regression

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|---------------|--------------------------------|--------------|----------|
| | | | | Package | Assembly |
| 1 | | SP319155-7035 | Slide Shoe, Grey (Pair) | 2 | 1 |
| 2 | | SP1435195 | Kit, Rivet Replacement | 1 | 1 |
| 3 | | SP1587778 | Kit, Tension Rod | 1 | 1 |
| 4 | | SP1587784 | Top Frame, Head End | 1 | 1 |
| 5 | | SP1587785 | Backrest | 1 | 1 |
| 6 | | SP1574230 | Mattress Holder (Pair) | 2 | 1 |
| 7 | | SP1584437 | Screw, Finger (M8 x 13) (Pair) | 2 | 1 |
| 8 | | SP1584419 | Insert Tubes (Pair) | 2 | 1 |