


Bed End

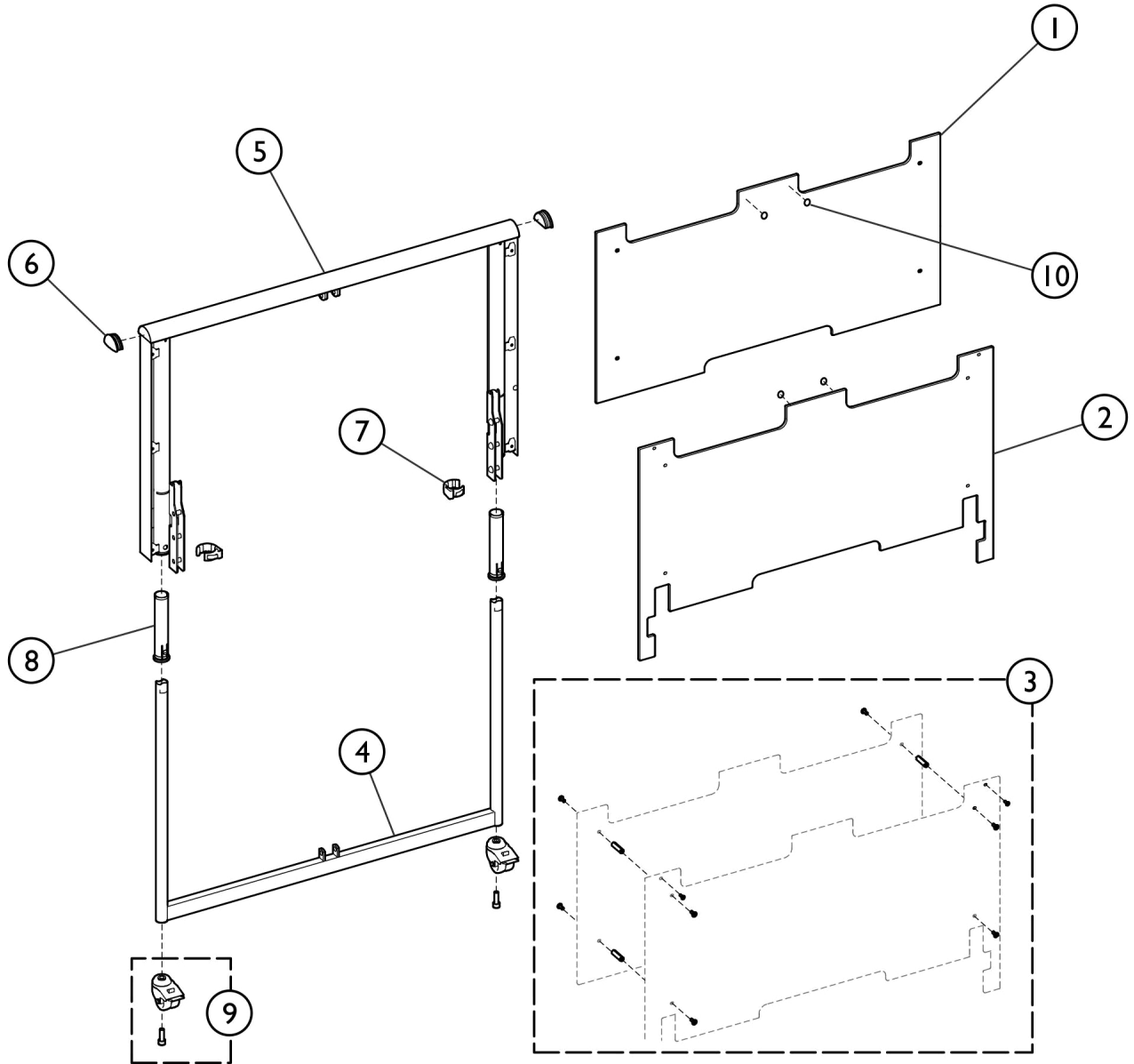

SBP_074A

Bed End

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP1580848	Outer Plate, Bed End	1	1
2		SP1580849	Inner Plate, Bed End	1	1
3		SP1177554	Kit, Cover Fixation, Wood Cover	1	1
			Screw, Furniture (M6 x 15)	1	8
			Standoff (M6 x 40 mm)	1	4
4		SP1587789	Bed End, Lower Part	1	1
5		SP1587790	Bed End, Upper Part	1	1
6		SP1441400-703 5	End Plug (Each)	1	2
7		SP418443-7035	Locking Ring (Pair)	2	1
8		SP1563579	Kit, Plastic Tube	1	1
			Plastic Tube	1	4
9		SP1177548	Kit, Caster Assembly w/ Hardware (50mm)	1	1
			Caster Assembly, Low (50mm)	1	2
			Screw (M10 x 30mm)	1	2
10		SP806775	Rubber Pad (4 ea.)	4	1