



Crossbraces and Hardware

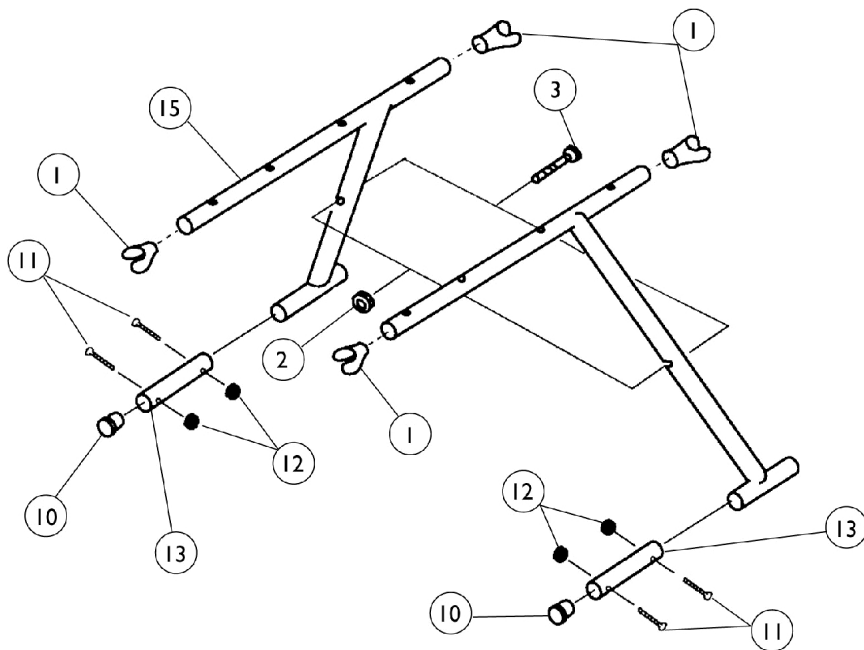

Permanent Arm Crossbrace Hardware

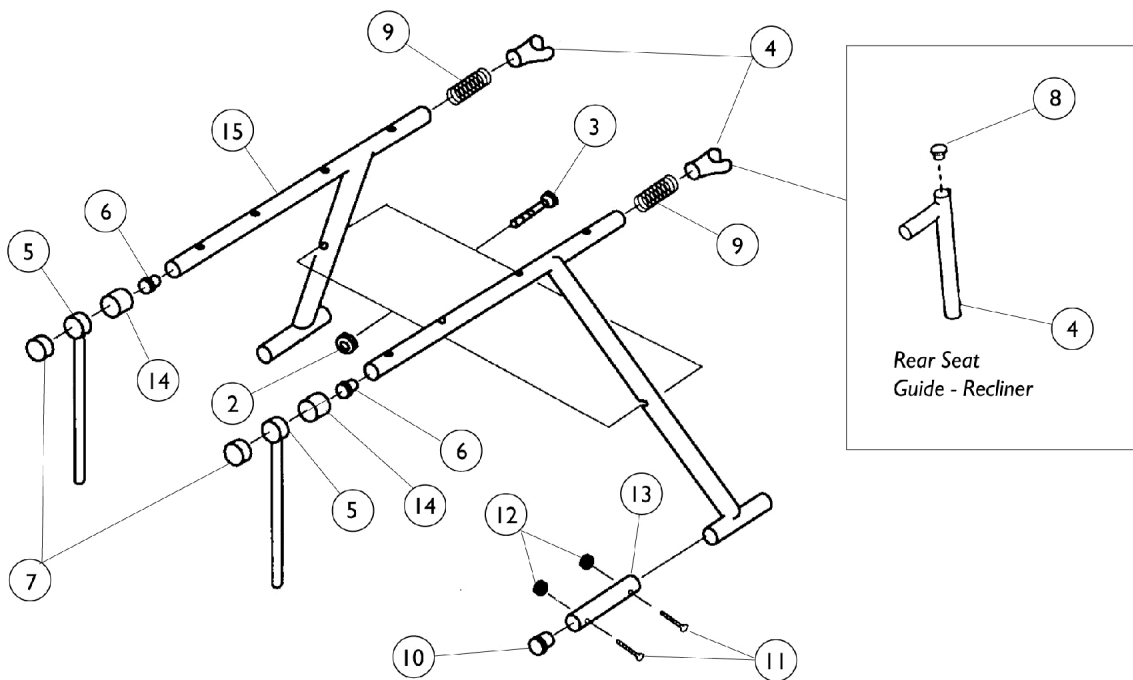

Removable Arm Crossbrace Hardware

Crossbraces and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1058416	Guide, Seat, Front, Black	1	4
2		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
3		40317X001	Screw, Hex Head (5/16-24 x 2-3/4")	1	1
4	A	04723X003-NLA	NLA - Guide, Seat, Chrome	1	2
4	B	04555X006-NLA	NLA - Guide, Seat - Rear (After 5/89)	1	2
5	C	04555X004-NLA	NLA - Guide, Seat - Front (12" W)	1	2
5	D	04555X007-NLA	NLA - Guide, Seat, Chrome - Front (After 5/89)	1	2
6		1027095	Plug Button, Black (7/8")	1	2
7	C	47397X024	Spacer (7/8 x 1 x 7/16")	1	2
7	D	47397X022	Spacer (7/8 x 1 x 7/8")	1	2
8		40144X001-NLA	NLA - Plug Button, Chrome (3/4")	1	2
9		47236X001	Spring, Tension	1	2
10		1027097	Plug Button, Black (3/4")	1	2
11		7930361-NLA	NLA - Package, Screw, Phillips Round Head (#10-32 x 1-1/8")	25	4
12		7930365-NLA	NLA - Locknut (#10-32)	25	4
13		42301X011-NLA	NLA - Tube, Insert	1	2
14	E	47397X020	Spacer (7/8 x 1 x 1-3/8")	1	2
15	C	04354X038-NLA	NLA - Crossbrace w/o Hardware - Right (12" W)	1	1
15	C	04354X039-NLA	NLA - Crossbrace w/o Hardware - Left (12" W)	1	1
15	D	04354X048-NLA	NLA - Crossbrace w/o Hardware - Right (14 - 16" W)	1	1
15	D	04354X049-NLA	NLA - Crossbrace w/o Hardware - Left (14 - 16" W)	1	1
NOTE: A - Non-Recliner B - Recliner C - 12" Wide Chairs D - 14-16" Wide Chairs E - 12" Wide Chairs Only					