


Aquatec R/SRB - Reclining Back Bath Lift

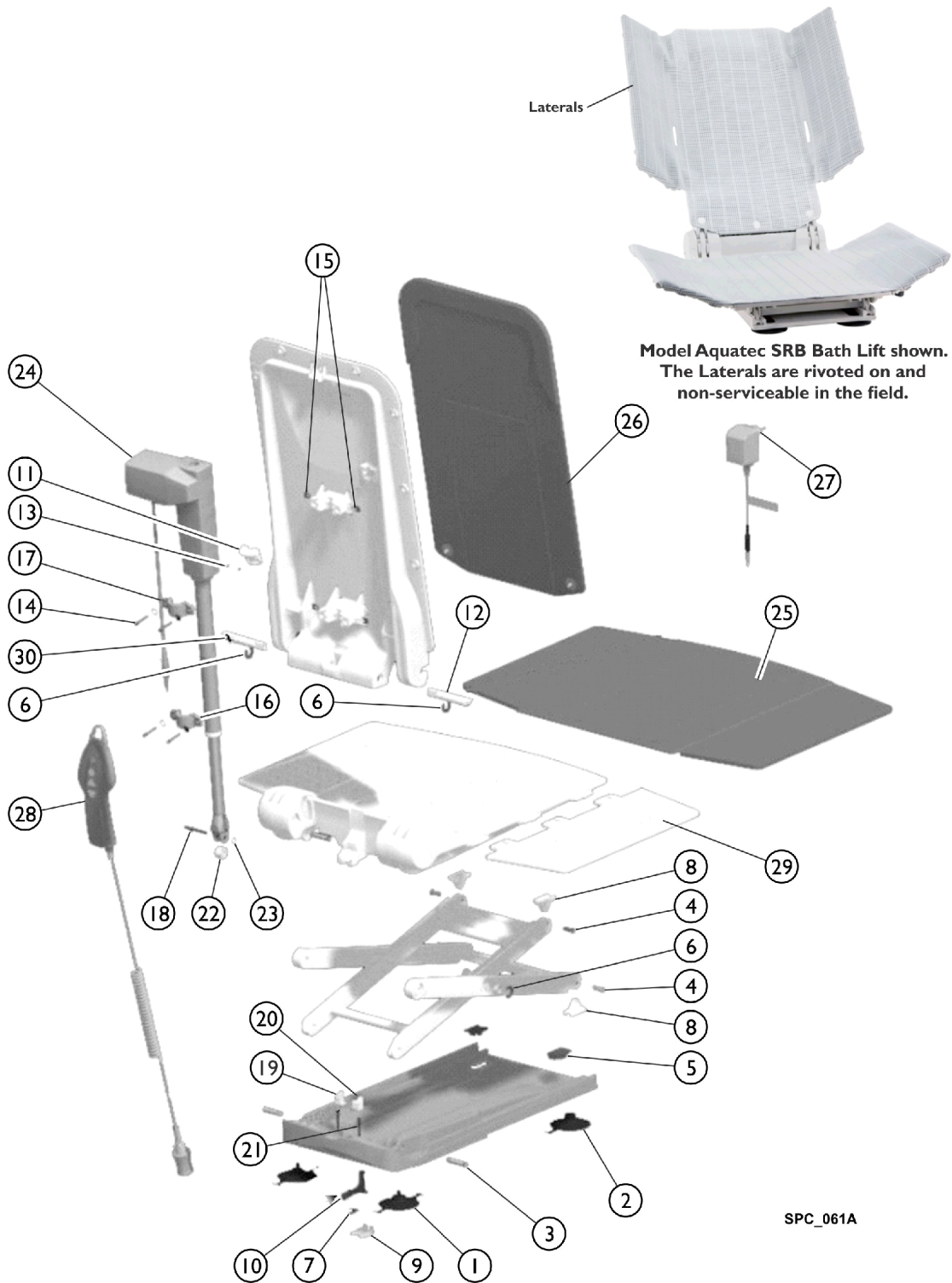

SPC_061A

Aquatec R/SRB - Reclining Back Bath Lift

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1+2		15353	Cups, Suction, Screw-In and Toggle (Set) (8 cm)	4	1
3		16679	Bolt (10mm)	1	4
4		15105	Bolt for Gliding Shoe	1	4
5		15121	Cap for Suction Cup	1	2
6		15344	Washer, Securing	1	2
7		15415	Spring	1	1
8		16446	Shoe, Gliding	1	4
9		16448	Latch, Bushing	1	1
10		16449	Latch	1	1
11		16455	Holding Device (Excluding Screws)	1	1
12		1470579	Bolt, Right	1	1
13		15348	Screws, Phone Jack (40 x 22)	1	2
14		16781	Lens Head Screw DIN 7985, M5 x 30	1	4
15		16779	Nut for Actuator Fastening	1	4
16		16778	Clamp, Pipe, Actuator Fastening	1	2
17		16780	Washer A2 DIN 9021	1	4
18		16793	Axle	1	1
19		16791	Locking - Left	1	1
20		16792	Locking - Right	1	1
21		16794	Pressure Spring for Locking	1	2
22		16450	Pulley	1	1
23		16456	Ring, Retaining, A2	1	2
24		16782	Actuator, Aquatec R	1	1
25		16606	Cover, Seat, White	1	1
26		16604	Cover, Backrest, White	1	1
27		16580	Charger, Battery	1	1
28		1470902	Hand Control, Complete	1	1
29		15892	Flap, Side with Film Hinge	1	2
30		1470580	Bolt, Left	1	1
		1470576	Cover Set, Seat and Backrest, White	1	1
		1470767	Cover Set, Seat and Backrest, Blue	1	1
		1470766	Cover, Backrest, Blue	1	1
		1471567	Cover, Backrest w/ Laterals SPRS, Blue	1	1
		1470765	Cover, Seat, Blue	1	1
		16440	Backrest, Frame	1	1
		1471606-NLA	NLA - Sideflap, Aquatec	1	1