


Reclining Back Hardware

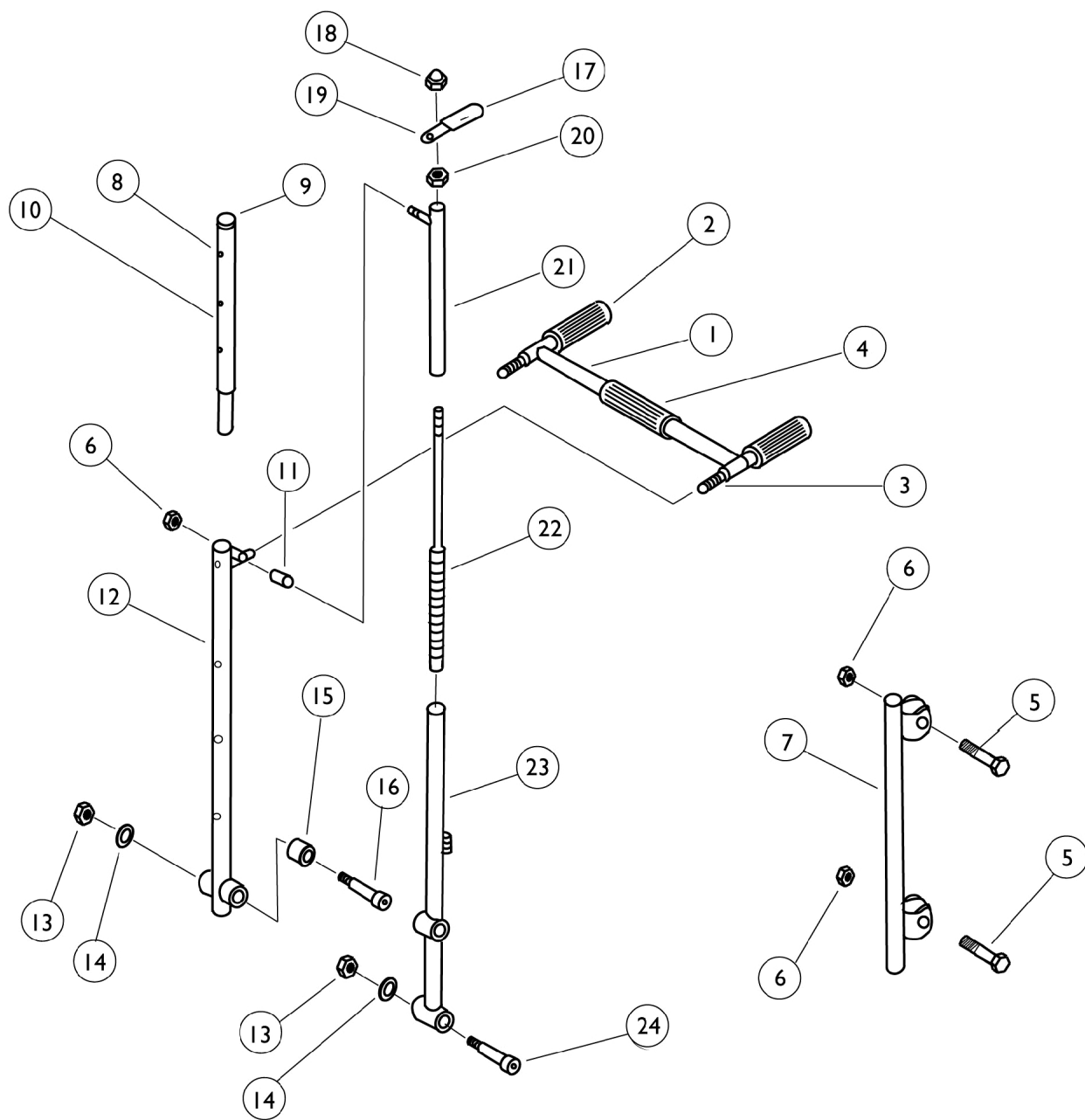

Reclining Back Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		04429X009-NLA	NLA - Tube, Strut (14" W)	1	1
1		04419X015-NLA	NLA - Bar, Adjustable Pivot, Spreader (Pair)	1	1
1		40197X000-NLA	NLA - Pin, Roll (3/16 x 7/8")	1	1
1		47340X003-NLA	NLA - Handle	1	1
2	A	40299X010-NLA	NLA - Grip, Hand, Grey	1	2
2	B	1029096-NLA	NLA - Grip, Hand, Black	1	2
3	A	16812X000-NLA	NLA - Grip, Hand, Center, Grey - NLA	1	1
3	B	1029099-NLA	NLA - Grip, Hand, Center	1	1
4		47340X006-NLA	NLA - Handle Tie Rod	1	1
5		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	2
6		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	3
7		04562X016-NLA	NLA - Tube, Back Support (14" Wide Chair)	1	1
8		47447X009-NLA	NLA - Screw, Phillips (#8 x 3/8")	1	3
9		1027095	Plug Button, Black (7/8")	1	1
10		1011725-NLA	NLA - Headrest Extension Tube Assembly	1	1
10		1054610-NLA	NLA - Headrest Extension Tube w/o Snap Fasteners Assembly	1	1
11		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
12		04562X003-NLA	NLA - Tube, Reclining Back - Right/Left	1	1
13		40363X000	Hex Nut (3/8"-16)	1	2
14		16325X000-NLA	NLA - Washer (3/8 x 13/16 x 1/16")	1	2
15		47324X004-NLA	NLA - Spacer (9/16 x 1 x 5/16")	1	1
16		60282X004-NLA	NLA - Bolt, Shoulder (1/2 x 2")	1	1
17		47376X025-NLA	NLA - Tip, Plastic, Black	1	1
18		1012386-NLA	NLA - Nut, Acorn (7/16-20)	1	1
19		47557X003-NLA	NLA - Handle, Locking	1	1
20		7930352-NLA	NLA Locknut (7/16-20)	25	1
21		04610X006-NLA	NLA - Tube, Slide, Chrome	1	1
22		04610X007-NLA	NLA - Rod, Locking	1	1
23		1015874-NLA	NLA - Tube, Ratchet, Chrome - Right	1	1
23		1015875-NLA	NLA - Tube, Ratchet, Chrome - Left	1	1
24		60282X003-NLA	NLA - Bolt, Shoulder (1/2 x 1-1/4")	1	1

NOTE: A - Before 6/93
B - After 5/93