


Ultra Low Maxx-"STANDARD" - Seat Tilt/Seat Tilt w/ Elevate, Main System Mounting and Interface Hardware (ULMT/ ULMTR/AELM)

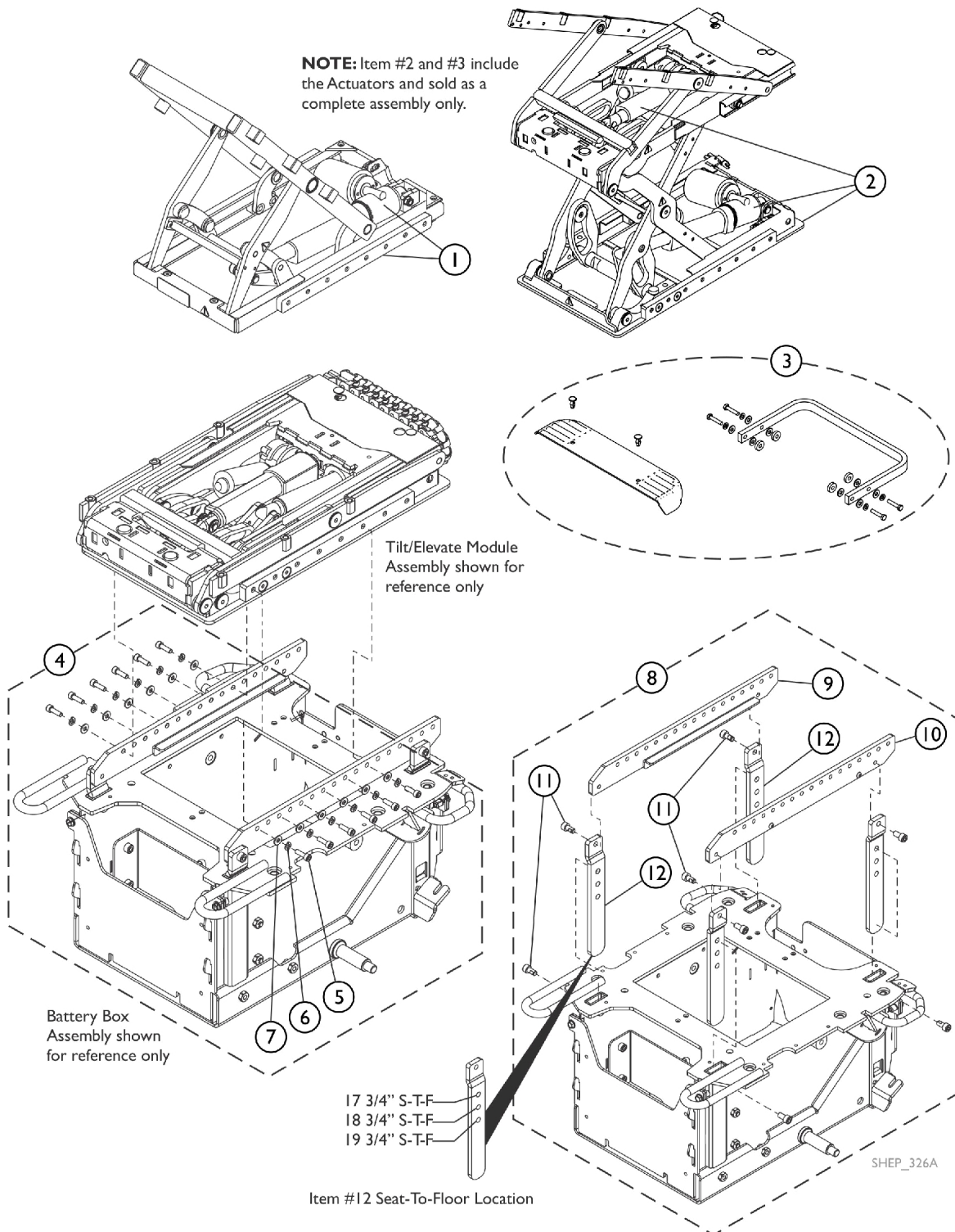

Ultra Low Maxx-"STANDARD" - Seat Tilt/Seat Tilt w/ Elevate, Main System Mounting and Interface Hardware (ULMT/ ULMTR/AELM)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60102606	Kit, Tilt Module Assembly w/ Actuator	1	1
2	A	60103390	Kit, Tilt/Elevate Module Assembly w/ Actuators	1	1
3		60102635	Kit, Bumper and Rear Wiring Cover w/ Mounting Hardware, Elevate	1	1
4	B	60124363	Kit, Seat Mounting Hardware, Standard	1	1
5			Screw, Socket Head Cap (M6 x 1 x 20mm)	1	12
6			Washer, Split Lock, Black Zinc (M6)	1	12
7			Washer, Plain, Black Zinc (1/4 x 9/16 x 1/16")	1	12
			Sheet, Insert Parts Service	1	1
8	C	60124361	Kit, Seat Mounting Bracket Hardware, Standard	1	1
9			Seat Rail Mounting Bracket - Right	1	1
10			Seat Rail Mounting Bracket - Left	1	1
11			Screw, Socket Head Cap w/ Patch (M8 x 1.25 x 16mm)	1	8
12			Bracket, Vertical Seat Mounting	1	4
			Sheet, Insert Parts Service	1	1
NOTE: A - Kit is sold as a complete assembly only. B - Includes items 5-7 C - Includes items 9-12					