


MicroAIR 800 Mattress

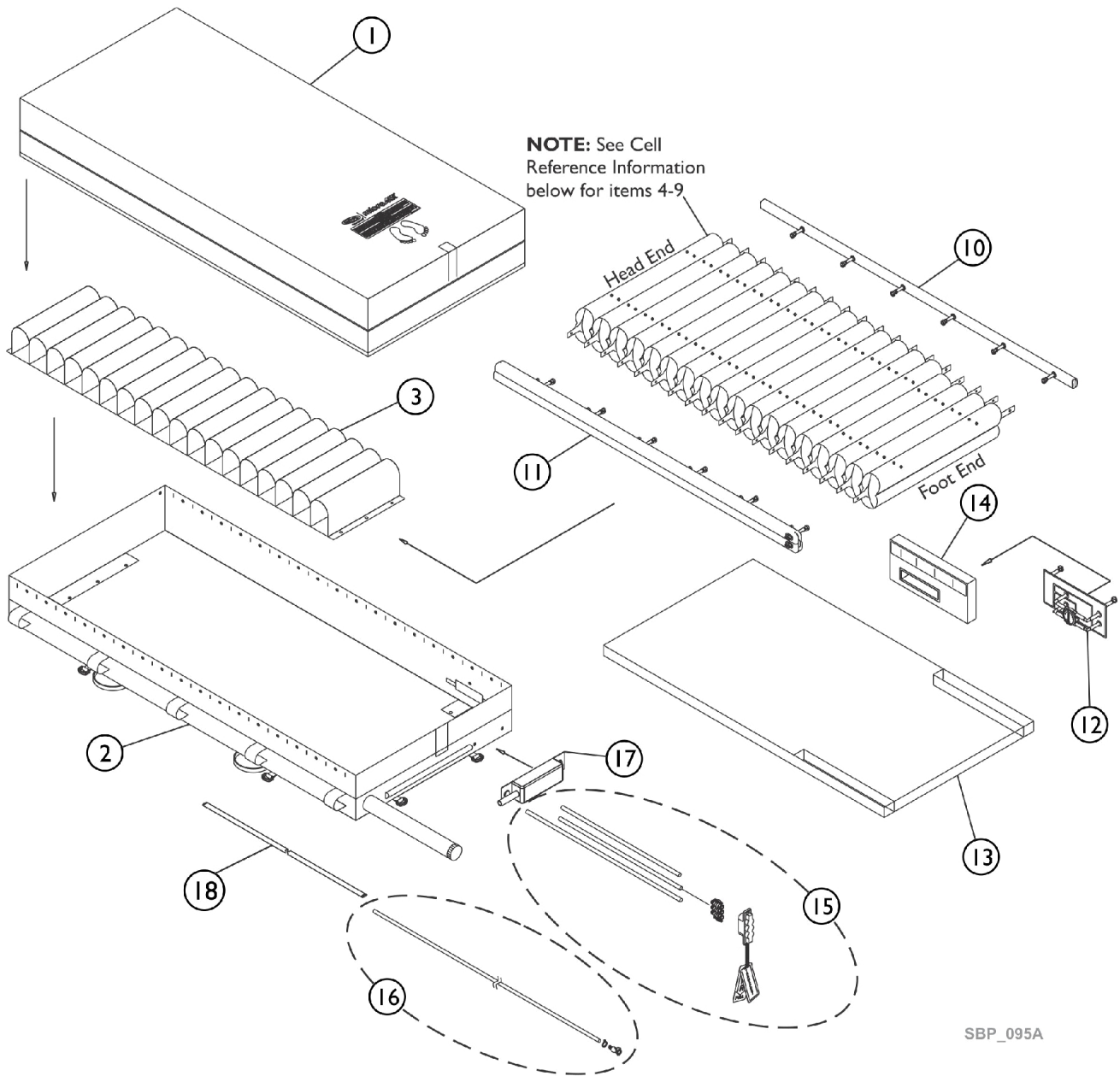

SBP_095A

Cell Position	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18
Item Number	4	5	6	4	5	6	4	5	6	4	5	6	4	5	9	7	8	9

← Head End - Cell Reference Information - Foot End →

MicroAIR 800 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124641	Top Cover	1	1
2		60124642	Bottom Cover	1	1
3		60124622	Belt Set	1	1
4		60124624	8" Vent Cell (Blue)	1	5
5		60124625	8" Vent Cell (White)	1	5
6		60124626	8" Vent Cell (Green)	1	4
7		60124627	8" Foot Vent Cell (Blue)	1	1
8		60124628	8" Foot Vent Cell (White)	1	1
9		60124629	8" Foot Vent Cell (Green)	1	2
10		60124633	Left Side Air Manifold	1	1
11		60124634	Right Side Air Manifold	1	1
12		60124635	Foot Side Rail For (14-1/2" x 7")	1	1
13		60124630	Foam Cover	1	1
14		60124631	Foot Foam Cover	1	1
15		60124671	Connecting Tube Set (Alternating Pipe Set)	1	1
16		60124621	Connecting Tube Set (Exhausting Pipe)	1	1
17		60124632	Vent/ Muffler Exhaust	1	1
18		60124623	Hose Sleeve	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124184	Exhaust O-Ring	1	1
	A	60124187	CPC Exhaust Connector (MA800)	1	1
	A	60124190	Heel Sense valve sticker	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124638	CPC Connector, 3/8" Male	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1

NOTE: A - Not Shown