


MicroAIR 800M42 Mattress

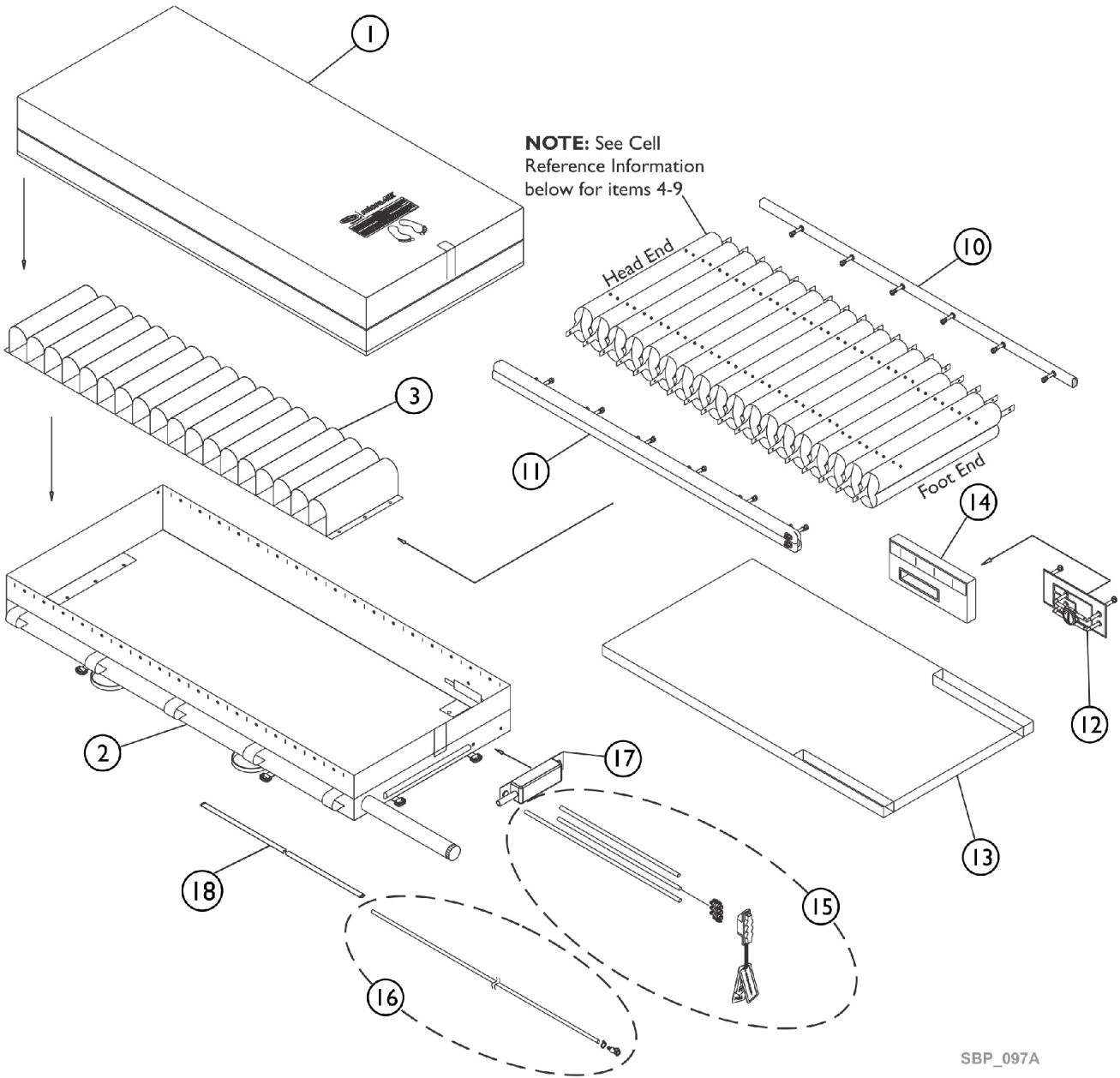

SBP_097A

| | | | | | | | | | | | | | | | | | | |
|---------------|---|---|---|---|---|---|---|---|---|----|----|----|----|----|----|----|----|----|
| Cell Position | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 10 | 11 | 12 | 13 | 14 | 15 | 16 | 17 | 18 |
| Item Number | 4 | 5 | 6 | 4 | 5 | 6 | 4 | 5 | 6 | 4 | 5 | 6 | 4 | 5 | 9 | 7 | 8 | 9 |

← Head End - Cell Reference Information - Foot End →

MicroAIR 800M42 Mattress

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|--|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 60124687 | Top Cover | 1 | 1 |
| 2 | | 60124688 | Bottom Cover | 1 | 1 |
| 3 | | 60124622 | Belt Set | 1 | 1 |
| 4 | | 60124674 | 8" Vent Cell for 42" Mattress (Blue) | 1 | 5 |
| 5 | | 60124675 | 8" Vent Cell for 42" Mattress (Clear) | 1 | 5 |
| 6 | | 60124676 | 8" Vent Cell for 42" Mattress (Green) | 1 | 4 |
| 7 | | 60124677 | 8" Foot Vent Cell for 42" Mattress (Blue) | 1 | 1 |
| 8 | | 60124678 | 8" Foot Vent Cell for 42" Mattress (Clear) | 1 | 1 |
| 9 | | 60124679 | 8" Foot Vent Cell for 42" Mattress (Green) | 1 | 2 |
| 10 | | 60124633 | Left Side Air Manifold | 1 | 1 |
| 11 | | 60124634 | Right Side Air Manifold | 1 | 1 |
| 12 | | 60124635 | Foot Side Rail For (14-1/2" x 7") | 1 | 1 |
| 13 | | 60124630 | Foam Cover | 1 | 1 |
| 14 | | 60124631 | Foot Foam Cover | 1 | 1 |
| 15 | | 60124620 | Connecting Tube Set (Alternating Pipe) | 1 | 1 |
| 16 | | 60124621 | Connecting Tube Set (Exhausting Pipe) | 1 | 1 |
| 17 | | 60124632 | Vent/ Muffler Exhaust | 1 | 1 |
| 18 | | 60124623 | Hose Sleeve | 1 | 1 |
| | A | 60124177 | 5A Fuse | 1 | 1 |
| | A | 60124178 | Air Filter - (MA800 / MA900) | 1 | 1 |
| | A | 60124180 | Fuse Holder | 1 | 1 |
| | A | 60124184 | Exhaust O-Ring | 1 | 1 |
| | A | 60124187 | CPC Exhaust Connector (MA800) | 1 | 1 |
| | A | 60124190 | Heel Sense valve sticker | 1 | 1 |
| | A | 60124192 | Rubber Foot, M10 Series (MA800 / MA900) | 1 | 1 |
| | A | 60124193 | Foam Bumper Strip, M10 Series (MA800 / MA900) | 1 | 1 |
| | A | 60124194 | Hook Set (MA800 / MA900) | 1 | 1 |
| | A | 60124636 | Trio Coupling Cap and Tag Set (MA800 / MA900) | 1 | 1 |
| | A | 60124637 | Trio Coupling Connector, Cap and Tag Set (MA800 / MA900) | 1 | 1 |
| | A | 60124638 | CPC Connector, 3/8" Male | 1 | 1 |
| | A | 60124639 | 1/2" Grey Tubing (MA800 / MA900) | 1 | 1 |
| | A | 60124640 | Trio Coupling (MA800 / MA900) | 1 | 1 |

NOTE: A - Not Shown