


# MicroAIR 800M4284 Mattress

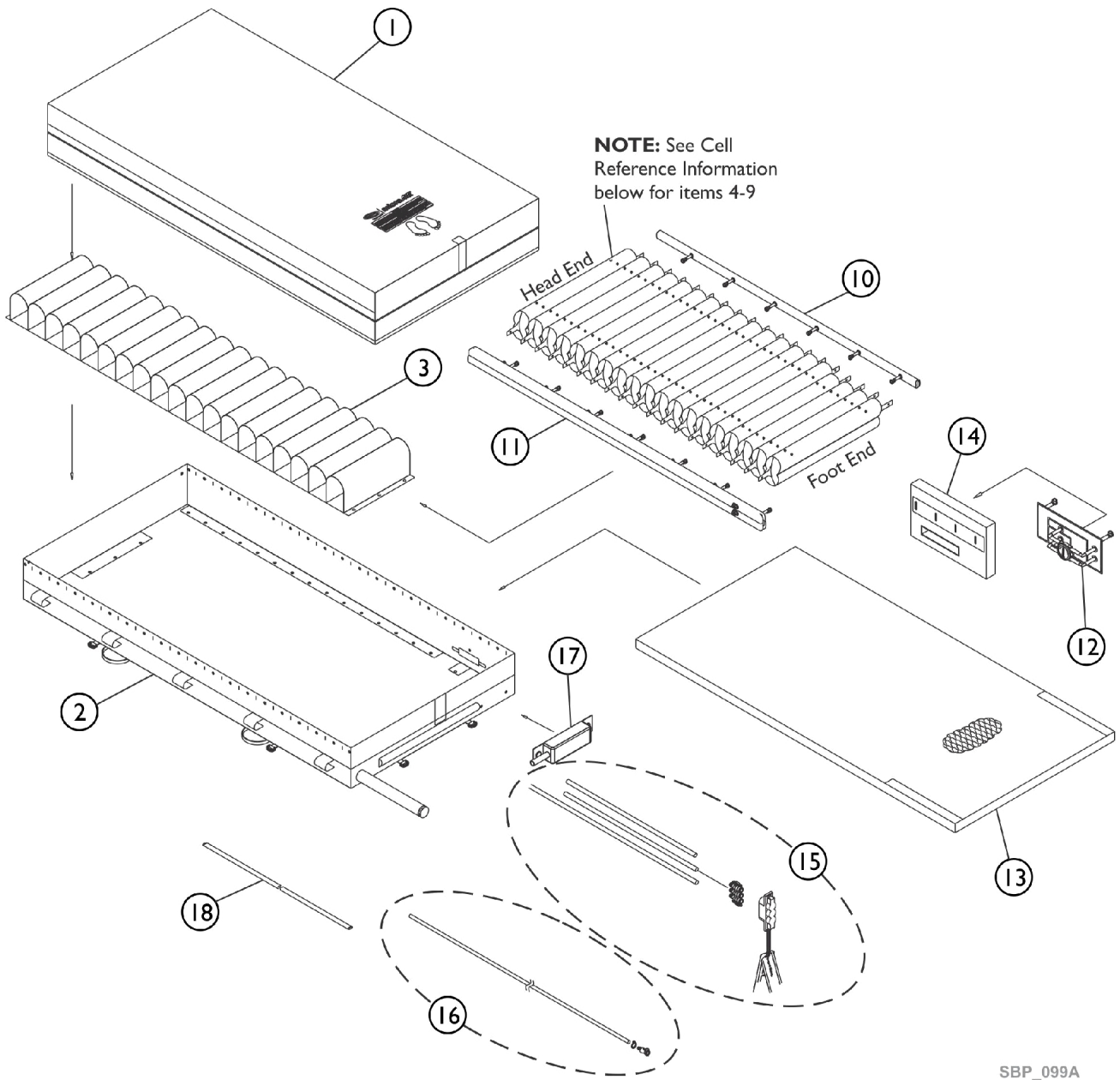

SBP\_099A

Cell Position	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19
Item Number	4	5	6	4	5	6	4	5	6	4	5	6	4	5	6	7	8	9	7

← Head End - Cell Reference Information - Foot End →

## MicroAIR 800M4284 Mattress

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124772	Top Cover	1	1
3		60124622	Belt Set	1	1
4		60124674	8" Vent Cell for 42" Mattress (Blue)	1	5
5		60124675	8" Vent Cell for 42" Mattress (Clear)	1	5
6		60124676	8" Vent Cell for 42" Mattress (Green)	1	5
7		60124677	8" Foot Vent Cell for 42" Mattress (Blue)	1	2
8		60124678	8" Foot Vent Cell for 42" Mattress (Clear)	1	1
9		60124679	8" Foot Vent Cell for 42" Mattress (Green)	1	1
10		60124749	Left Side Air Manifold	1	1
11		60124750	Right Side Air Manifold	1	1
12		60124635	Foot Side Rail For (14-1/2" x 7")	1	1
13		60124680	Foam Cover	1	1
14		60124631	Foot Foam Cover	1	1
15		60124671	Connecting Tube Set (Alternating Pipe Set)	1	1
16		60124621	Connecting Tube Set (Exhausting Pipe)	1	1
17		60124632	Vent/ Muffler Exhaust	1	1
18		60124623	Hose Sleeve	1	1
	A	60124177	5A Fuse	1	1
	A	60124178	Air Filter - (MA800 / MA900)	1	1
	A	60124180	Fuse Holder	1	1
	A	60124184	Exhaust O-Ring	1	1
	A	60124187	CPC Exhaust Connector (MA800)	1	1
	A	60124190	Heel Sense valve sticker	1	1
	A	60124192	Rubber Foot, M10 Series (MA800 / MA900)	1	1
	A	60124193	Foam Bumper Strip, M10 Series (MA800 / MA900)	1	1
	A	60124194	Hook Set (MA800 / MA900)	1	1
	A	60124636	Trio Coupling Cap and Tag Set (MA800 / MA900)	1	1
	A	60124637	Trio Coupling Connector, Cap and Tag Set (MA800 / MA900)	1	1
	A	60124638	CPC Connector, 3/8" Male	1	1
	A	60124639	1/2" Grey Tubing (MA800 / MA900)	1	1
	A	60124640	Trio Coupling (MA800 / MA900)	1	1

NOTE: A - Not Shown

Item #2 Bottom Cover is non-serviceable.