


Battery Harness, Base, AVIVA FX

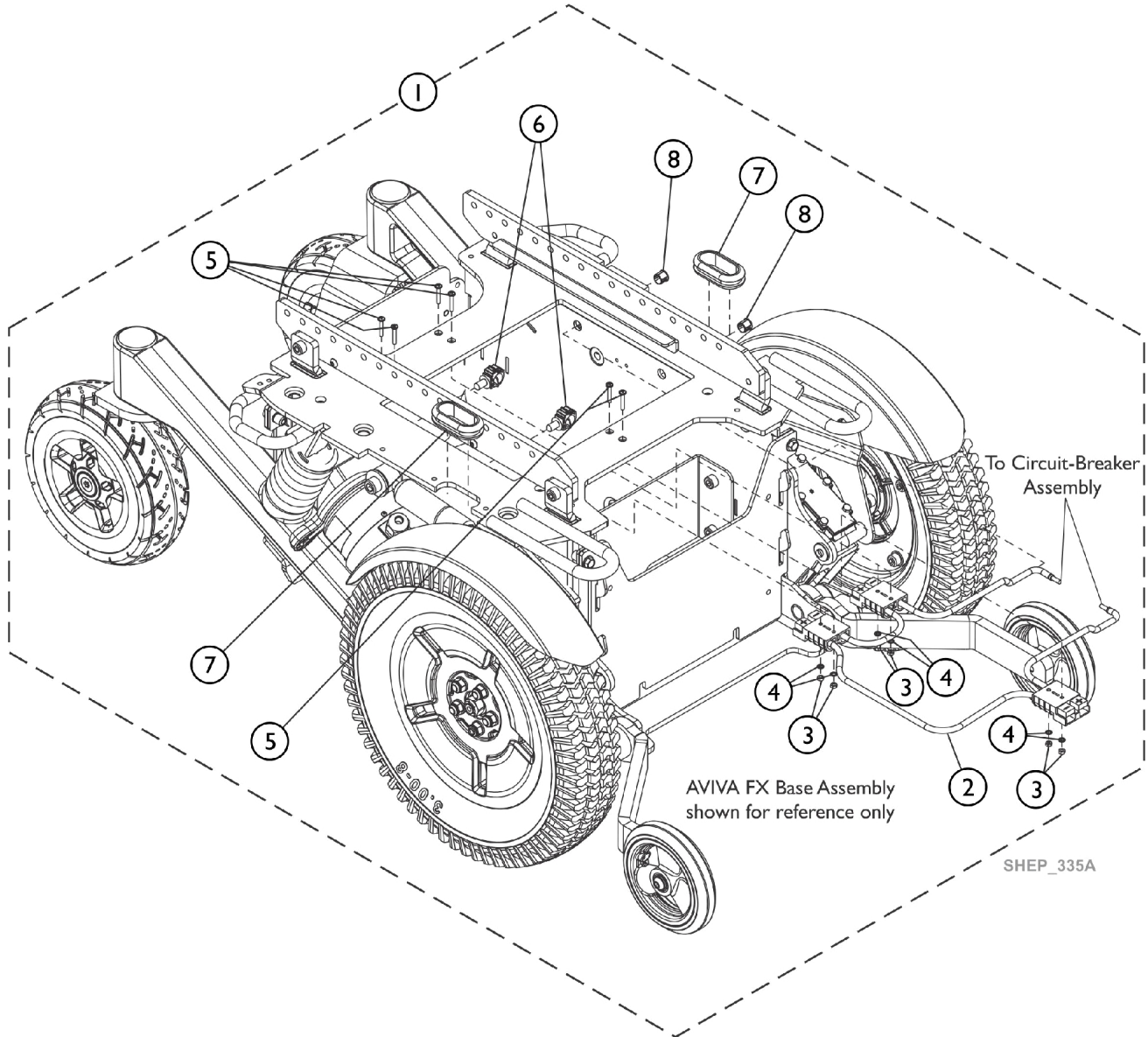

Battery Harness, Base, AVIVA FX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60124375	Kit, Battery Base Harness w/ Hardware	1	1
2			Harness, Base Battery	1	1
3			Nut, Hex (M3.5 x 0.6)	1	6
4			Washer, Lock Int-Tooth (M3.5)	1	6
5			Screw, Phillips Pan Head (M3.5 x 0.6 x 20mm)	1	6
6			Clip, Wire Locking (5-9mm)	1	2
7			Grommet, Rubber	1	2
8			Bushing, Open/Close Sheet, Insert Parts Service	1	2 1

NOTE: A - Includes items 2-8