


Caster - 8" Rear - Single Sided Fork, Nylon

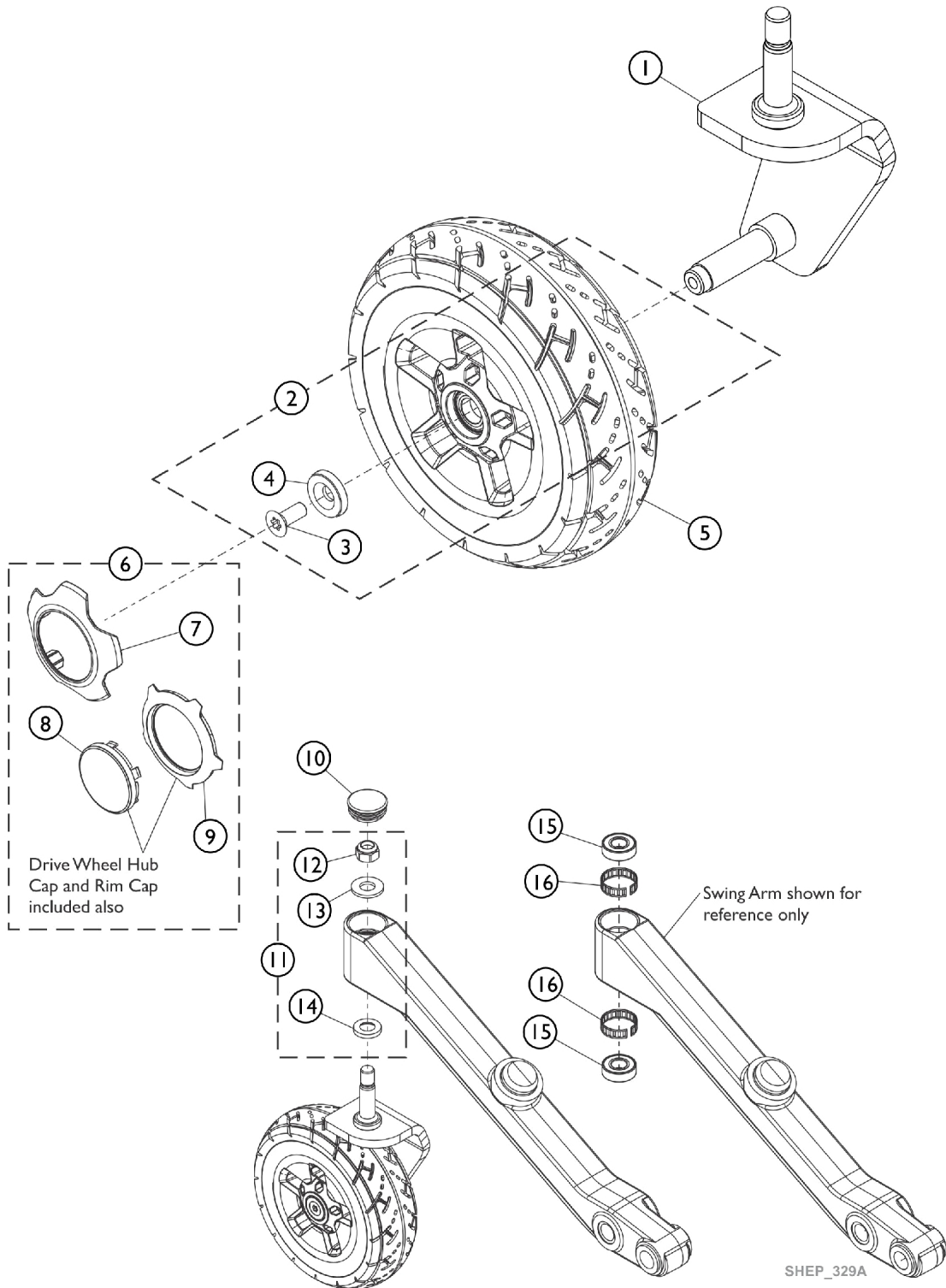

Caster - 8" Rear - Single Sided Fork, Nylon

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60124368	Kit, Single Sided Fork Assembly - Right	1	1
1		60124369	Kit, Single Sided Fork Assembly - Left	1	1
2	A	60124367	Kit, Caster Mounting Hardware	1	1
3			Screw, DIN7991SVZ TX PB (M8 x 20 x 12.9mm)	1	1
4			Spacer, Black (9 x 27 x 5.5mm)	1	1
5	B	60124366	Kit, Wheel Assembly, Flat Free, Black (8x2.50-4) Right and Left (Pair)	2	1
6	C	See Chart	Item #6 Rim Cap, Caster and Drive Wheel Kit, Nylon	1	1
10		1110709	Cap, Head Tube, Black	1	1
11	D	60124370	Kit, Fork Mounting Hardware	1	1
12			Locknut (M14 x 2mm)	1	1
13			Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
14			Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
15		40117X006	Bearing (5/8")	1	2
16		1110717	Ring, Tolerance	1	2

NOTE: A - Includes items 3 and 4.
 B - Right and Left side Wheel Assembly sold as pair only.
 C - Includes items 7-9.
 D - Includes items 12-16.

Item #6 Rim Cap, Caster and Drive Wheel Kit, Nylon

Description	Standard Black	Anthracite Black	Mercury Blue	Bengal Red	Silver Star	Midnight Purple
Rim Cap, Caster and Drive Wheel Kit	60124853-NLA See Note(s) A	60124854-NLA See Note(s) A	60124855-NLA See Note(s) A	60124856-NLA See Note(s) A	60124857-NLA See Note(s) A	60124858-NLA See Note(s) A

NOTE: A - Front Drive Wheel Center Rim Cap remains Black in color.
 Nylon Colors are No Longer Available. Select from Polycarbonate Colors.